# COLLANEX COLLAGEN PARTICLES - dressing, wound, collagen powder STRATUS PHARMACEUTICALS INC

-----

# COLLANEX™ Collagen Particles

Five Vials, 1g each

# Collagen Particles Triple Helical Matix

Conforms to FDA Quality System Requirements.

Manufactured as per cGMP standards.

#### **Product Information**

- This device is intended for single use only.
- All open and unused portions must be discarded.
- Sterility guaranteed in unopened, undamaged package only.
- Store in a cool and dry place.

## **Product Application**

- 1. Cleanse the wound.
- 2. Apply medication to the wound if indicated.
- 3. Apply Collanex™ Particles 1/4 inch thick to the wound surface. Do not pack tightly; allow for expansion of particles.
- 4. Cover with non-stick dressing.
- 5. Use Collanex<sup>™</sup> Particles until wound is fully healed.
- 6. Maintain a moist wound environment.

#### **Product Removal**

- 1. Change as indicated by the amount of drainage and instructions provided with the absorbent cover dressing.
- 2. After removal of the cover dressing, repeat steps under Product Application as needed.

#### **Precautions**

- Wounds may appear to be larger during the first several days of treatment due to the reduction of swelling.
- An increase in drainage may be seen in the first several days of treatment.
- The product is made from bovine materials. **Do not use if you have an allergy to bovine derived materials (milk, beef products, leather, etc.)**
- **Warnings:** Discontinue use of Collanex<sup>™</sup> Particles and notify your doctor if excessive redness, pain, swelling or blistering occurs.
- All open and unused portions must be discarded.
- Warnings: Potencial risk of loss of sterility and infection if opened and reused.

### **RX Only**

#### Distributed by:

Stratus Pharmaceuticals Inc. Miami, FL 33186

### Made in the USA

800-442-7882

STRATUS
PHARMACEUTICALS INC

Reorder 5898075405

Rev1, 062021

# PRINCIPAL DISPLAY PANEL - 1 g Vial Carton

COLLANEX™ Collagen Particles Five Vials, 1g each

Collagen Particles Triple Helical Matix

Conforms to FDA Quality System Requirements.

Manufactured as per cGMP standards.

#### Product Information:

- This device is intended for single use only.
- All open and unused portions must be discarded.
- Sterility guaranteed in unopened, undamaged package only.
- Store in a cool and dry place.

# RX Only

Five Vials, 1g each

STRATUS
PHARMACEUTICALS INC

Reorder 5898075405



Collagen Particles Triple Helical Matix

Conforms to FDA Quality System Requirements.

Manufactured as per cGMP standards.

#### Product Information:

- . This device is intended for single use only.
- · All open and unused portions must be discarded.
- · Sterility guaranteed in unopened, undamaged package only.
- . Store in a cool and dry place.

**RX Only** 

Five Vials, 1g each **STRATUS** 

**COLLANEX**™

Reorder 5898075405

# COLLANE

Collagen Particles Five Vials, 1g each

#### Product Application:

- 1. Cleanse the wound.
- Apply medication to the wound if indicated.
   Apply Collanex™ Particles 1/4 inch thick to the wound surface. Do not pack tightly; allow for expansion of particles.
- 4. Cover with non-stick dressing.
- Use Collanex™ Particles until wound is fully healed.
- 6. Maintain a moist wound environment.

#### Product Removal:

- 1. Change as indicated by the amount of drainage and instructions provided with the absorbent cover dressing.
- 2. After removal of the cover dressing, repeat steps under Product Application as needed.

- Wounds may appear to be larger during the first several days of treatment due to the reduction of swelling.
- · An increase in drainage may be seen in the first several days of treatment.
- The product is made from bovine materials. Do not use if you have an allergy to bovine derived materials (milk, beef products, leather, etc.)
- Warnings: Discontinue use of Collanex™ Particles and notify your doctor if excessive redness, pain, swelling or blistering
- All open and unused portions must be discarded.
- · Warnings: Potencial risk of loss of sterility and infection if opened and reused.

#### Distributed by:

Stratus Pharmaceuticals Inc. Miami, FL 33186

Made in the USA

STRATUS

Reorder 5898075405

### **COLLANEX COLLAGEN PARTICLES**

dressing, wound, collagen powder

Product Information			
Product Type	MEDICAL DEVICE	Item Code (Source)	NHRIC:58980-754
Route of Administration	TRANS DERMAL		

Active Ingredient/Active Moiety			
Ingredient Name	<b>Basis of Strength</b>	Strength	
BOVINE TYPE I COLLAGEN (UNII: FHJ3ATL51C) (BOVINE TYPE I COLLAGEN - UNII:FHJ3ATL51C)	BOVINE TYPE I COLLAGEN	1 g	

P	Packaging			
#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:58980- 754-05	5 in 1 BOX		
1		1 in 1 VIAL, SINGLE-DOSE; Type 0: Not a Combination Product		

Marketing Information			
Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
Premarket Notification	K910944	05/01/1991	

# Labeler - STRATUS PHARMACEUTICALS INC (789001641)

Establishment				
Name	Address	ID/FEI	Business Operations	
Human BioSciences Inc		014511541	MANUFACTURE, LABEL	

Revised: 11/2021 STRATUS PHARMACEUTICALS INC