

CITRACID-50- desinfectant solution for hemodialysis machine solution, concentrate
Aqua Medica, S.A. de C.V.

Disclaimer: This drug has not been found by FDA to be safe and effective, and this labeling has not been approved by FDA. For further information about unapproved drugs, click here.

This product is used to descale and disinfect Hemodialysis machines.

Citric Acid

For use with 3-pump hemodialysis machines only.

For use with 3-pump hemodialysis machines only, using purified water (Standard UNE-EN ISO 23500-3:2019).

Avoid contact with skin and eyes.

Irritation hazard if spilled or splashed.

If the warranty seal is damaged or broken and do not allow debris to fall into the concentrate.

Take appropriate security measures (protective clothing, glasses, masks).

Wash splashes on skin or clothing with plenty of water.

If do not use the entire contents, discard the excess.

Keep out of reach of children. If swallowed, get medical help or contact a Poison Control Center right away.

By diluting one part of this acid concentrate with one part of purified water.

Store at room temperature.

Keep container tightly closed when not in use.

Water.

5,0 liters

CITRACID[®]-50

Each 1,000 g contains:

Citric acid.....500,0 g
Purified water (standard UNE-EN ISO 23500-3:2019) ..500,0 g

Made in Mexico by:
AQUA Médica, S.A. de C.V.
Carr. Fed. Mex. Cuautla km. 65.8 No. 8,
Col. Tetelcingo, 62757 Cuautla, Mor.

INSTRUCTIONS FOR USE:
This product is used to descale and disinfect Hemodialysis machines.

WARNINGS:
Avoid contact with skin and eyes
Irritation hazard if spilled or splashed.

LOT No.:
Date of Expiry:

* Take appropriate security measures (protective clothing, glasses, masks).
* Wash splashes on skin or clothing with plenty of water.

NON STERILE PRODUCT

AQUA Médica, S.A. de C.V. IS NOT RESPONSIBLE FOR THE IMPROPER DISPOSAL OF THIS CONTAINER. DISPOSE OF THE SURPLUS OF THIS PRODUCT IN ACCORDANCE WITH THE SAFETY DATA SHEET.

REV. 13

CITRACID-50

desinfectant solution for hemodialysis machine solution, concentrate

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:81943-701
Route of Administration	HEMODIALYSIS		

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
CITRIC ACID MONOHYDRATE (UNII: 2968PHW8QP) (ANHYDROUS CITRIC ACID - UNII:XF417D3PSL)	ANHYDROUS CITRIC ACID	0.5 g in 100 mL

Inactive Ingredients

Ingredient Name	Strength
WATER (UNII: 059QF0KO0R)	0.5 g in 100 mL

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:81943-701-01	5000 mL in 1 CONTAINER; Type 0: Not a Combination Product	03/30/2023	

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
unapproved drug other		03/30/2023	

Labeler - Aqua Medica, S.A. de C.V. (589696442)

Establishment

Name	Address	ID/FEI	Business Operations
Aqua Medica, S.A. de C.V.		589696442	manufacture(81943-701)

Revised: 11/2025

Aqua Medica, S.A. de C.V.