


DRESSOL-X RADIOPAQUE D-502- aspirin dressing
Rainbow Specialty & Health Products, Inc.

A Non Absorbable Dental Packing Gauze With A.S.A. (Acetylsalicylic Acid) and Eugenol

For Dry Sockets: cut off a 2" strip and roll into toothpick shape. Pack firmly entire socket from the base up the walls. Cut away excess material. Finish plug with fingertip pressure. After 4-5 days, remove all components from socket. See insert for further usage.

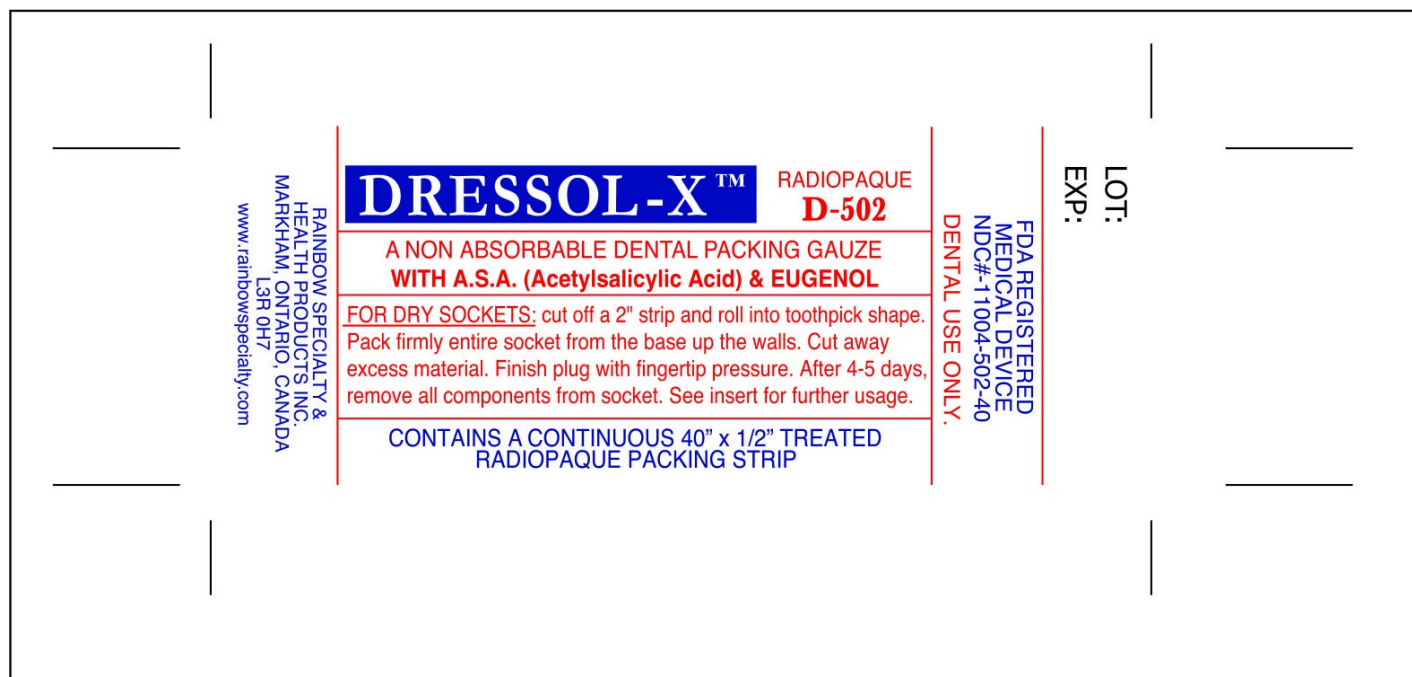

DRESSOL-X RADIOPAQUE D-502			
aspirin dressing			
Product Information			
Product Type	MEDICAL DEVICE	Item Code (Source)	NHRIC:11004-502
Route of Administration	DENTAL		
Active Ingredient/Active Moiety			
Ingredient Name		Basis of Strength	Strength
ASPIRIN (UNII: R16CO5Y76E) (ASPIRIN - UNII:R16CO5Y76E)		ASPIRIN	115 mg
EUGENOL (UNII: 3T8H1794QW) (EUGENOL - UNII:3T8H1794QW)		EUGENOL	80 mg
Inactive Ingredients			
Ingredient Name			Strength
PARAFFIN (UNII: I9O0E3H2ZE)			

MINERAL OIL (UNII: T5L8T28FGP)

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:11004-502-40	1 in 1 BOTTLE		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
Exempt device	GDY	03/31/2009	

Labeler - Rainbow Specialty & Health Products, Inc. (206962040)

Establishment

Name	Address	ID/FEI	Business Operations
Rainbow Specialty & Health Products, Inc.		206962040	manufacture

Revised: 12/2011

Rainbow Specialty & Health Products, Inc.