NEKVNRO SCAR REMOVAL SHEETS- scar removal sheets patch Shenzhen Xingqi Technology Co., Ltd.

ACTIVE INGREDIENT

Silicone gel 1%

Purpose

SCAR REMOVAL SHEETS

Uses

- 1.Clean and dry scar area.
- 2. Cut sheet according to the size and shape of the scar.
- 3. Peel off liner from adhesive side of sheet.
- 4. Apply adhesive side of sheet directly on scar. Apply 4 hours a day for the first two days, 8 hours a day for the next two days, and then increase by 2 hours a dayUntil you can apply 24 hours a day each scar sheet can be used repeatedly and when another new scar patch becomes difficult to clean or paste replace another new scar sheet.

WARNINGS

Use this product once scar or wound has been healing for at least 7 days. You may start using immediately on older scars. For old scars, the product can be used immediately.

Must not be used for not completely healed or infected- wounds and damagedoracneskin. If it's affixed to unclean skin or too tightly, it may cause rashes. At this time, it should be reduced to stick and stick for less time.

Dosage and administration

For external use only.

When using section

Use this product once scar or wound has been healing for at least 7 days. You may start using immediately on older scars. For old scars, the product can be used immediately.

Must not be used for not completely healed or infected-wounds and damagedoracneskin. If it's affixed to unclean skin or too tightly, it may cause rashes. At this time. it should be reduced to stick and stick for less time

stop use

on broken or irritated scalp

Keep out of reach of children.

Keep Away From Children Under 3 Years Old And Pets, Out Of Direct Sunlight And Store In A Cool, Dry Place

Inactive ingredients

polydimethylsiloxane, polyurethane film, pet.

PRINCIPAL DISPLAY PANEL

Nekvnro

Nekvnro

SCAR REMOVAL **SHEETS**

DERMATOLOGIST RECOMMENDED









- Mypertrophic Scar
- Post -Acne
- **M** Keloid
- ✓ C-Section
- **Y** Post-Surgery
- Stretch Marks

Nekvnro

INGREDIENTS

Silicone gel, polydimethylsiloxane, polyurethane film, pet.

Improve, heals and lighten old and new scars soften and inhibits proliferative scars and keloids reduce the burns. trauma, surgery caused by the scar area relieve skin itching. redness or irritation promote healthy collagen production and skin hydration.

DIRECTIONS

- Clean and dry scar area.
 Cut sheet according to the size and shape of the scar.
- 3. Peel off liner from adhesive side of sheet.
- 4. Apply adhesive side of sheet directly on scar. Apply 4 hours a day for the first two days, 8 hours a day for the next two days, and then increase by 2 hours a dayUntil you can apply 24 hours a day each scar sheet can be used repeatedly and when another new scar patch becomes difficult to clean or paste replace another new scar sheet.

Silicone Scar Removal Sheets

Use this product once scar or wound has been healing for at least 7 days. You may start using immediately on older scars. For old scars, the product can be used immediately.

Must not be used for not completely healed or infectedwounds and damagedoracneskin. If it's affixed to unclean skin or too tightly, it may cause rashes. At this time, it should be reduced to stick and stick for less time.

Distributor:

Shenzhen Xingqi Technology Co., Ltd. address: Room 1023, Building 3.Heshuikou Community, Matian Street, Guangming District Shenzhen, Guangdong, 518000 China



Keep Away From Children Under 3 Years Old And Pets, Out Of Direct Sunlight And Store In A Cool, Dry Place









MGKVNIO

NEKVNRO SCAR REMOVAL SHEETS

scar removal sheets patch

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:84613-058
Route of Administration	TOPICAL		

Active Ingredient/Active Moiety

- 10 111 g. 0 110 11 g. 10 11				
Ingredient Name	Basis of Strength	Strength		
SILICON DIOXIDE (UNII: ETJ7Z6XBU4) (SILICON DIOXIDE - UNII:ETJ7Z6XBU4)	SILICON DIOXIDE	1 g in 1 g		

Inactive Ingredients		
Ingredient Name	Strength	
POLYURETHANE-39 (UNII: J9PXX566KC)		
PEGOTERATE (UNII: JHW12Q65DD)		
DIMETHICONE (UNII: 92RU3N3Y1O)		

ı	Packaging				
	#	Item Code	Package Description	Marketing Start Date	Marketing End Date
	1	NDC:84613-058- 02	25 g in 1 PATCH; Type 0: Not a Combination Product	09/13/2024	

Marketing Information			
Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC Monograph Drug	505G(a)(3)	09/13/2024	

Labeler - Shenzhen Xingqi Technology Co., Ltd. (974493783)

Establishment				
Name	Address	ID/FEI	Business Operations	
Shenzhen Xingqi Technology Co., Ltd.		974493783	manufacture(84613-058)	

Revised: 9/2024 Shenzhen Xingqi Technology Co., Ltd.