

**REPARA - zinc oxide dressing**  
**PSS World Medical, Inc.**

*Disclaimer: Most OTC drugs are not reviewed and approved by FDA, however they may be marketed if they comply with applicable regulations and policies. FDA has not evaluated whether this product complies.*

-----

**Drug Facts**

**Active Ingredients**

Zinc Oxide 25%

**Purpose**

Skin Protectant

**Uses**

Dries the oozing and weeping of poison:

- ivy
- oak
- sumac

**Warning**

**For external use only.**

**When using this product**

- do not get into eyes

**Stop and ask a doctor if**

- Condition worsens
- symptoms last more than 7 days or clear up and occur again within a few days.

**Keep out of reach of children.**

If swallowed, get medical help or contact a Poison Control Center right away.

**Directions**

- Apply as needed
- Cleanse the leg and dry the skin.
- If a wound is present, apply an appropriate wound dressing for moist wound healing.
- With a foot positioned at 90° or right angle to the leg (dorsiflexed) begin wrapping the medicated bandage without tension beginning at the base of the toes. Wrap the leg without tension overlapping each layer by 50% or half the width of the dressing. Smooth and mold the bandage while wrapping.
- Wrap all the way up the leg stopping one finger width below the posterior knee.
- Apply an elastic adhesive bandage (not included), following the manufacturer's instructions.

**Other Ingredients**

Acacia, Castor oil, Glycerin, and White Petrolatum.

## **Principal Display Panel (Front and Back)**

NDC # 68345-270-01

### **REPARA®**

3" Unna Boot

- 3" x 10 yds.
- Non-hardening zinc oxide formulation
- Conforms easily to foot or leg
- Provides moist healing environment

Questions or Comments about Repara  
Products? 800-777-4908

Manufactured for PSS World Medical Inc.  
4345 Southpoint Blvd., Jacksonville, FL 32216  
Made in Mexico  
[www.myselectonline.com](http://www.myselectonline.com)

1 each/box

Reorder # 2070

### **Directions for Use**

1. Cleanse the leg and dry the skin.
2. If a wound is present, apply an appropriate wound dressing for moist wound healing.
3. With the foot positioned at a 90 or right angle to the leg (dorsiflexed) begin wrapping the medicated bandage without tension beginning at the base of the toes. Wrap the leg without tension overlapping each layer by 50% or half the width of the dressing. Smooth and mold the bandage while wrapping.
4. Wrap all the way up the leg stopping one finger width below the posterior knee.
5. Apply an adhesive bandage (not included), following the manufacturer's instructions




**Directions for Use**


1. Cleanse the leg and dry the skin.
2. If a wound is present, apply an appropriate wound dressing for moist wound healing.
3. With the foot positioned at a 90° or right angle to the leg (dorsiflexed) begin wrapping the medicated bandage without tension beginning at the base of the toes. Wrap the leg without tension overlapping each layer by 50% or half the width of the dressing. Smooth and mold the bandage while wrapping.



4. Wrap all the way up the leg stopping one finger width below the posterior knee.

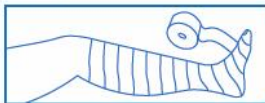

5. Apply an elastic adhesive bandage (not included), following the manufacturer's instructions.

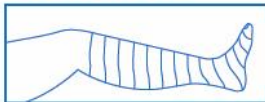

NDC 68345-270-01

**REPARA®**

**3" Unna Boot**

Questions or Comments about Repara Products? 800-777-4908  
 Manufactured for PSS World Medical, Inc.  
 4345 Southpoint Blvd., Jacksonville, FL 32216  
 Made in Mexico • www.myselectonline.com

**REPARA®**

**3" Unna Boot**

- 3" x 10 yds.
- Non-hardening zinc oxide formulation
- Conforms easily to foot or leg
- Provides moist healing environment

Questions or Comments about Repara Products? 800-777-4908  
 Manufactured for PSS World Medical, Inc.  
 4345 Southpoint Blvd., Jacksonville, FL 32216  
 Made in Mexico • www.myselectonline.com

Drug Facts	
Active Ingredients	Purpose
Zinc Oxide 25%	Skin Protectant
<b>USES</b>	
Dries the oozing and weeping of poison:	
■ Ivy ■ oak ■ sumac	
<b>WARNINGS</b>	
For external use only.	
When using this product ■ do not get into eyes.	
Stop and ask a doctor if ■ condition worsens	
■ symptoms last more than 7 days or clear up and occur again within a few days.	
Keep out of reach of children.	
If swallowed, get medical help or contact a Poison Control Center right away.	
<b>DIRECTIONS</b>	
• Apply as needed	
• Cleanse the leg and dry the skin.	
• If a wound is present, apply an appropriate wound dressing for moist wound healing.	
• With the foot positioned at a 90° or right angle to the leg (dorsiflexed) begin wrapping the medicated bandage without tension beginning at the base of the toes. Wrap the leg without tension overlapping each layer by 50% or half the width of the dressing. Smooth and mold the bandage while wrapping.	
• Wrap all the way up the leg stopping one finger width below the posterior knee.	
• Apply an elastic adhesive bandage (not included), following the manufacturer's instructions.	
<b>OTHER INGREDIENTS</b>	
Acacia, Castor oil, Glycerin, and White Petrolatum.	

me6 05-10

1 each/box  
Reorder # 2070

1 each/box  
Reorder # 2070

1 each/box  
Reorder # 2070

LOT



+M34020702

NDC # 68345-270-02

**REPARA®**

4" Unna Boot

- 4" x 10 yds.

- Non-hardening zinc oxide formulation
- Conforms easily to foot or leg
- Provides moist healing environment

Questions or Comments about Repara  
Products? 800-777-4908

Manufactured for PSS World Medical Inc.  
4345 Southpoint Blvd., Jacksonville, FL 32216  
Made in Mexico  
[www.myselectonline.com](http://www.myselectonline.com)

1 each/box

Reorder # 2071

#### Directions for Use

1. Cleanse the leg and dry the skin.
2. If a wound is present, apply an appropriate wound dressing for moist wound healing.
3. With the foot positioned at a 90 or right angle to the leg (dorsiflexed) begin wrapping the medicated bandage without tension beginning at the base of the toes. Wrap the leg without tension overlapping each layer by 50% or half the width of the dressing. Smooth and mold the bandage while wrapping.
4. Wrap all the way up the leg stopping one finger width below the posterior knee.
5. Apply an adhesive bandage (not included), following the manufacturer's instructions.



# REPARA

zinc oxide dressing

## Product Information

<b>Product Type</b>	HUMAN OTC DRUG	<b>Item Code (Source)</b>	NDC:68345-270
<b>Route of Administration</b>	TOPICAL		

**Active Ingredient/Active Moiety**

Ingredient Name	Basis of Strength	Strength
ZINC OXIDE (UNII: SOI2LOH54Z) (ZINC CATION - UNII:13S1S8SF37)	ZINC OXIDE	25 g

**Inactive Ingredients**

Ingredient Name	Strength
ACACIA (UNII: 5C5403N26O)	
CASTOR OIL (UNII: D5340Y2I9G)	
GLYCERIN (UNII: PDC6A3C0OX)	
PETROLATUM (UNII: 4T6H12BN9U)	

**Packaging**

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:68345-270-01	1 in 1 BOX		
2	NDC:68345-270-02	1 in 1 BOX		

**Marketing Information**

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph final	part347	05/17/2010	

**Labeler** - PSS World Medical, Inc. (101822682)**Registrant** - LABORATORIOS LE ROY, S.A. DE C.V. (588387394)**Establishment**

Name	Address	ID/FEI	Business Operations
LABORATORIOS LE ROY, S.A. DE C.V.		588387394	manufacture