

DERMFREE SCAR SHEETS- silicone gel 1% scar sheets patch
Jiangxi Hemei Pharmaceutical Co., Ltd

84010-066

Active Ingredient

Silicone gel 1%

Purpose

Scar Removal Sheets

Use

1. Clean and dry scar area.
2. Cut sheet according to the size and shape of the scar.
3. Peel off liner from adhesive side of sheet.
4. Apply adhesive side of sheet directly on scar. Apply 4 hours a day for the first two days, 8 hours a day for the next two days, and then increase by 2 hours a day until you can apply 24 hours a day. Each scar sheet can be used repeatedly and when another new scar patch becomes difficult to clean or paste, replace another new scar sheet.

Warnings

Use this product once scar or wound has been healing for at least 7 days. You may start using immediately on older scars. For old scars, the product can be used immediately.

Do not use on unhealed wounds, infected areas, or acne-prone skin.

If it's affixed to unclean skin or too tightly, it may cause rashes. At this time, it should be reduced to stick and stick for less time.

Dosage and administration For external use only.

Do not use

if you have an open wound. It may disrupt healing and up infection risk.

When Using

Do not use on unhealed wounds, infected areas, or acne-prone skin.

If it's affixed to unclean skin or too tightly, it may cause rashes. At this time,

it should be reduced to stick and stick for less time.

Dosage and administration For external use only.

Stop Use

stop use on broken or irritated scalp

Ask Doctor

Stop use and ask a doctor if skin issues, patch problems.

Keep Out Of Reach Of Children

Keep Away From Children Under 3 Years Old And Pets, Out Of Direct Sunlight And Store In A Cool, Dry Place

Directions

Use this product once scar or wound has been healing for at least 7 days, You may start using immediately on older scars, For old scars, the product can be used immediately.

Do not use on unhealed wounds. infected areas. or acne-prone skin.

If it's affixed to unclean skin or too tightly, it may cause rashes. At this time. it should be reduced to stick and stick for less time.

Dosage and administration For external use only.


Inactive ingredients

polydimethylsiloxane, polyurethane film, pet.

PRINCIPAL DISPLAY PANEL



<p>Drug Facts Active Ingredient.....Purpose Silicone gel 1%.....Scar Removal Sheets</p> <p>Uses 1. Clean and dry scar area. 2. Cut sheet according to the size and shape of the scar. 3. Peel off liner from adhesive side of sheet. 4. Apply adhesive side of sheet directly on scar. Apply 4 hours a day for the first two days, 8 hours a day for the next two days, and then increase by 2 hours a day until you can apply 24 hours a day. Each scar sheet can be used repeatedly and when another new scar patch becomes difficult to clean or paste replace another new scar sheet. Stop use and ask a doctor if skin issues, patch problems.</p> <p>WARNINGS Use this product once scar or wound has been healing for at least 7 days. You may start using immediately on older scars. For old scars, the product can be used immediately. Do not use on unhealed wounds, infected areas, or acne-prone skin.</p>	<p>SILICONE DERMFREE[®] PREMIUM SCAR SHEETS PREMIUM MEDICAL-GRADE SILICONE</p> <p>8 WEEKS ON NEW SCARS</p> <p>4-6 MONTHS ON OLD SCARS</p> <p>✓ Non-Irritant ✓ Non Invasive</p> <p>Breathable Flexible Comfortable 1 ROLL 1.6 x60 4cmx150cm</p> <p>If it's affixed to unclean skin or too tightly, it may cause rashes. At this time, it should be reduced to stick and stick for less time. Dosage and administration For external use only. When using this product Use this product once scar or wound has been healing for at least 7 days. You may start using immediately on older scars. DO NOT USE if you have an open wound. It may disrupt healing and up infection risk. stop use on broken or irritated scalp Keep out of reach of children. Keep Away From Children Under 3 Years Old And Pets, Out Of Direct Sunlight And Store In A Cool, Dry Place Inactive ingredients polydimethylsiloxane, polyurethane film, pet.</p> <p>NINTH FLOOR, UNIT 1, BUILDING 1, ORIENTAL PEARL, ZHANGSHU, JIANGXI 331200, CHINA (CHN). MANUFACTURED BY:JIANGXI HEMEI PHARMACEUTICAL CO.,LTD. 6 974603 693324</p>	<p>SILICONE DERMFREE[®] PREMIUM SCAR SHEETS PREMIUM MEDICAL-GRADE SILICONE</p> <p>8 WEEKS ON NEW SCARS</p> <p>4-6 MONTHS ON OLD SCARS</p> <p>✓ Non-Irritant ✓ Non Invasive</p> <p>Breathable Flexible Comfortable 1 ROLL 1.6 x60 4cmx150cm</p>	<p>SILICONE DERMFREE[®] PREMIUM SCAR SHEETS PREMIUM MEDICAL-GRADE SILICONE</p> <p>8 WEEKS ON NEW SCARS</p> <p>4-6 MONTHS ON OLD SCARS</p> <p>✓ Non-Irritant ✓ Non Invasive</p> <p>Breathable Flexible Comfortable 1 ROLL 1.6 x60 4cmx150cm</p>
--	---	--	--

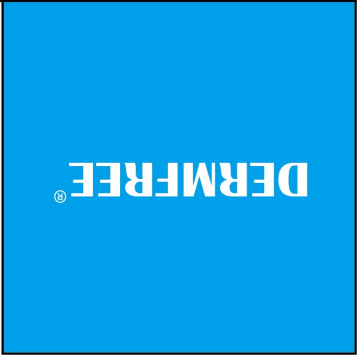

DERMFREE SCAR SHEETS			
silicone gel 1% scar sheets patch			
Product Information			
Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:84010-066
Route of Administration	TOPICAL		
Active Ingredient/Active Moiety			
	Ingredient Name	Basis of Strength	Strength
	CYCLOMETHICONE (UNII: NMQ347994Z) (CYCLOMETHICONE - UNII:NMQ347994Z)	CYCLOMETHICONE	1 g in 100 g
Inactive Ingredients			
	Ingredient Name		Strength
	POLYURETHANE-39 (UNII: J9PXX566KC)		
	DIMETHICONE, UNSPECIFIED (UNII: 92RU3N3Y10)		
	POLYETHYLENE TEREPHTHALATE (INTRINSIC VISCOSITY 0.40-0.70) (UNII: Y78AJD3DED)		

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:84010-066-01	36.2 g in 1 BOX; Type 0: Not a Combination Product	01/08/2025	

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC Monograph Drug	M016	01/08/2025	

Labeler - Jiangxi Hemei Pharmaceutical Co., Ltd (724892056)

Establishment

Name	Address	ID/FEI	Business Operations
Jiangxi Hemei Pharmaceutical Co., Ltd		724892056	manufacture(84010-066)

Revised: 1/2025

Jiangxi Hemei Pharmaceutical Co., Ltd