

NBGS SCARSILK MEDICAL GRADE SILICONE SCAR PAD- elastomer, silicone, for scar management patch
NATIONAL BIO GREEN SCIENCES LIMITED LIABILITY COMPANY

nbgs ScarSilk

nbgs ScarSilk Medical Grade Silicone Scar Pad
Rx only
For external use only
Not for ophthalmic use

nbgs ScarSilk Medical Grade Silicone Scar Pad Description

nbgs ScarSilk Medical Grade Silicone Scar Pad is intended for use in the management of closed hyperproliferative scars, both old and new hypertrophic or keloid scars resulting from burns, surgical procedures or trauma wounds.

Each **PAD** contains: Medical Grade Silicone

CLINICAL PHARMACOLOGY

The exact mechanism of action in improving the appearance of scar tissue from using silicone remains unknown. However, various suggestions have been made to explain the efficacy of silicone sheets, including hydration, pressure, temperature, oxygen transmission and silicone absorption. There is some evidence that the treatment affects the stratum corneum and, by reducing evaporation, restores better homeostasis in the tissue. In keloid and hypertrophic scarring, the stratum corneum allows more evaporation of water from the underlying tissue than occurs in normal skin. Silicone pads may prevent this, keeping the stratum corneum in optimal hydration and protecting the skin from environmental hazards, both of which can reduce abnormal scarring. The gel may also affect the stratum corneum by inhibiting mast cell activity, diminishing edema, vasodilatation and excessive extracellular matrix formation but the simple changes in temperature, pressure, oxygen tension and hydration produced by wound coverage probably constitute the main mechanism of action. Another hypothesis is that the effect of static electricity on silicone may influence the alignment of collagen deposition.

INDICATIONS AND USES

For use in the management of closed hyperproliferative (hypertrophic or keloid) scars.

CONTRAINDICATIONS

nbgs ScarSilk Medical Grade Silicone Scar Pad is contraindicated in patients with known hypersensitivity to silicone or any of the listed ingredients.

WARNINGS

For external use only. Avoid contact with eyes. Do not apply on areas of broken skin. Do not apply on third degree burns and open wounds. Never use on sutured wound until sutures have been removed.

PRECAUTIONS

Stop use and ask a doctor if irritation develops. In rare instances, silicone sheets may cause a rash on the

skin. This condition may result from improper cleansing of the scar area where the silicone pad has been applied. If this product is applied properly and skin irritation still occurs, discontinue use and consult your physician. If ingested, get medical help or contact Poison Control Center right away. **KEEP THIS AND ALL MEDICATION OUT OF THE REACH OF CHILDREN.**

This medication should be used as directed by your physician during pregnancy or while breastfeeding. Consult your doctor about the risks and benefits.

CAUTION: Federal law restricts this device to sale by or on the order of a physician.

ADVERSE REACTIONS

On rare occasions, the Pads may cause temporary redness, stinging, burning or irritation and normally disappear when the medication is discontinued.

DOSAGE AND ADMINISTRATION

1. Wash hands and scar area thoroughly with soap and water. Completely dry the area after.
2. If needed, cut the nbgs ScarSilk Pad to size making sure to leave at least 1/4” beyond the area of the scar.
3. Peel off the backing of the sheet and place it sticky side down on the affected area.
4. For best results, leave on for 8-12 hours per day.
5. Each sheet lasts 7 days, box contains 1 month supply.

How is nbgs ScarSilk Medical Grade Silicone Scar Pad Supplied

nbgs ScarSilk Medical Grade Silicone Scar Pad (Silicone Gel Matrix) is supplied in:
4 Non-Sterile Silicone Gel Pads – 5 cm × 14 cm in 1 Carton

Store at 20°-25°C (68° to 77°F); Keep away from heat and protect from freezing. [See USP Controlled Room Temperature.]

Manufactured for:

NATIONAL BIO GREEN SCIENCES LIMITED LIABILITY COMPANY


Packaging

NBGS ScarSilk Individual Label



ScarSilk

Medical-Grade Silicone Scar Pad

Rx ONLY 

INSTRUCTIONS

Wash hands and scar area thoroughly with soap and water, then dry area completely.

If Needed, cut silicone pad to size making sure to leave at least 1/4" beyond the area of the scar.

Peel off the backing of the pad and place it sticky side down on the affected area.

For best results leave the pad on for 6-23 hours.

Medical Silicone Scar Pads should be worn every day for a minimum of 2 months. Your kit contains 4 pads - each pad can last for 2-3 weeks.

Manufactured for;
National Bio Green Sciences llc
6b Culnen Drive
Branchburg NJ 08876 Made In PRC

CAUTIONS

Do not apply to open or oozing wounds, third degree burns, or any area that is crusting over. This could lead to serious infections or other complications.

Avoid direct contact with eyes or other mucous membranes

For external use only

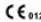
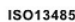

Consult physician immediately and discontinue use if rash or allergic reaction occurs

CHOKING HAZARD: Keep out of reach of children under 3 years and pets. Sheets could present a choking hazard.

Store at 20 - 25 C (68 to 77 F); Keep away from excessive heat and keep from freezing

NDC: 72678-001-01

Contains: 1 Pad
5cm x 14cm x 2.5mm

Rx ONLY   

NBGS ScarSilk 4 pack package



ScarSilk

Medical-Grade Silicone Scar Pad

FOR TREATMENT OF HYPERTROPHIC, KELOID,
AND HYPERPIGMENTED SCARS

MEDICAL GRADE DRESSINGS

INCLUDES 4 PADS

5cm x 14cm x 2.5mm

NDC: 72678-001-01



ScarSilk

Medical-Grade Silicone Scar Pad

MEDICAL-GRADE SILICONE DRESSING

NBGS4UNITSSP



3 72678 00101 7

CAUTIONS

Do not apply to open or oozing wounds, third degree burns, or any area that is crusting over. This could lead to serious infections or other complications.

Avoid direct contact with eyes or other mucous membranes

For external use only

Consult physician immediately and discontinue use if rash or allergic reaction occurs

CHOKING HAZARD: Keep out of reach of children under 3 years and pets. Sheets could present a choking hazard.

Store at 20 - 25 C (68 to 77 F); Keep away from excessive heat and keep from freezing

INSTRUCTIONS

Wash hands and scar area thoroughly with soap and water, then dry area completely.

If Needed, cut silicone pad to size making sure to leave at least 1/4" beyond the area of the scar.

Peel off the backing of the pad and place it sticky side down on the affected area.

For best results leave the pad on for 6-23 hours.

Medical Silicone Scar Pads should be worn every day for a minimum of 2 months. Your kit contains 4 pads - each pad can last for 2-3 weeks.

Manufactured for:
National Bio Green Sciences llc
6b Culnen Drive
Branchburg NJ 08876
Made in PRC

NDC: 72678-001-01    Rx ONLY





NBGS SCARSILK MEDICAL GRADE SILICONE SCAR PAD

elastomer, silicone, for scar management patch

Product Information

Product Type	MEDICAL DEVICE	Item Code (Source)	NHRIC:72678-001
Route of Administration	TOPICAL		

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:72678-001-01	4 in 1 CARTON		
1		1 in 1 POUCH; Type 0: Not a Combination Product		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
PREMARKET NOTIFICATION	K003948	11/13/2018	

Labeler - NATIONAL BIO GREEN SCIENCES LIMITED LIABILITY COMPANY (967054623)

Revised: 4/2019

NATIONAL BIO GREEN SCIENCES LIMITED LIABILITY COMPANY