MEDCLOT HEMOSTATIC DRESSING 10 CM X 10 CM- medclot hemostatic dressing dressing

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Disclaimer: This drug has not been found by FDA to be safe and effective, and this labeling has not been approved by FDA. For further information about unapproved drugs, click here.

This is a HEMOSTATIC DRESSING - Sterile Dressing 10 cm x 10 cm

CarboxymethylCellulose

Emergency bleeding control.

For use by trained emergency responders.

1. Open package and remove MED-CLOT.
2. Pack MED-CLOT into the wound and use it to apply pressure directly over bleeding source. More than one MED-CLOT may be required.
3. Continue to apply pressure until bleeding is under control.
4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show PRODUCT REMOVAL directions on package to medical personnel.

For external use only.

Sterility not guaranteed if package is damaged.

Avoid contact with eyes.

Reuse will cause risk of infection and loss of efficacy.

More than one MED-CLOT may be required.

Do not use if package is damaged.

Reuse will cause risk of infection and loss of efficacy.

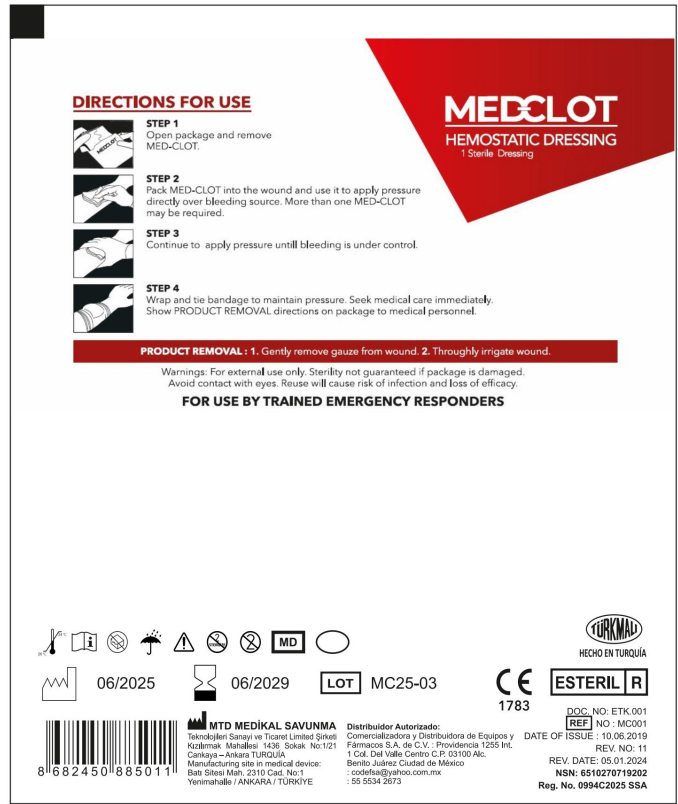
Keep out of reach of children.

Avoid contact with eyes.

Keep at room temperature.

Purified water (standard ISO 13959:2014)

85925-101-01

MEDCLOT HEMOSTATIC DRESSING 10 CM X 10 CM

medclot hemostatic dressing dressing

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:85925-101
Route of Administration	CUTANEOUS		

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
CARBOXYMETHYLCELLULOSE (UNII: 05JZI7B19X) (CARBOXYMETHYLCELLULOSE - UNII:05JZI7B19X)	CARBOXYMETHYLCELLULOSE	0.002 g in 100 mL

Inactive Ingredients

Ingredient Name	Strength
WATER (UNII: 059QF0KO0R)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:85925-101-01	10 mL in 1 BOTTLE; Type 0: Not a Combination Product	03/01/2025	

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
unapproved drug other		03/01/2025	

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Labeler - Comercializadora y Distribuidora de Equipos y Fármacos, S.A. de C.V. (951637153)
Comercializadora y Distribuidora de Equipos y Fármacos, S.A. de C.V.