

**DERMFREE SCAR SHEETS- silicone gel 1% scar sheets patch**  
**Jiangxi Yudexi Pharmaceutical Co., LTD**

-----  
**85248-110**

**Active Ingredient**

Silicone gel 1%

**Purpose**

Scar Reduction Sheets

**Use**

1. Clean and dry scar area.
2. Cut sheet according to the size and shape of the scar.
3. Peel off the liner from adhesive side of sheet.
4. Apply adhesive side of sheet directly on scar. Apply 4 hours a day for the first two days, 8 hours a day for the next two days, and then increase by 2 hours a day until you can wear it for 24 hours a day. Each scar sheet can be used repeatedly and when another new scar patch becomes difficult to clean or paste replace another new scar sheet.

**Warnings**

Use this product once scar or wound has healed for at least 7 days, You may start using immediately on older scars, For old scars, the product can be used immediately. Do not use on unhealed wounds, infected areas, or acne-prone skin. If applied to unclean skin or worn too tightly, it may cause skin irritation or rashes. If this occurs, reduce wear time and ensure proper application. It should be reduced to stick and stick for less time. Dosage and administration For external use only.

**Do not use**

Do not use on unhealed wounds, infected areas, or acne-prone skin. If you have an open wound. It may disrupt healing and increase the risk of infection.

**When Using**

Use this product once scar or wound has healed for at least 7 days. You may start using immediately on older scars.

**Stop Use**

stop use on skin becomes broken, irritated, or inflamed

### **Ask Doctor**

Stop use and ask a doctor if skin irritation, redness, or other adverse skin reactions occur.

### **Keep Out Of Reach Of Children**

Keep out of reach of children and pets. Store in a cool, dry place, away from direct sunlight.

### **Directions**

Use this product once scar or wound has healed for at least 7 days, You may start using immediately on older scars, For old scars, the product can be used immediately. Do not use on unhealed wounds. infected areas. or acne-prone skin. If applied to unclean skin or worn too tightly, it may cause skin irritation or rashes. If this occurs, reduce wear time and ensure proper application. it should be reduced to stick and stick for less time. Dosage and administration For external use only.

### **Inactive ingredients**

polydimethylsiloxane, polyurethane film, PET.

### **PRINCIPAL DISPLAY PANEL**

DERMFREE®

**Drug Facts**  
**Active Ingredient**..... **Purpose**  
 Silicone gel 1%..... Scar Reduction Sheets

**Uses**  
 1. Clean and dry scar area.  
 2. Cut sheet according to the size and shape of the scar.  
 3. Peel off the liner from adhesive side of sheet.  
 4. Apply adhesive side of sheet directly on scar.

Apply 4 hours a day for the first two days, 8 hours a day for the next two days, and then increase by 2 hours a day until you can wear it for 24 hours a day. Each scar sheet can be used repeatedly and when another new scar patch becomes difficult to clean or paste replace another new scar sheet.

**Stop use and ask a doctor** if skin irritation, redness, or other adverse skin reactions occur.

**WARNINGS**  
 Use this product once scar or wound has healed for at least 7 days. You may start using immediately on older scars. For old scars, the product can be used immediately.  
 Do not use on unhealed wounds, infected areas, or acne-prone skin.

SILICONE DERMFREE® PREMIUM  
**SCAR SHEETS**  
 PREMIUM MEDICAL-GRADE SILICONE

**8 WEEKS**  
**ON NEW SCARS**

**4-6 MONTHS**  
**ON OLD SCARS**

✓ Non-Irritant  
 ✓ Non Invasive



Breathable Flexible Comfortable **1 ROLL** 4cmx 150cm

If applied to unclean skin or worn too tightly, it may cause skin irritation or rashes. If this occurs, reduce wear time and ensure proper application.  
 Dosage and administration For external use only.

**When using this product**  
 Use this product once scar or wound has healed for at least 7 days. You may start using immediately on older scars.  
**DO NOT USE** if you have an open wound. It may disrupt healing and increase the risk of infection  
**stop use** on skin becomes broken, irritated, or inflamed.  
**Keep out of reach of children.**  
 Keep out of reach of children and pets.  
 Store in a cool, dry place, away from direct sunlight.

**Inactive ingredients**  
 polydimethylsiloxane, polyurethane film, PET.

Distributor: Jiangxi Yudexi Pharmaceutical Co., LTD  
 Address: No. 1908, Jingkai West 5th Road, Chengdei Jingkai District, Zhangshu City, Yichun City, Jiangxi Province



6 19 79843 981064

SILICONE DERMFREE® PREMIUM  
**SCAR SHEETS**  
 PREMIUM MEDICAL-GRADE SILICONE

**8 WEEKS**  
**ON NEW SCARS**

**4-6 MONTHS**  
**ON OLD SCARS**

✓ Non-Irritant  
 ✓ Non Invasive



Breathable Flexible Comfortable **1 ROLL** 4cmx 150cm

DERMFREE SCAR SHEETS			
silicone gel 1% scar sheets patch			
Product Information			
Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:85248-110
Route of Administration	TOPICAL		
Active Ingredient/Active Moiety			
Ingredient Name	Basis of Strength	Strength	
CYCLOMETHICONE (UNII: NMQ347994Z) (CYCLOMETHICONE - UNII:NMQ347994Z)	CYCLOMETHICONE	1 g in 100 g	
Inactive Ingredients			
Ingredient Name	Strength		
POLYETHYLENE TEREPHTHALATE (INTRINSIC VISCOSITY 0.40-0.70) (UNII: Y78AJD3DED)			
DIMETHICONE, UNSPECIFIED (UNII: 92RU3N3Y10)			
POLYURETHANE-39 (UNII: J9PXX566KC)			

**Packaging**

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:85248-110-01	36.2 g in 1 BOX; Type 0: Not a Combination Product	02/08/2026	

**Marketing Information**

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC Monograph Drug	M016	02/08/2026	

**Labeler** - Jiangxi Yudexi Pharmaceutical Co., LTD (455662836)

Revised: 2/2026

Jiangxi Yudexi Pharmaceutical Co., LTD