

VERIFINE ALCOHOL SWABS- alcohol swabs swab
Promisemed Hangzhou Meditech Co.,LTD.

Active Ingredient (in each pad)

Isopropyl Alcohol 70% v/v

Purpose

First aid antiseptic

Uses

First aid to help prevent the risk of infection in minor cuts, scrapes, and burns.

Warnings

- For external use only
- Flammable. Keep away from fire or flame.

Do not use

- in the eyes or apply over large areas of the body.
- in case of deep puncture wounds, animal bites, or serious burns, consult a doctor.
- on mucous membranes or irritated skin.

Stop use and ask doctor if

- the condition persists or gets worse. Do not use longer than 1 week unless directed by a doctor.

Keep out of reach of children

If swallowed, get medical help or contact a Poison Control Center right away.

Directions

- Clean the affected area and discard. After use, may be covered with a sterile bandage.
- Single use only; Use immediately after opening.
- Do not use if package is damaged.

Other Information

Store in a dry place at room temperature.

Inactive Ingredients

Water

Questions?

(888) 655-3812

Please use immediately after opening. Single use. Wipe the injection site and discard it.

ALCOHOL SWAB PACKET

For antiseptic skin use


Drawing Number: 1500ALS-Vf-1-A0-02

Size: 50X50mm

Material: 73g 纸铝箔膜


Color: Pantone 286 U

REV. 版本	Change / 变更历史	Date / 日期	Description of Change / 变更描述
00		2022-08-22	新增规格
01		2022-11-18	增加中文规格及包装图
02		2024-03-26	增加物料编号, 增加COC号

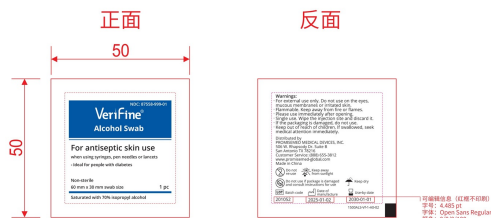

1:1

整版仅作为排版参考样式

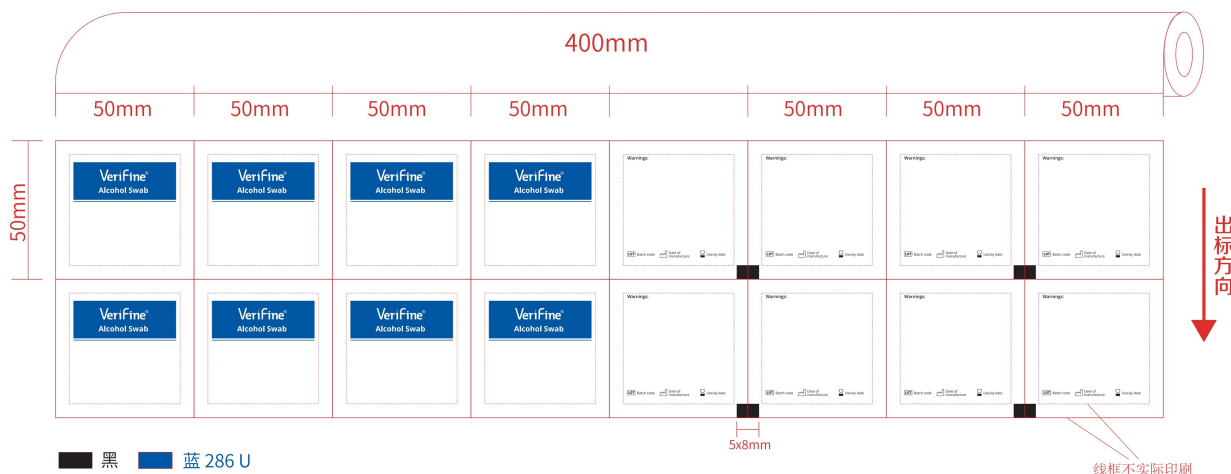

when using syringes, pen needles or lancets

ideal for people with diabetes

Non-sterile

60mm x 30mm swab size

1pc

Saturated with 70% isopropyl alcohol

KIT BAG

For antiseptic skin use

when using syringes, pen needles or lancets

ideal for people with diabetes

Non-sterile

60mm x 30mm swab size

1pc

Saturated with 70% isopropyl alcohol


Drawing Number: 1500ALS-Vf-1-A0-02

Size: 50X50mm


Material: 73g 纸铝膜

Color: Pantone 286 U

REV. 版本	Date 日期	Description of change 变更历史
00	2023-09-22	新增包装
01	2023-11-19	增加中文说明, 增加中英文对照
02	2024-03-20	增加中英文对照, 增加中文

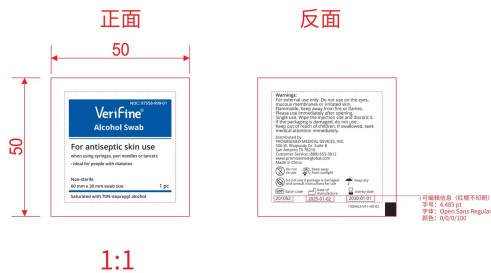

整版仅作为排版参考样式

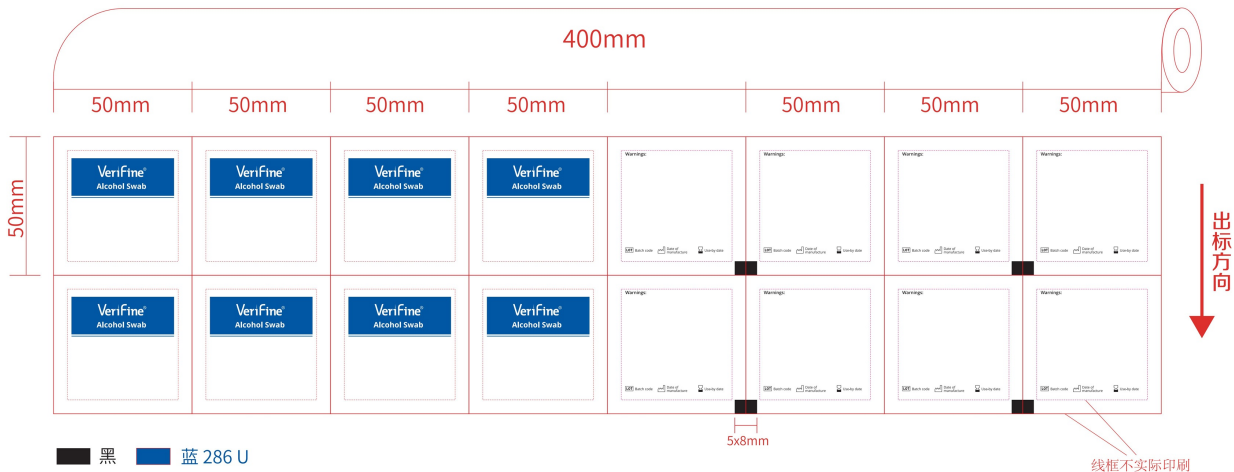

KIT BOX

6 insulin syringes + 6 alcohol swabs

ideal for people with diabetes

Alcohol Swabs

For antiseptic skin use when using syringes, pen needles or lancets

Non-sterile

60 mm x 30mm swab size

Saturated with 70% isopropyl alcohol

Do not re-use

Keep away from sunlight

Keep dry

Do not use if package is damaged and consult instructions for use


INJECTION KIT

6 insulin syringes and 6 alcohol swabs per pack

NDC: 87558-999-05

Verifine[®] INSULIN SYRINGES

1mL U-100 With Needle 31G x 8mm

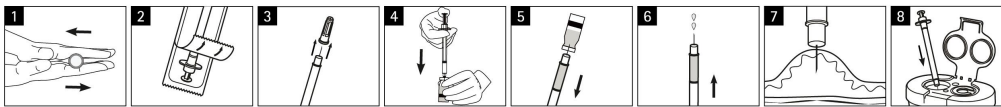

- 1). Roll the bottle between your hands if you are injecting cloudy insulin.
- 2). Single blister packaging: Remove syringe from packaging.
- 3). Remove needle cap without tilting it to avoid exposure to sharp needle point. Prepare the syringe for injection according to established procedures.
- 4). Inject the air into the upright insulin bottle.
- 5). Pull plunger down to desired units, filling syringe with insulin.
- 6). Remove air bubbles. Check to see if dose is correct.
- 7). Perform subcutaneous injection according to established procedures.
- 8). Immediately dispose of used syringe in an approved sharps container following the standard procedures of your facility and/or in accordance with the applicable local regulations for sharps disposal.

- Not made with natural rubber latex
- Discard after use
- Non-toxic
- Do not re-use
- Do not resterilize
- Sterilized using ethylene oxide

- MD Medical Device
- Non-pyrogenic
- Do not use if package is damaged and consult instructions for use
- Consult instructions for use or consult electronic instructions for use

EN
Precaution:
Reuse of single use devices creates a potential risk to the patient or user. It may lead to the contamination and/or limited functionality of the device and cause injury, illness or death.

ES
Precaución:
La reutilización de aparatos de un solo uso crea un riesgo potencial para el paciente o usuario. Puede conducir contaminación y/o la funcionalidad del aparato. Puede provocar lesiones, enfermedades, o la muerte del paciente.

Verifine[®] Alcohol Swabs

For antiseptic skin use
when using syringes, pen needles or lancets
- Ideal for people with diabetes

Non-sterile
60 mm x 30 mm swab size
Saturated with 70% isopropyl alcohol

- Do not re-use
- Keep away from sunlight
- Keep dry
- Do not use if package is damaged and consult instructions for use

NDC: 87558-999-01

Drug Facts

Active Ingredient (in each pad).....
Isopropyl alcohol 70% v/v ...First aid antiseptic

Uses
First aid to help prevent the risk of infection in minor cuts, scrapes, and burns.

- Warnings**
- For external use only.
 - Flammable. Keep away from fire or flame.
- Do not use**
- in the eyes or apply over large areas of the body.
 - in case of deep or puncture wounds, animal bites, or serious burns, consult a doctor.
 - on mucous membranes or irritated skin.

Drug Facts (continued)

Stop use and ask a doctor if

- the condition persists or gets worse. Do not use longer than 1 week unless directed by a doctor.

Keep out of reach of children. If swallowed, get medical help or contact a Poison Control Center right away.

- Directions**
- Clean the affected area and discard. After use, may be covered with a sterile bandage.
 - Single use only; Use immediately after opening.
 - Do not use if package is damaged.

Other Information

- Store in a dry place at room temperature.

Inactive Ingredients : Water

Questions? (888) 655-3812

UDI



(01) 00850062218833

(17) 300928

(10) 929052

Lot

929052

Exp

2025-09-29

Exp

2030-09-28

Distributed by:
TONMIT GROUP LIMITED
UNIT 1406A, 14/F THE BELGIAN BANK
BUILDING, NOS. 721-725 NATHAN ROAD,
KOWLOON, HONGKONG
www.tonmit.com
Made In China



(01) 0 0850062 21883 3

**200
packs**

Symbols: See Instructions for use
15581SY,ALS-VF-7103108-200-D0-00

PO No.: PMD1558XX

VERIFINE ALCOHOL SWABS

alcohol swabs swab

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:87558-999
Route of Administration	TOPICAL		

Active Ingredient/Active Moiety

Ingredient Name		Basis of Strength	Strength	
ISOPROPYL ALCOHOL (UNII: ND2M416302) (ISOPROPYL ALCOHOL - UNII:ND2M416302)		ISOPROPYL ALCOHOL	70 mL in 100 mL	
Inactive Ingredients				
Ingredient Name		Strength		
WATER (UNII: 059QF0KO0R)		30 mL in 100 mL		
Packaging				
#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:87558-999-05	200 in 1 BOX	04/07/2026	
1	NDC:87558-999-04	6 in 1 BAG		
1	NDC:87558-999-01	1 mL in 1 PACKET; Type 1: Convenience Kit of Co-Package		
Marketing Information				
Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date	
OTC Monograph Drug	M003	04/07/2026		

Labeler - Promised Hangzhou Meditech Co.,LTD. (421335278)

Establishment

Name	Address	ID/FEI	Business Operations
Promised Hangzhou Meditech Co.,LTD		421335278	manufacture(87558-999)

Revised: 4/2026

Promised Hangzhou Meditech Co.,LTD.