

**VERIFINE ALCOHOL SWABS- alcohol swabs swab**  
**Promisemed Hangzhou Meditech Co.,LTD.**

-----  
***Active Ingredient (in each pad)***

Isopropyl Alcohol 70% v/v

***Purpose***

First aid antiseptic

***Uses***

First aid to help prevent the risk of infection in minor cuts, scrapes, and burns.

***Warnings***

- For external use only
- Flammable. Keep away from fire or flame.

***Do not use***

- in the eyes or apply over large areas of the body.
- in case of deep puncture wounds, animal bites, or serious burns, consult a doctor.
- on mucous membranes or irritated skin.

***Stop use and ask doctor if***

- the condition persists or gets worse. Do not use longer than 1 week unless directed by a doctor.

***Keep out of reach of children***

If swallowed, get medical help or contact a Poison Control Center right away.

***Directions***

- Clean the affected area and discard. After use, may be covered with a sterile bandage.
- Single use only; Use immediately after opening.
- Do not use if package is damaged.

***Other Information***

Store in a dry place at room temperature.

## ***Inactive Ingredients***

Water

## ***Questions?***

(888) 655-3812

Please use immediately after opening. Single use. Wipe the injection site and discard it.

## **ALCOHOL SWAB PACKET**

For antiseptic skin use

when using syringes, pen needles or lancets

ideal for people with diabetes

Non-sterile

60mm x 30mm swab size

1pc

Saturated with 70% isopropyl alcohol


Drawing Number: 1500ALS-Vf-1-A0-02

Size: 50X50mm


Material: 73g 纸铝膜

Color: Pantone 286 U

Change log 变更记录		
REV. 版本	Date 日期	Record of change 变更记录
00	2025-09-29	新增规格
01	2025-11-19	新增中文说明
02	2026-03-28	更新说明内容, 增加中文

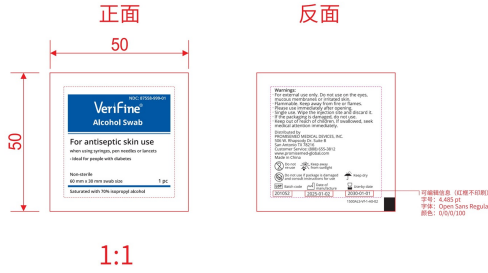

整版仅作为排版参考样式

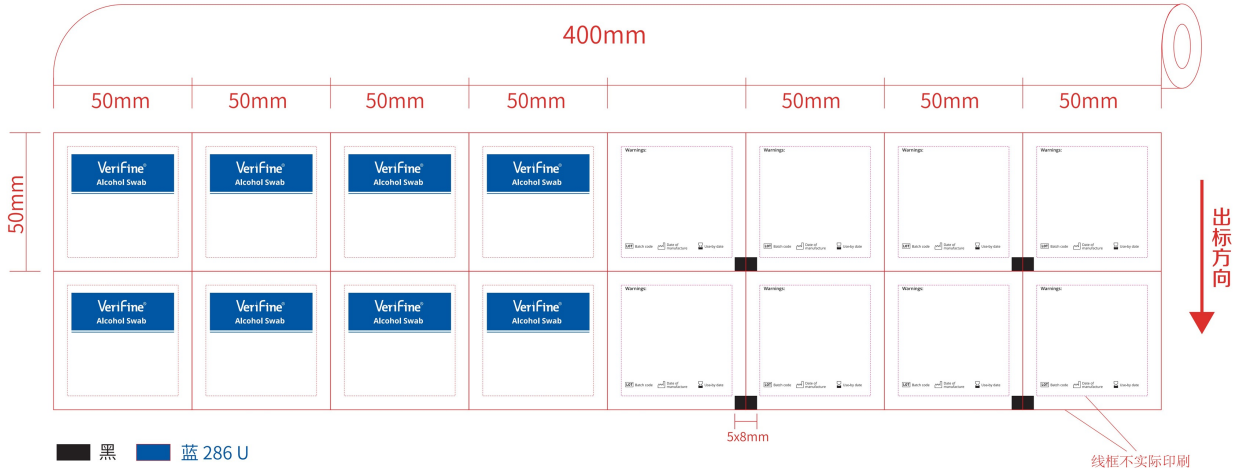

## KIT BAG

For antiseptic skin use

when using syringes, pen needles or lancets

ideal for people with diabetes

Non-sterile

60mm x 30mm swab size

1pc

Saturated with 70% isopropyl alcohol


Drawing Number: 1500ALS-Vf-1-A0-02

Size: 50X50mm


Material: 73g 纸铝膜

Color: Pantone 286 U

Change log 变更记录		
REV. 版本	Date 日期	Description 变更描述
01	2025-09-29	新增规格
01	2025-11-18	修正包装尺寸及重量
02	2025-03-28	添加物料号, 增加尺寸

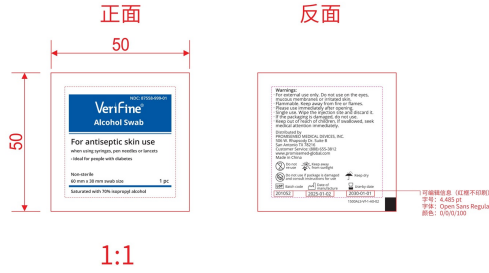

整版仅作为排版参考样式

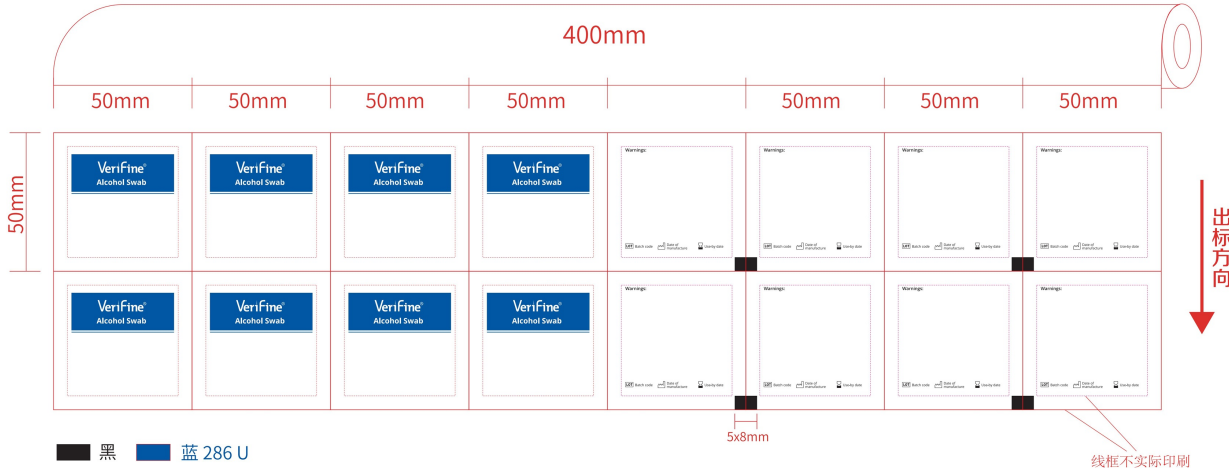

## KIT BOX

6 insulin syringes + 6 alcohol swabs

ideal for people with diabetes

Alcohol Swabs

For antiseptic skin use when using syringes, pen needles or lancets

Non-sterile

60 mm x 30mm swab size

Saturated with 70% isopropyl alcohol

Do not re-use

Keep away from sunlight

Keep dry


Do not use if package is damaged and consult instructions for use

# INJECTION KIT

6 insulin syringes and 6 alcohol swabs per pack

NDC: 87558-999-07

## Verifine<sup>®</sup> INSULIN SYRINGES 0.5mL U-100 With Needle 31G x 6mm

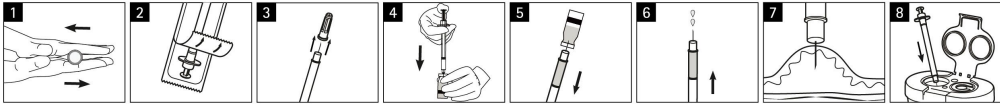

- Roll the bottle between your hands if you are injecting cloudy insulin.
- Single blister packaging: Remove syringe from packaging.
- Remove needle cap without tilting it to avoid exposure to sharp needle point. Prepare the syringe for injection according to established procedures.
- Inject the air into the upright insulin bottle.
- Pull plunger down to desired units, filling syringe with insulin.
- Remove air bubbles. Check to see if dose is correct.
- Perform subcutaneous injection according to established procedures.
- Immediately dispose of used syringe in an approved sharps container following the standard procedures of your facility and/or in accordance with the applicable local regulations for sharps disposal.

- Not made with natural rubber latex
- Discard after use
- Non toxic
- Do not re-use
- Do not resterilize
- Sterilized using ethylene oxide

- Medical Device
- Non-pyrogenic
- Do not use if package is damaged and consult instructions for use
- Consult instructions for use or consult electronic instructions for use

**EN**  
**Precaution:**  
Reuse of single use devices creates a potential risk to the patient or user. It may lead to the contamination and/or limited functionality of the device and cause injury, illness or death.

**ES**  
**Precaución:**  
La reutilización de aparatos de un solo uso crea un riesgo potencial para el paciente o usuario. Puede conducir contaminación y/o la funcionalidad del aparato. Puede provocar lesiones, enfermedades, o la muerte del paciente.

## Verifine<sup>®</sup> Alcohol Swabs

**For antiseptic skin use**  
when using syringes, pen needles or lancets  
- Ideal for people with diabetes

Non-sterile  
60 mm x 30 mm swab size  
Saturated with 70% isopropyl alcohol

- Do not re-use
- Keep away from sunlight
- Keep dry
- Do not use if package is damaged and consult instructions for use

NDC: 87558-999-01

### Drug Facts

**Active Ingredient (in each pad).....**  
.....**Purpose**  
Isopropyl alcohol 70% v/v ...First aid antiseptic

**Uses**  
First aid to help prevent the risk of infection in minor cuts, scrapes, and burns.

**Warnings**  

- For external use only.
- Flammable. Keep away from fire or flame.

**Do not use**

- in the eyes or apply over large areas of the body.
- in case of deep or puncture wounds, animal bites, or serious burns, consult a doctor.
- on mucous membranes or irritated skin.

### Drug Facts (continued)

**Stop use and ask a doctor if**  

- the condition persists or gets worse. Do not use longer than 1 week unless directed by a doctor.

**Keep out of reach of children.** If swallowed, get medical help or contact a Poison Control Center right away.

**Directions**  

- Clean the affected area and discard. After use, may be covered with a sterile bandage.
- Single use only; Use immediately after opening.
- Do not use if package is damaged.

**Other Information**  

- Store in a dry place at room temperature.

**Inactive Ingredients :** Water

**Questions?** (888) 655-3812

UDI



(01) 00850062218840  
(17) 300928  
(10) 929052

LOT

929052

EXP

2025-09-29

EXP

2030-09-28

Distributed by:  
TONMIT GROUP LIMITED  
UNIT 1406A, 14/F THE BELGIAN BANK BUILDING, NOS. 721-725 NATHAN ROAD, KOWLOON, HONGKONG  
www.tonmit.com  
Made in China



(01) 0 0850062 21884 0

**440**  
packs

Symbols: See Instructions for use

1558ISY,ALS-Vf-7053106-440-D0-00

PO No.: PMD1558XX

## VERIFINE ALCOHOL SWABS

alcohol swabs swab

### Product Information

<b>Product Type</b>	HUMAN OTC DRUG	<b>Item Code (Source)</b>	NDC:87558-999
<b>Route of Administration</b>	TOPICAL		

### Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
ISOPROPYL ALCOHOL (UNII: ND2M416302) (ISOPROPYL ALCOHOL - UNII:ND2M416302)	ISOPROPYL ALCOHOL	70 mL in 100 mL

### Inactive Ingredients

Ingredient Name	Strength
WATER (UNII: 059QF0KO0R)	30 mL in 100 mL

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:87558-999-07	440 in 1 BOX	04/13/2026	
1	NDC:87558-999-04	6 in 1 BAG		
1	NDC:87558-999-01	1 mL in 1 PACKET; Type 1: Convenience Kit of Co-Package		

## Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC Monograph Drug	M003	04/13/2026	

**Labeler** - Promisemed Hangzhou Meditech Co.,LTD. (421335278)

## Establishment

Name	Address	ID/FEI	Business Operations
Promisemed Hangzhou Meditech Co.,LTD		421335278	label(87558-999) , manufacture(87558-999) , pack(87558-999)

Revised: 4/2026

Promisemed Hangzhou Meditech Co.,LTD.