

**GENERAL SURGERY KIT- general surgery tray (kit)**  
**Centurion Medical Products**

-----  
**General Surgery Kit**

**Drug Description**

**AQUEOUS SOLUTIONS FOR INFILTRATION  
AND NERVE BLOCK**

**Ampul**

**Plastic Multiple-dose Fliptop Vial**

**Glass Teartop Vial**

Rx only

**DESCRIPTION**

Lidocaine Hydrochloride Injection, USP is a sterile, nonpyrogenic solution of lidocaine hydrochloride in water for injection for parenteral administration in various concentrations with characteristics as follows:

Concentration 0.5% 1% 1.5% 2%

mg/mL lidocaine HCl (anhyd.) 5 10 15 20

mg/mL sodium chloride 8 7 6.5 6

Multiple-dose vials contain 0.1% of methylparaben added as preservative. May contain sodium hydroxide and/or hydrochloric acid for pH adjustment. The pH is 6.5 (5.0 to 7.0).

Lidocaine is a local anesthetic of the amide type.

Lidocaine Hydrochloride, USP is chemically designated 2-(diethylamino)-N-(2,6-dimethylphenyl)-acetamide monohydrochloride monohydrate, a white powder freely soluble in water. The molecular weight is 288.82.

**General Surgery Kit - Primary Label**

MM1

**CENTURION®**

030712G

**Example General  
Surgery Tray**

Reorder

**XXXXXXXXX**

# CONTENTS:

- 1 CSR WRAP
- 1 FENESTRATED DRAPE
- 1 PAIR of GLOVES
- 1 DRAPE
- 2 GAUZE SPONGES
- 1 AMPULE CRACKER
- 1 LIDOCAINE HCl SOLUTION
- 2 SYRINGES<sup>[1]</sup>
- 1 FILTER NEEDLE<sup>[1,2]</sup>
- 1 SAFETY NEEDLE<sup>[1]</sup>
- 1 NEEDLE CUBE
- 1 PANEL OF LABELS
- 1 TRAY

## Example Only - Components & Title May Vary

LOT 00000



N/A



(01) 0 0653160 00000 9

NOTES: 1) Non-Pyrogenic. 2) Not for injection.  
Store between 20-25°C (68-77°F).

STERILE EO



Do not use if package is open or damaged.



Do not reuse. Single use only.

LATEX FREE



Consult instructions for use.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

CENTURION MEDICAL PRODUCTS CORP. | WILLIAMSTON | MI | 48895  
USA | 800.248.4058 | [www.centurionmp.com](http://www.centurionmp.com)

**GENERAL SURGERY KIT**

general surgery tray (kit) kit

### Product Information

<b>Product Type</b>	MEDICAL DEVICE	<b>Item Code (Source)</b>	NHRIC:24840-1504
---------------------	----------------	---------------------------	------------------

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:24840-1504-2	10 in 1 CASE		
1	NHRIC:24840-1504-1	1 in 1 PACKAGE, COMBINATION		

### Quantity of Parts

Part #	Package Quantity	Total Product Quantity
Part 1	1 AMPULE	10 mL

### Part 1 of 1

#### LIDOCAINE HYDROCHLORIDE

lidocaine hydrochloride anhydrous injection, solution

### Product Information

<b>Item Code (Source)</b>	NDC:0409-4282
---------------------------	---------------

<b>Route of Administration</b>	INFILTRATION
--------------------------------	--------------

### Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
LIDOCAINE HYDROCHLORIDE ANHYDROUS (UNII: EC2CNF7XFP) (Lidocaine - UNII:98PI200987)	LIDOCAINE HYDROCHLORIDE ANHYDROUS	20 mg in 1 mL

### Inactive Ingredients

Ingredient Name	Strength
Sodium Chloride (UNII: 451W47IQ8X)	6 mg in 1 mL
Water (UNII: 059QF0KO0R)	
Sodium Hydroxide (UNII: 55X04QC32I)	
Hydrochloric Acid (UNII: QTT17582CB)	

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:0409-4282-02	10 mL in 1 AMPULE		

## Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
ANDA	ANDA080408	03/30/2010	

## Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
exempt device	LRO	02/20/2012	

**Labeler** - Centurion Medical Products (017246562)

## Establishment

Name	Address	ID/FEI	Business Operations
Centurion Medical Products		148522279	manufacture, repack

## Establishment

Name	Address	ID/FEI	Business Operations
Centurion Medical Products		626660810	manufacture, repack

## Establishment

Name	Address	ID/FEI	Business Operations
Hospira		093132819	manufacture

Revised: 3/2012

Centurion Medical Products