

**LIDOBAND EZE- lidocaine ring**  
**Alberta Veterinary Laboratories Ltd**

*Disclaimer: This drug has not been found by FDA to be safe and effective, and this labeling has not been approved by FDA. For further information about unapproved drugs, click [here](#).*

-----  
**LidoBand EZE**

**Description:**

LidoBand™ EZE is a latex rubber band designed to be placed around the base of the scrotum of bulls for castration. Each castration band is infused with 400 mg of lidocaine, which provides local anesthesia. Lidocaine will quickly diffuse through the skin and remain in the local tissue due to vasoconstriction from the band.

**Directions for Use:**

Do not touch the band with bare fingers. Wear gloves when handling or applying Lidoband EZE. Because lidocaine is used in LidoBand EZE, change gloves frequently while handling multiple bands. A tension band applicator (TI Banders type) is recommended. Follow the manufacturer's directions for the tension band applicator when applying LidoBand EZE. The properties of latex can change with temperature. Cold or hot air temperatures may cause bands to break when expanded. It is recommended to apply bands at temperatures between 60°F and 85°F (approximately 15°C and 30°C).

**Precautions:**

Lidocaine is typically well tolerated. As with all local anesthetics, untoward effects may occur due to hypersensitivity. LidoBand EZE should not be used on animals with a known lidocaine hypersensitivity. Castration of cattle may result in the development of tetanus. For tetanus prevention, it is highly recommended to vaccinate with a tetanus toxoid at least three (3) weeks prior to castration or provide tetanus antitoxin at the time of castration. Consult your veterinarian on prevention of tetanus in cattle.

**User Safety:**

If skin comes in contact with LidoBand EZE, wash affected area immediately with soap and water.

**Storage:**

Keep stored in a cool, dry place. Do not leave bands in direct sunlight or allow exposure to high temperatures. Do not re-use the band.

For castration of bulls and to deliver lidocaine as a local anesthetic.


For use in bulls weighing greater than 400 pounds.  
**FOR ANIMAL USE ONLY**  
**KEEP OUT OF REACH OF CHILDREN**

**LidoBand EZE Labels:**

**LidoBand EZE Jar Label**

Description: LidoBand™ EZE is a latex rubber band designed to be placed around the base of the scrotum of bulls for castration. Each castration band is infused with 400 mg of lidocaine, which provides local anesthesia. Lidocaine will quickly diffuse through the skin and remain in the local tissue due to vasoconstriction from the band.

Directions for use: Do not touch the band with bare fingers. Wear gloves when handling or applying LidoBand EZE. Because lidocaine is used in LidoBand EZE, change gloves frequently while handling multiple bands. A tension band applicator (T1 Banders type) is recommended. Follow the manufacturer's directions for the tension band applicator when applying LidoBand EZE. The properties of latex can change with temperature. Cold or hot air temperatures may cause bands to break when expanded. It is recommended to apply bands at temperatures between 60°F and 85°F (approximately 15°C and 30°C).

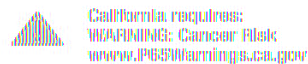

Manufactured by :  
**AVL/Solvat Ltd. 7228 107 Ave, Calgary, AB T2C 5N6**  
**1-(877) 456-2755. www.solvat-US.com**

Solvat is a trademark of Alberta Veterinary Laboratories Ltd.



VI Rev 02/15

# lidoBand™ EZE

**Castration Bands**  
Each band contains 400 mg of lidocaine.

For castration of bulls and to deliver lidocaine as a local anesthetic.  
For use in bulls weighing greater than 400 pounds.

**FOR ANIMAL USE ONLY**  
**KEEP OUT OF REACH OF CHILDREN**  
**NDC: 88174-104-07**


20 Bands + Clips

LOT: xxxxxxxxxxxx  
EXP: MMY




**U.S. Patent # 11,509,510**

Precautions:  
anesthetics, u  
LidoBand EZE  
lidocaine type  
Castration of c  
tetanus preven  
tetanus toxoid  
provide tetanu  
veterinarian or  
User Safety: I  
affected area i  
Storage: Keep  
direct sunlight  
re-use the ban



Please scan for rx  
and information  
or go to us

**LidoBand EZE Sample Pack Label**



**SAMPLE**  
**NOT FOR U**

# lidoBand™ E

**Castration Bands**



Each case is a latex rubber band that is placed around the base of the earlobe to hold the lidocaine ring in place. The lidocaine ring is a small, circular, white, flexible ring that is placed around the earlobe. The lidocaine ring is made of a soft, flexible material that is designed to be comfortable to wear. The lidocaine ring is used to numb the area around the earlobe for the removal of the earlobe. The lidocaine ring is used to numb the area around the earlobe for the removal of the earlobe. The lidocaine ring is used to numb the area around the earlobe for the removal of the earlobe.



## LIDOBAND EZE

lidocaine ring

### Product Information

Product Type	OTC ANIMAL DRUG	Item Code (Source)	NDC:86174-104
Route of Administration	TOPICAL		

### Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
LIDOCAINE (UNII: 98PI200987) (LIDOCAINE - UNII:98PI200987)	LIDOCAINE	400 mg

### Inactive Ingredients

Ingredient Name	Strength
ISOPROPYL MYRISTATE (UNII: 0RE8K4LNJS)	60 mg

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:86174-104-07	20 in 1 JAR		
2	NDC:86174-104-08	3 in 1 PACKAGE		

## Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
unapproved drug other		04/03/2025	

**Labeler** - Alberta Veterinary Laboratories Ltd (204293518)

## Establishment

Name	Address	ID/FEI	Business Operations
Alberta Veterinary Laboratories Ltd		204293518	manufacture

## Establishment

Name	Address	ID/FEI	Business Operations
Swati Spentose Private Limited		650621472	api manufacture

Revised: 4/2025

Alberta Veterinary Laboratories Ltd