

**JUVAZIN- elastomer, silicone, for scar management**  
**Misemer Pharmaceutical**

-----  
**Juvazin**

**INGREDIENTS**

Dimethicone, Dimethicone Crosspolymer, Trimethylsiloxysilicate.

**INDICATIONS AND USES**

Juvazin™ is intended for the management of old and new scars, including hypertrophic and keloid scars, resulting from general surgical procedures, trauma, wounds and burns.

**WARNINGS**

Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. Juvazin™ should not be used on dermatological conditions that disrupt the integrity of the skin. For external use only. If irritation occurs, discontinue use and consult your physician.

Keep this product out of the reach of children.

Store below 25°C/77°F.

**PRECAUTIONS**

Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

**CALL YOUR DOCTOR ABOUT SIDE EFFECTS.**

Call your doctor about side effects. You may report side effects to the FDA at 1-800-FDA-1088.

**DOSAGE AND ADMINISTRATION**

Ensure that the affected area is clean and dry. Apply Juvazin™ to the area as a very thin coat and allow to dry. Apply Juvazin™ twice daily. Once dry, Juvazin™ can be covered with cosmetics or sunscreen. Recommended duration of treatment 60-90 days.

0276-0600-30  
MISEMER  
PHARMACEUTICAL

Rx Only

Rx Only

Rx Only

Rx Only



0276-0600-30

**JUVAZIN**  
(Silicone)

**PRESCRIPTION  
SCAR CARE GEL**

**Ingredients:** Dimethicone,  
Dimethicone Crosspolymer,  
Trimethylsilylsilicate

**Caution:** Federal law restricts  
this device to sale by or on the  
order of a licensed healthcare  
practitioner.

**Distributed by:**  
Miseimer Pharmaceutical, Inc  
Ripley, MS 38663

Made in USA

Item # 0600  
Rev 1021-01



0276-0600-30

**JUVAZIN**  
(Silicone)

**PRESCRIPTION  
SCAR CARE GEL**

**WARNING:** Avoid direct contact  
with eyes, mucous  
membranes, third degree  
burns and open wounds.  
Juvazin should not be used on  
dermatological conditions that  
disrupt the integrity of the skin.  
For external use only. If  
irritation occurs, discontinue  
use and consult your physician.  
**Keep this product out of the  
reach of children.**  
Store below 25°C/77°F.

**INDICATION FOR USE:** Juvazin  
is intended for the  
management of old and new  
scars, including hypertrophic  
and keloid scars, resulting  
from general surgical  
procedures, trauma, wounds  
and burns.



0276-0600-30

**JUVAZIN**  
(Silicone)

**PRESCRIPTION  
SCAR CARE GEL**

30g

**DIRECTIONS:** Ensure that  
the affected area is clean  
and dry. Apply Juvazin to the  
area as a very thin coat and  
allow to dry. Apply Juvazin  
twice daily. Once dry, Juvazin  
can be covered with  
cosmetics or sunscreen.

Recommended duration of  
treatment 60-90 days.



0276-0600-30  
MISEMER  
PHARMACEUTICAL

Rx Only

Rx Only

Rx Only

Rx Only



0276-0600-30

**JUVAZIN**  
(Silicone)

**PRESCRIPTION  
SCAR CARE GEL**

**Ingredients:** Dimethicone,  
Dimethicone Crosspolymer,  
Trimethylsilylsilicate

**Caution:** Federal law restricts  
this device to sale by or on the  
order of a licensed healthcare  
practitioner.

**Distributed by:**  
Miseimer Pharmaceutical, Inc  
Ripley, MS 38663

Made in USA

Item # 0600  
Rev 1021-01



0276-0600-30

**JUVAZIN**  
(Silicone)

**PRESCRIPTION  
SCAR CARE GEL**

**WARNING:** Avoid direct contact  
with eyes, mucous  
membranes, third degree  
burns and open wounds.  
Juvazin should not be used on  
dermatological conditions that  
disrupt the integrity of the skin.  
For external use only. If  
irritation occurs, discontinue  
use and consult your physician.  
**Keep this product out of the  
reach of children.**  
Store below 25°C/77°F.

**INDICATION FOR USE:** Juvazin  
is intended for the  
management of old and new  
scars, including hypertrophic  
and keloid scars, resulting  
from general surgical  
procedures, trauma, wounds  
and burns.



0276-0600-30

**JUVAZIN**  
(Silicone)


**PRESCRIPTION  
SCAR CARE GEL**

30g

**DIRECTIONS:** Ensure that  
the affected area is clean  
and dry. Apply Juvazin to the  
area as a very thin coat and  
allow to dry. Apply Juvazin  
twice daily. Once dry, Juvazin  
can be covered with  
cosmetics or sunscreen.

Recommended duration of  
treatment 60-90 days.



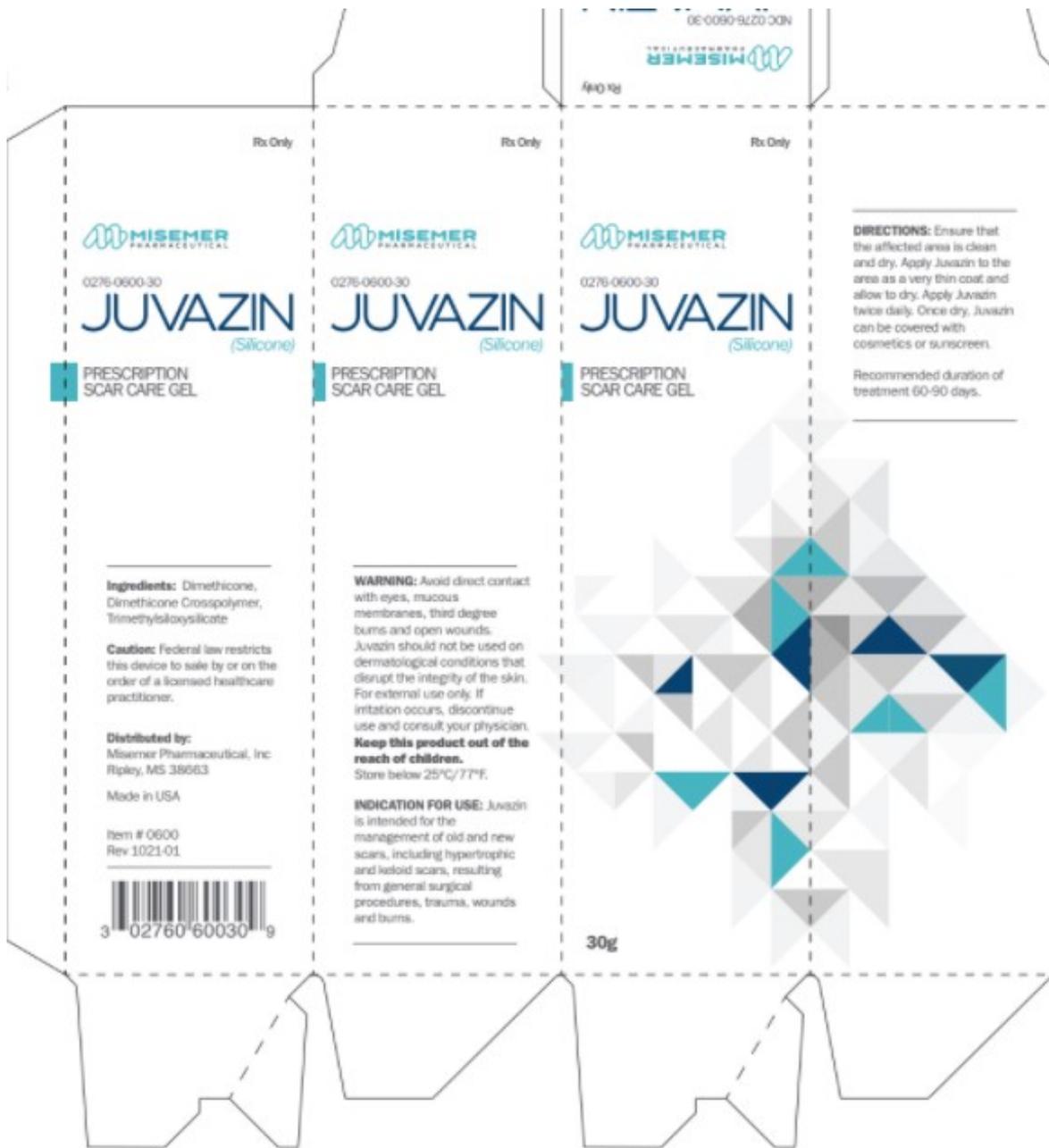

## JUVAZIN

elastomer, silicone, for scar management

### Product Information

<b>Product Type</b>	MEDICAL DEVICE	<b>Item Code (Source)</b>	NHRIC:0276-0600
---------------------	----------------	---------------------------	-----------------

### Inactive Ingredients

Ingredient Name	Strength
<b>TRIMETHYLSILOXYSILICATE (M/Q 0.6-0.8)</b> (UNII: 5041RX63GN)	
<b>DIMETHICONE CROSSPOLYMER (450000 MPA.S AT 12% IN CYCLOPENTASILOXANE)</b> (UNII: UF7620L1W6)	
<b>DIMETHICONE</b> (UNII: 92RU3N3Y1O)	

**Packaging**

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:0276-0600-30	30 in 1 TUBE; Type 0: Not a Combination Product		

**Marketing Information**

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
exempt device	MDA	12/01/2021	

**Labeler** - Misemer Pharmaceutical (784121365)

Revised: 7/2019

Misemer Pharmaceutical