


# WELLDOC BLUESTAR- accessories, pump, infusion WellDoc, Inc

-----  
**Prescription Medical Device**

## WellDoc BlueStar User Guide

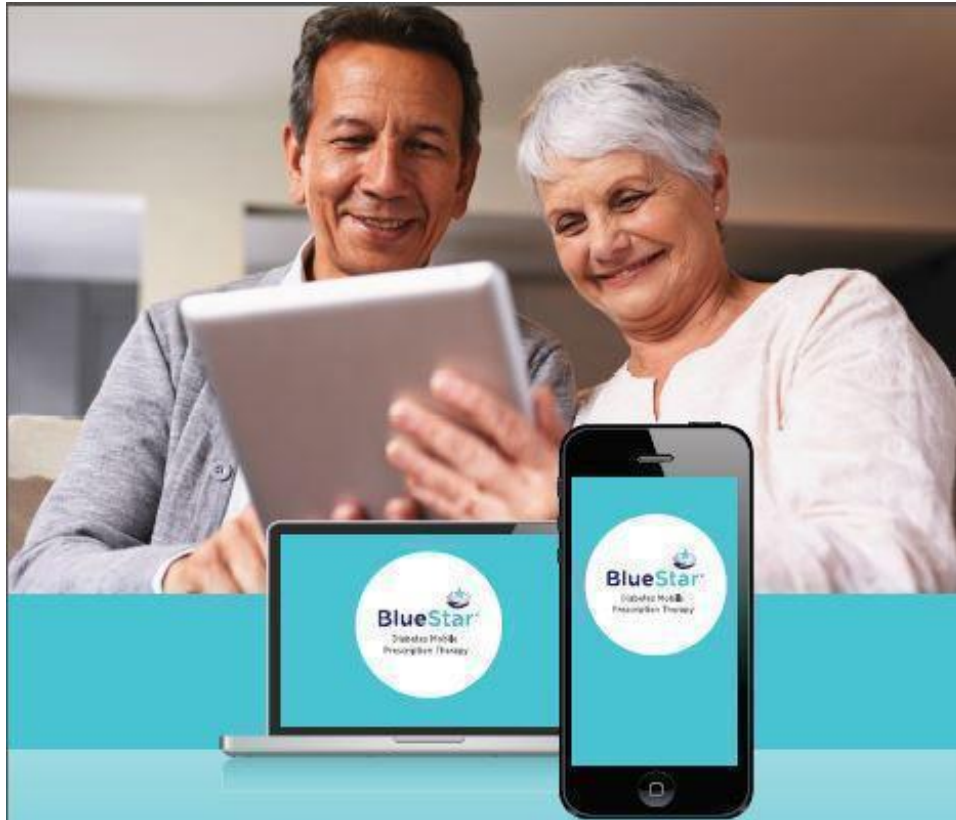

**Mobile Prescription Therapy**  
for adults with type 2 diabetes

**User Guide**

©2014 WellDoc, Inc. All Rights Reserved. WellDoc and the WellDoc logo are registered trademarks, and BlueStar, the BlueStar logo, and Automated Expert Analytic System are trademarks of WellDoc, Inc. BlueStar® is manufactured by WellDoc, Inc., Baltimore, MD. iPhone is a registered trademark of Apple Inc. DBM-101 Version 9

Rx only

POWERED BY WellDoc

## Product Description

### **Introducing BlueStar**

A new way to better manage your diabetes

BlueStar is the first mobile prescription therapy for adults with Type 2 diabetes. BlueStar gives you real-time guidance that makes managing diabetes between office visits easier and more doable. BlueStar must be prescribed by a provider.

**Is BlueStar right for you?**

Most people struggle to manage type 2 diabetes. Sometimes it is overwhelming and confusing. Understanding and remembering your doctor's advice about medications, diet, and exercise isn't easy. If you need help controlling your blood glucose, understanding everyday diabetes decisions, and keeping all your information in one place between office visits, BlueStar can help.

### **Diabetes Support, Anytime Anywhere™**

BlueStar is not a pill or a shot — it is a completely new kind of approach to support you and help you manage your type 2 diabetes. BlueStar's real-time feedback and guidance fits into your daily life to help you stay on track with your diabetes care plan. It works with you, struggles with you, achieves with you, and celebrates with you. BlueStar can help make living with diabetes easier which makes for a healthier, happier you.

BlueStar is accessible from your mobile phone or computer. It supports your healthcare provider's instructions while helping you build your knowledge about diabetes and stay motivated.

### **Share your results with your doctor between visits**

Before your next diabetes check-up, you can use BlueStar to send the information you put into BlueStar to your doctor. This can help your healthcare provider understand the challenges that you face between appointments. That way, during your office visit, you can both be ready to discuss your treatment plan and experiences managing diabetes.

### **Indication for use**

BlueStar is indicated for use by healthcare providers and their adult patients, aged 21 years and older, who have type 2 diabetes. BlueStar is not intended to replace the care provided by a licensed healthcare professional, including prescriptions, diagnosis, or treatment. BlueStar is not indicated for people with type 1 diabetes, gestational diabetes or who use an insulin pump.

**Important Safety Information:** For medical questions, please contact your healthcare provider. If you are experiencing an emergency, please dial 911.

For more information visit [www.bluestardiabetes.com](http://www.bluestardiabetes.com) or call BlueStar Customer Care at **1-888-611-4794**

©2014 WellDoc, Inc. All Rights Reserved. WellDoc and the WellDoc logo are registered trademarks, and BlueStar, the BlueStar logo, and Automated Expert Analytic System are trademarks of WellDoc, Inc. BlueStar® is manufactured by WellDoc, Inc., Baltimore, MD. iPhone is a registered trademark of Apple Inc.

DSM-161 Version 09

### **BlueStar Customer Care**



## Ready to Answer Your Questions

BlueStar® Customer Care will be happy to help you with the following:


- How to use BlueStar
- Reporting errors when trying to use BlueStar
- Technical questions related to BlueStar on mobile and/or Web
- Questions about your prescription or health insurance reimbursement

**Available:** Monday-Friday, 8:00 a.m.-7:00 p.m. (ET)

**Phone:** (888) 611-4794

## Getting Started

If you are new to BlueStar®, tap Sign Up to start registering or Log In with your username and password.



Already have an account? [Log In](#)

---

### Register for your BlueStar account


Complete this form to create BlueStar account

If you have any question, please contact our customer support at 1-888-611-4794 on Monday - Friday 8:00 am - 7:00 pm(ET)

Access Code <input type="text"/>	Last Name <input type="text"/>
First Name <input type="text"/>	Gender <input type="radio"/> Female <input type="radio"/> Male
Date of Birth <input type="text"/>	
Email <input type="text"/>	
Choose Username <input type="text"/>	
Choose Password <input type="text"/>	Confirm Password  <input type="text"/>
Choose a security question Select a question 	Answer <input type="text"/>
Do you take medication for diabetes? <input type="radio"/> Yes <input type="radio"/> No	
<input type="checkbox"/> I am at least 21 years old with type 2 diabetes. I am not pregnant and currently not using any insulin pump	
<input type="checkbox"/> I agree to <a href="#">Terms of service</a>	

## Virtual Training

- After you complete registration, you will be taken to BlueStar training video to learn how to set up your medication profile and enter your diabetes data.
- Once you complete your training, it is important that you enter your medications in BlueStar so you can receive personalized guidance in managing your diabetes.
- The training video is available in BlueStar Help section for you to watch again if needed. To learn more about the Health Info Section, sending your SMART Visit report to your doctor before your visit or any other features within BlueStar, visit the Help section.

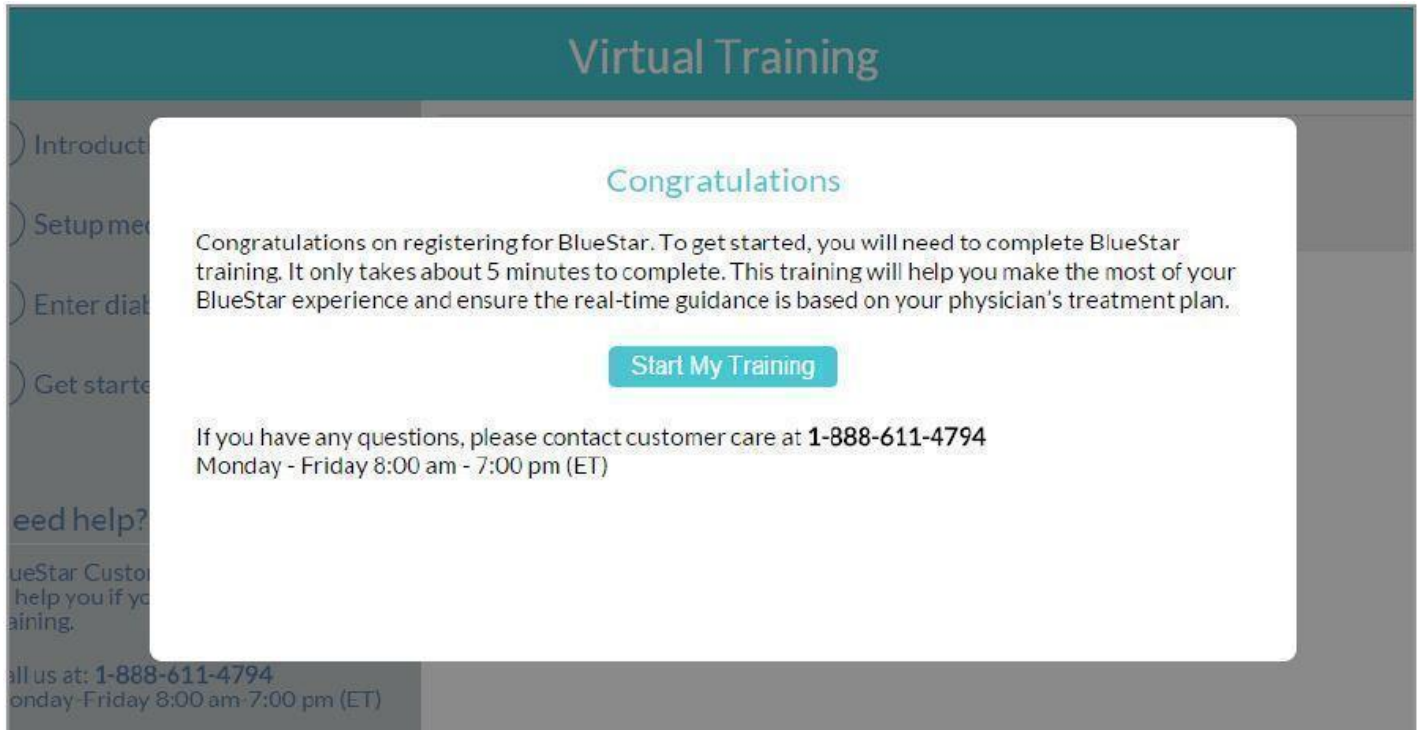

The screenshot shows a web interface for 'Virtual Training'. At the top, there is a dark teal header with the text 'Virtual Training' in a light color. Below the header is a navigation menu on the left with items like 'Introduction', 'Setup medication', 'Enter diabetes data', and 'Get started'. The main content area is a white box with a teal border. It features a 'Congratulations' heading, a paragraph of text, a teal 'Start My Training' button, and contact information for customer care. At the bottom of the page, there is a dark grey footer with the phone number '1-888-611-4794' and the hours 'Monday-Friday 8:00 am - 7:00 pm (ET)'.

### Virtual Training

#### Congratulations

Congratulations on registering for BlueStar. To get started, you will need to complete BlueStar training. It only takes about 5 minutes to complete. This training will help you make the most of your BlueStar experience and ensure the real-time guidance is based on your physician's treatment plan.

[Start My Training](#)

If you have any questions, please contact customer care at **1-888-611-4794**  
Monday - Friday 8:00 am - 7:00 pm (ET)

Need help?  
BlueStar Customer Support can help you if you need help during training.  
Call us at: 1-888-611-4794  
Monday-Friday 8:00 am - 7:00 pm (ET)

## Congratulations

Congratulations on registering for BlueStar. To get started, you will need to complete BlueStar training. It only takes about 5 minutes to complete. This training will help you make the most of your BlueStar experience and ensure the real-time guidance is based on your physician's treatment plan.

[Start My Training](#)


If you have any questions, please contact customer care at **1-888-611-4794**  
Monday - Friday 8:00 am - 7:00 pm (ET)

### **Getting Started (continued)**

**Follow the simple instructions below to create your BlueStar account.**

To sign up:

- a. Enter your Access Code
- b. Enter your personal information
- c. Tap on phone or click on website to agree to Terms of Service
- d. Read Patient Authorization then tap or click on Accept
- e. Tap or click on Create Account

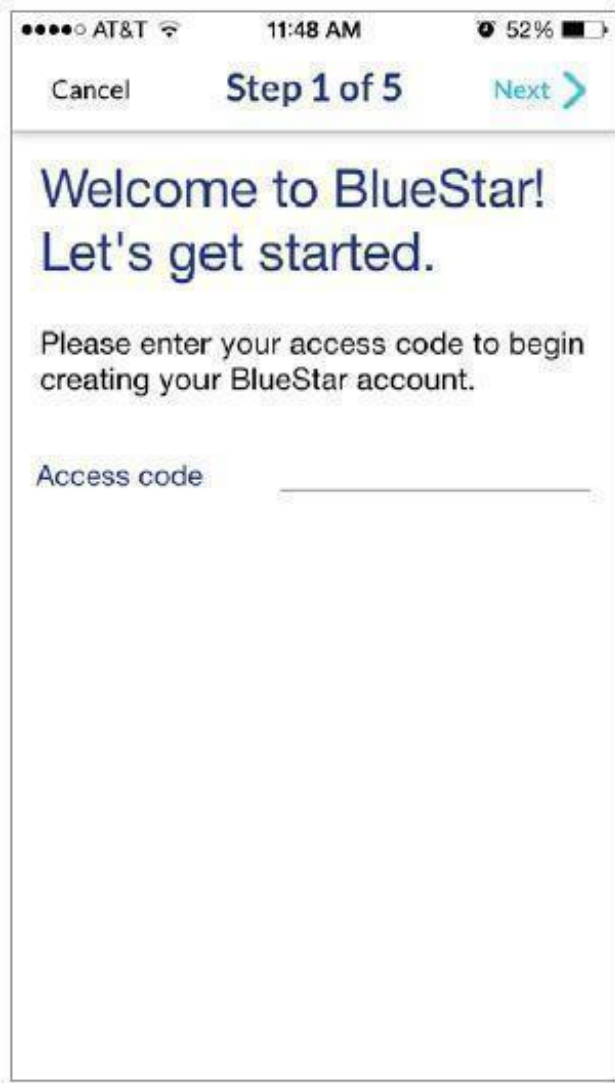

**Using BlueStar on Your Phone**



BlueStar® is recommended for use on smartphones with an iOS 7 or greater or Android OS 4.0 or greater.


©2014 WellDoc, Inc. All Rights Reserved. WellDoc and the WellDoc logo are registered trademarks, and BlueStar, the BlueStar logo, and Automated Expert Analytic System are trademarks of WellDoc, Inc. BlueStar® is manufactured by WellDoc, Inc., Baltimore, MD. iPhone is a registered trademark of Apple Inc.

### **BlueStar Mobile**

BlueStar® is the first prescription app for adults with Type 2 diabetes. BlueStar gives you real-time guidance that makes managing diabetes between office visits easier and more doable. BlueStar must be prescribed by a provider.

#### **Log In**

Enter your Username and Password then tap **LOG IN**

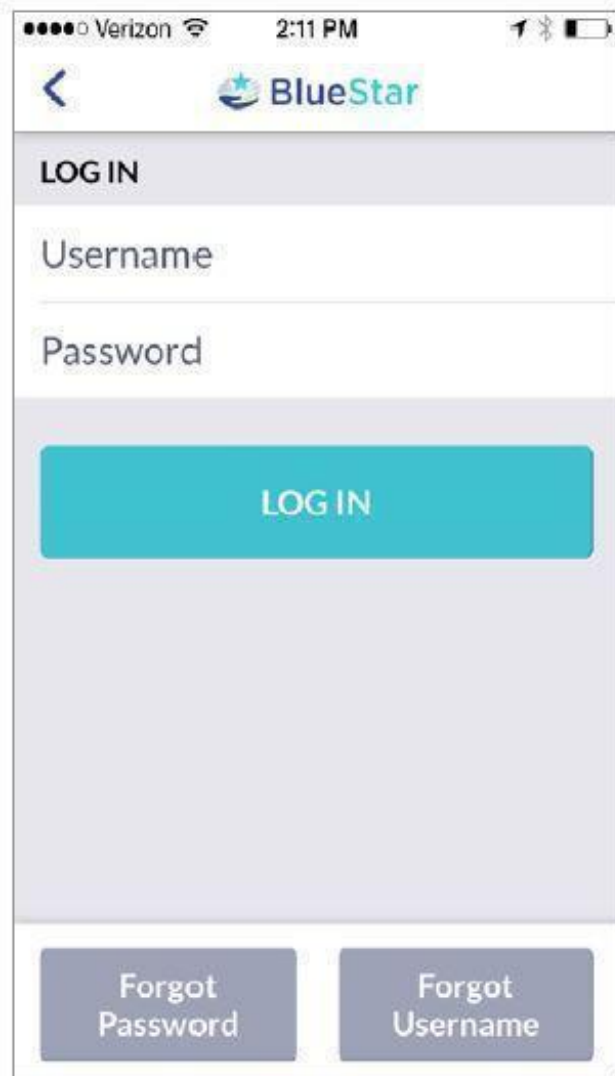

### Trouble logging in?

Tap **FORGOT PASSWORD?** to retrieve your BlueStar password or **FORGOT USERNAME?** to retrieve your username

The next time you log in to BlueStar, you will be asked to create a 4-digit PIN.  
Your PIN will make logging in faster and easier.  
Enter your PIN on the dotted line each time you log in



AT&T 11:49 AM 51%

< **Forgot Password** Next >

To retrieve your password, please enter the username and email address associated with your BlueStar account.




Username \_\_\_\_\_

Email \_\_\_\_\_

### **Home Screen (mobile)**

**The BlueStar® carousel displays important information to help you manage your diabetes.**

- a. When BlueStar sees something interesting about your BG, a message is delivered to your carousel and logbook for that day.
- b. Check for renewal reminders for your BlueStar prescription on the carousel
- c. Get a daily boost with a new message to motivate you every day about your general health and well-being.
- d. Short, 2- to 5-minute educational videos are designed to build your knowledge about diabetes and fit into your busy schedule. Each video is available for 48 hours before it is replaced by the next one in the carousel.
- e. Menu icon top left. Select the page you want to use or view.
- f. Tap + icon top right from any screen to make a new entry

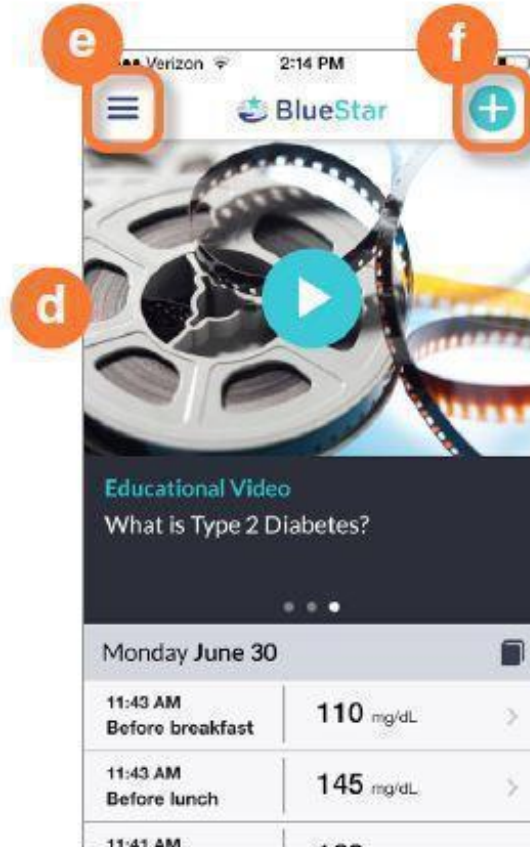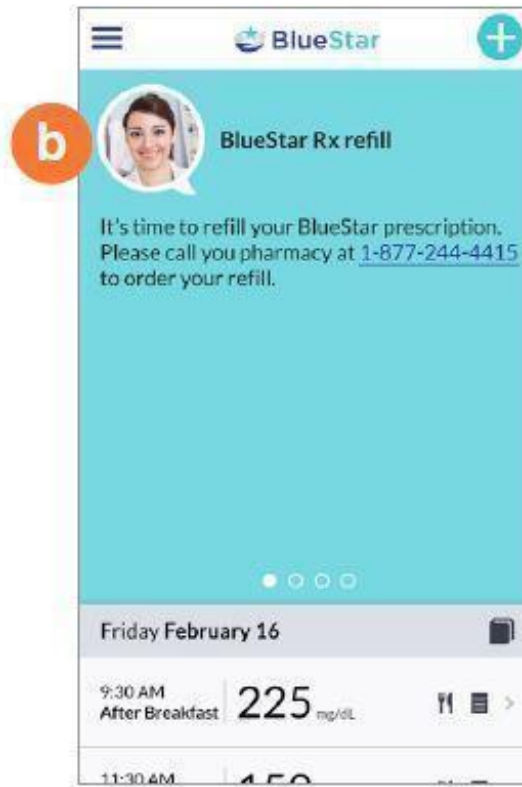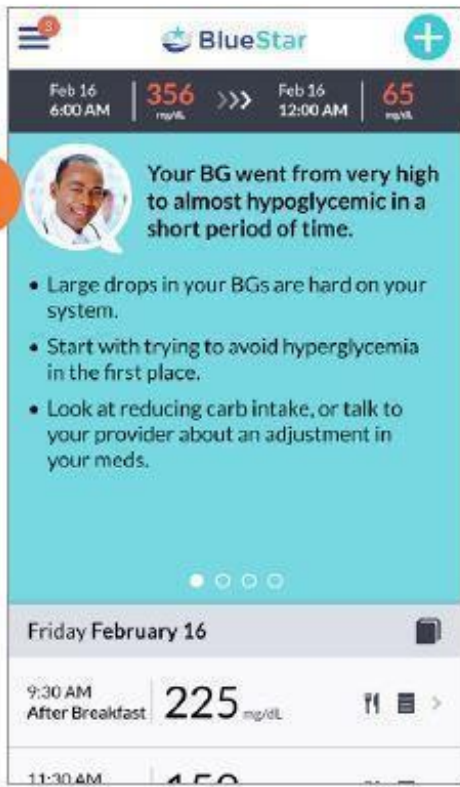


**Making entries**

**Entering your blood glucose when you test and the medications you take can help you stay on track with your diabetes treatment plan. In return, BlueStar gives you real-time guidance. Your experience is personalized so you understand more about your diabetes every day.**

**You can enter BG, carbs, exercise information, medications, and notes about your activities or how you are feeling each day. Below is the type of information you can input in BlueStar:**

- Blood Glucose (BG)—enter your numbers every time you test your BG
- Carbs—enter how many carbs you had in a meal.
- Date/Time—displays current Date and Time. You have the flexibility to adjust the date to view or edit entries from an earlier date
- Activity Type—(Before Breakfast, Before Lunch, etc.) Your activity type defaults to how your Daily Schedule is preset on the website. You can update your Daily Schedule activity times in Settings
- Record Medications—record all your prescribed, over-the-counter medications, vitamins, and supplements. BlueStar will personalize your guidance based on your diabetes medications
- Add a Note—add important information that relates to your BG entry

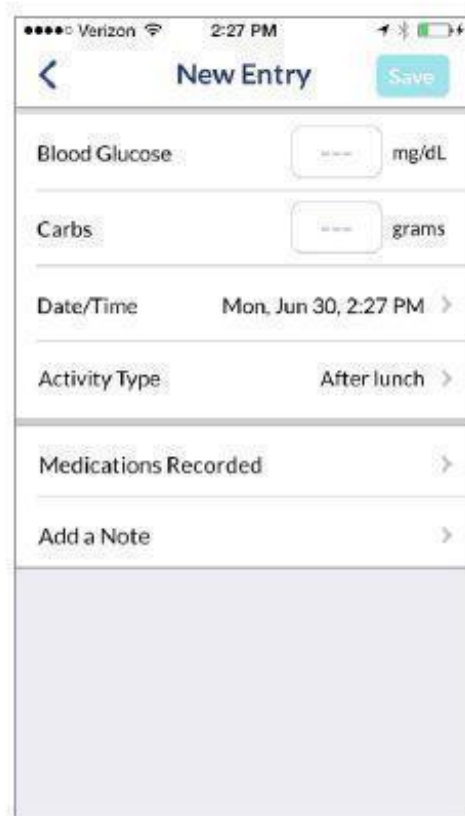

### □ Making entries (continued)

#### To make a New Entry


- Tap on **NEW ENTRY + ICON** top right from any screen
- Select **DATE AND TIME**.

Note: Will default to current date and time

- Select **ACTIVITY** (Before Breakfast, Fasting, Before Lunch, etc.)

Note: Will default to an activity based on your Daily Schedule

- Enter your Blood Glucose reading (BG) and Carbs (if applicable)

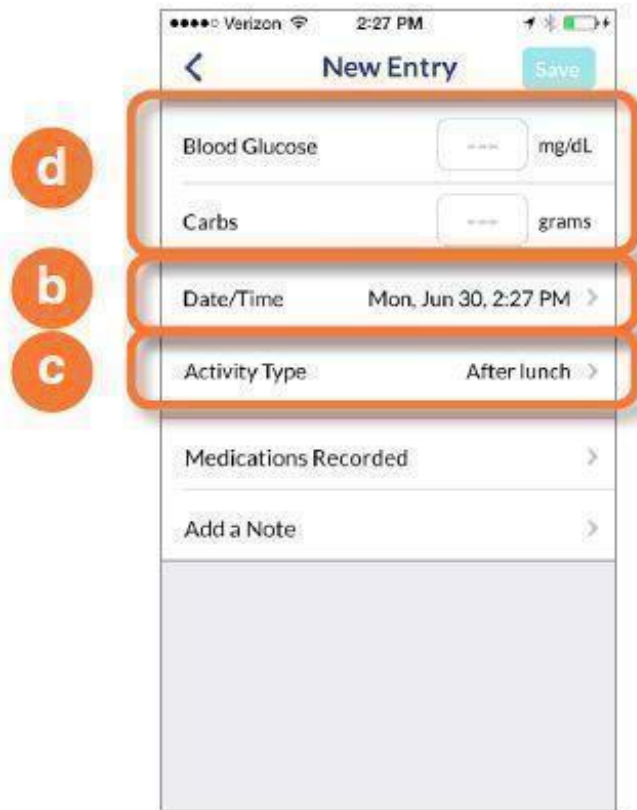

### Medications (Medication list)

This is where you can view your medication information. It is important to record all meds you are taking—prescription, over-the-counter, supplements, and vitamins. BlueStar will give you personalized feedback based on your diabetes medications.




- Your Medication List gives you a view of your medications and how many times you take them daily.

Note: Always continue to take your medications as prescribed and confirm your prescription information with your healthcare provider at your earliest convenience.



### **Add Medication:**

- a. Tap the + button and enter at least 3 letters to search for a med
- b. Select the medication you want to add
- c. Enter how often you take your med
- d. Enter the amount you take and save

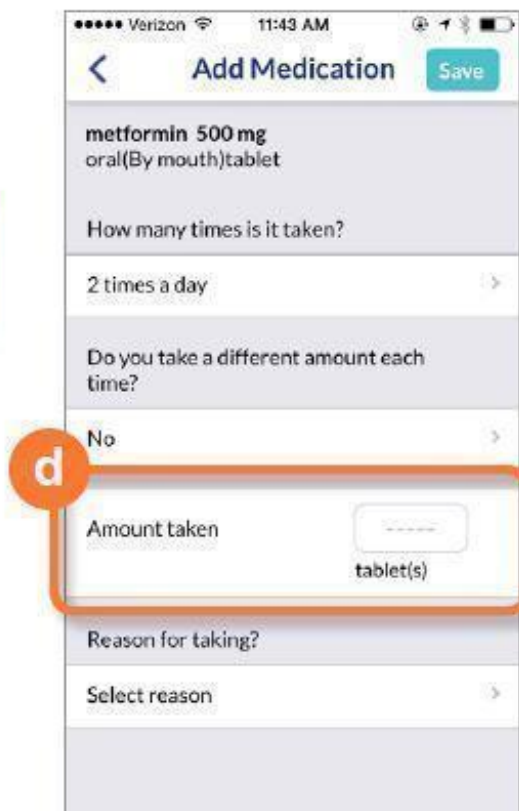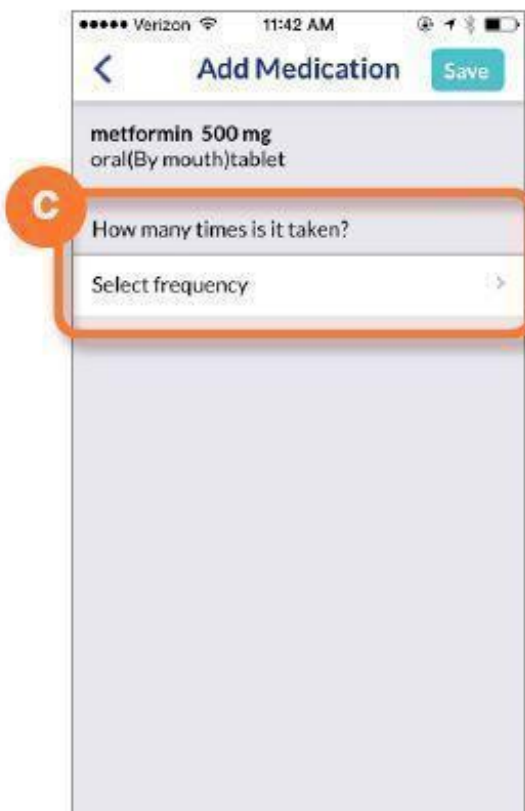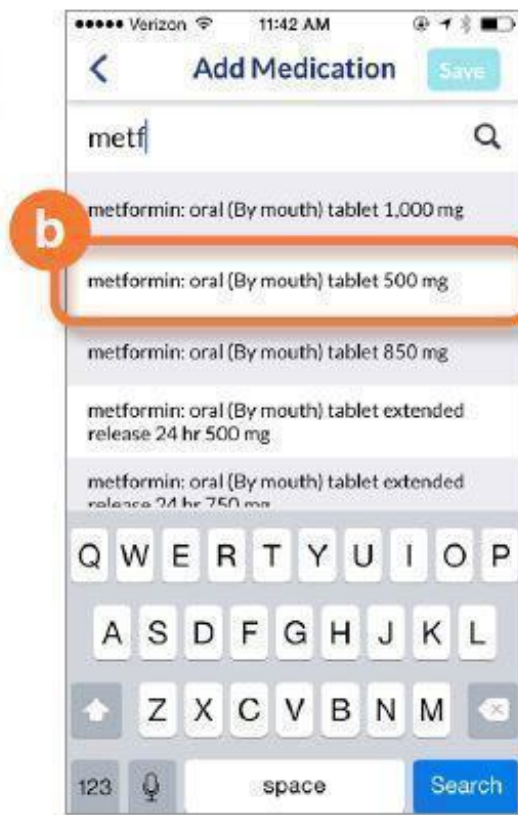


## Medications (Medication List) (continued)

Edit Medication:

You can change your dose, how often you take your med, or delete it from your list

- Tap the **PENCIL** icon next to the medication name
- Change how often you take your med or the amount you take
- Tap the **DELETE** button to remove the med from your list

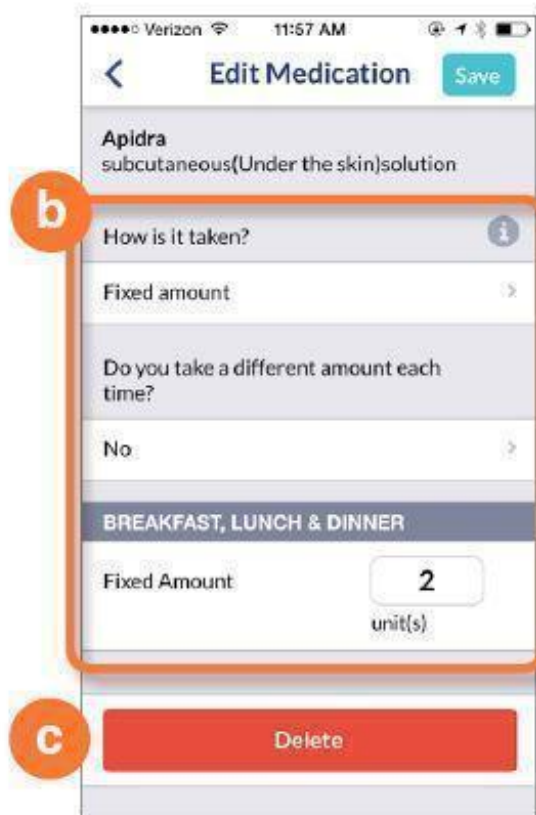

## Medications (Schedule Setup and Med Reminders)



Here is where you can change your medication schedule and set a reminder to take your med.

### Medication Schedule

Your medications are scheduled by Time of the Day (Morning, Noon, Evening and Bedtime or by Meal (Breakfast, Lunch or Dinner).)

### Change your med schedule:

- Tap the "arrow symbol"
- Select another Meal or Time of the Day to take your med and save. Your med schedule is now updated for you to record in your Daily Med Log

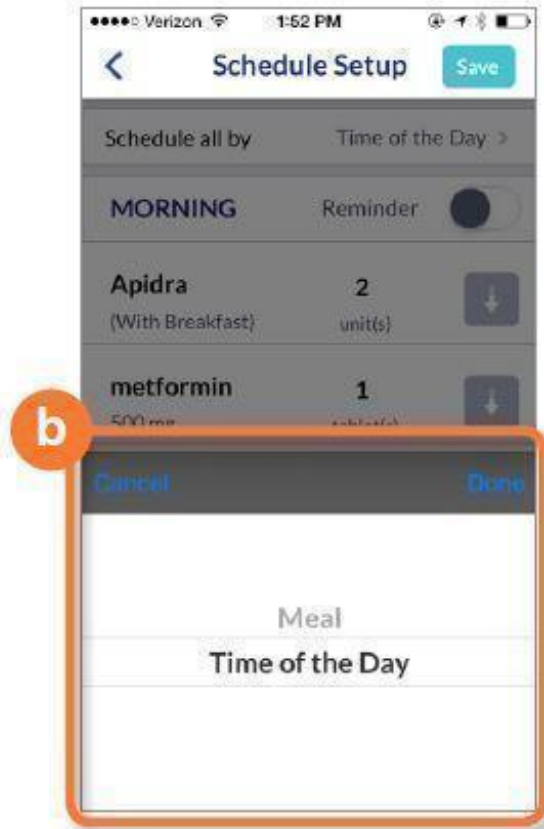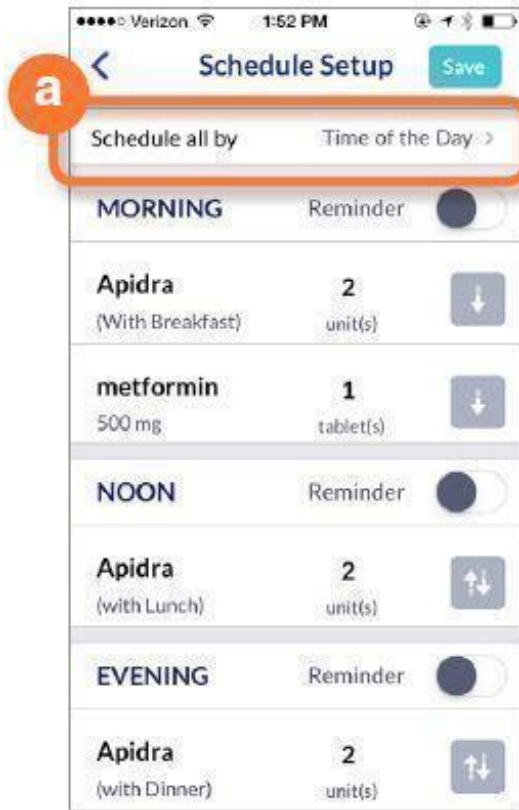

## Medication Reminders

You can set a reminder to take your medication.

### To set a reminder:

- a. Tap the slider next to “Reminder”
- b. Tap the time to change if needed

A notification will pop up on your phone when it is time for you to take your med






### Medications (Daily Med Log)

Your medications are displayed for a day according to your schedule (Meal or Time of the Day)



You can change the date if you want to record medications for an earlier day

- a. Tap the Date
- b. Change the day to a previous day

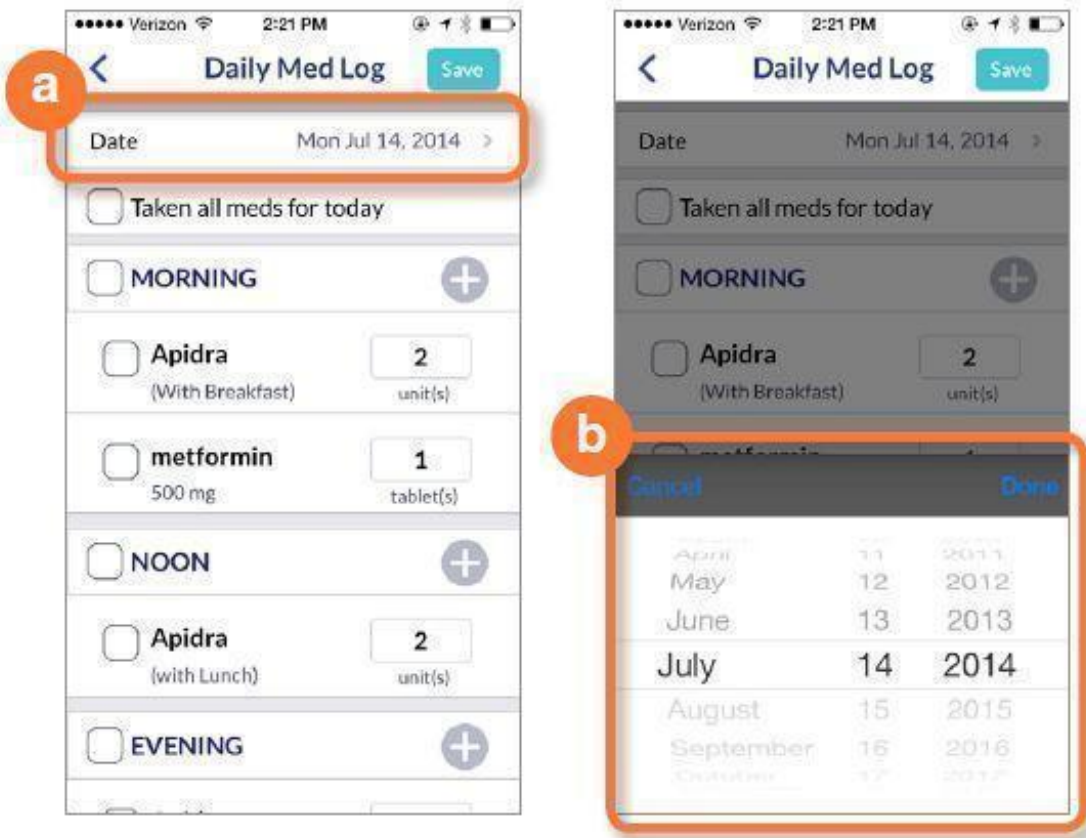

Tap to check you have taken all your meds for the day, by meal or time of day. You can also Tap one medication at a time.

- a. Tap the Menu button
- b. Select medications and then Daily Med Log
- c. Tap to check that the meds you have taken

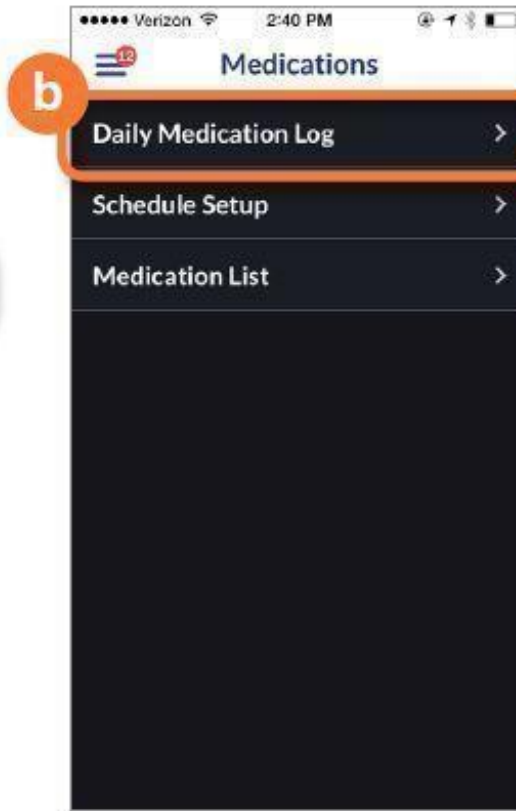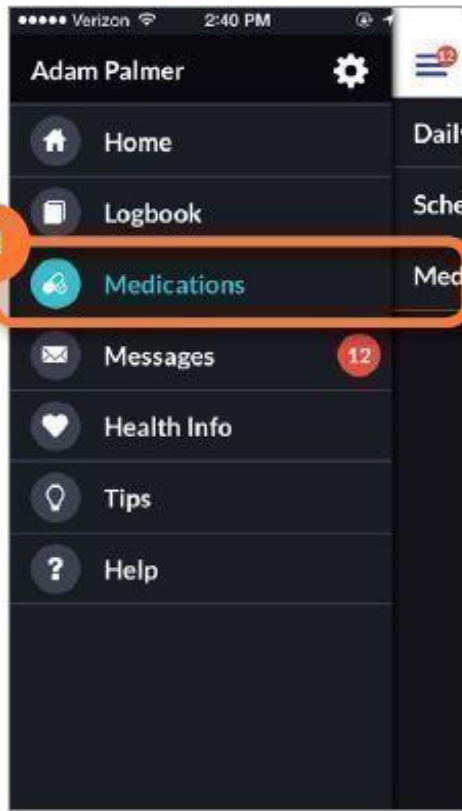

Verizon 2:41 PM

← Daily Med Log Save

Date Mon Jul 14, 2014 >

Taken all meds for today

MORNING +

Apidra 2  
(With Breakfast) unit(s)

metformin 1  
500 mg tablet(s)

NOON +

Apidra 2  
(with Lunch) unit(s)




EVENING +

### Medications (Daily Med Log) (continued)

You have the flexibility to add a dose of a med from your list or record a one-time dose of a med that is not on your list.

#### To add a dose of a new med:

- Tap + to add med
- Search for the med
- Enter the amount you have taken
- Tap **DONE**
- Tap **ADD TO PRESCRIPTION** to add the med to your list

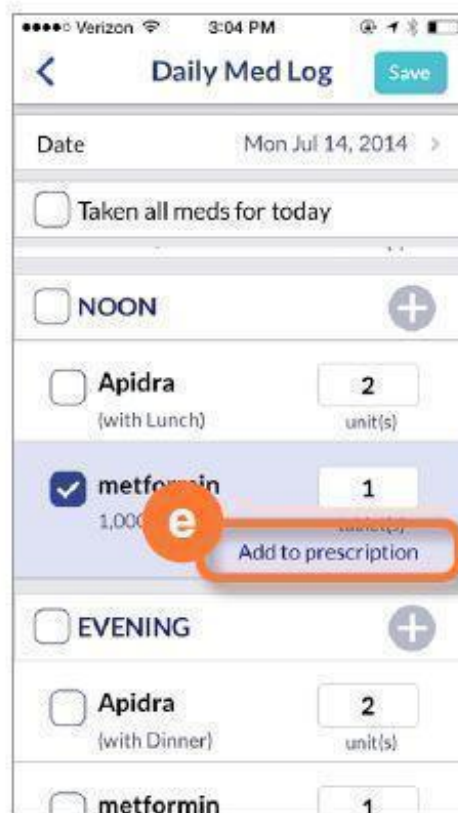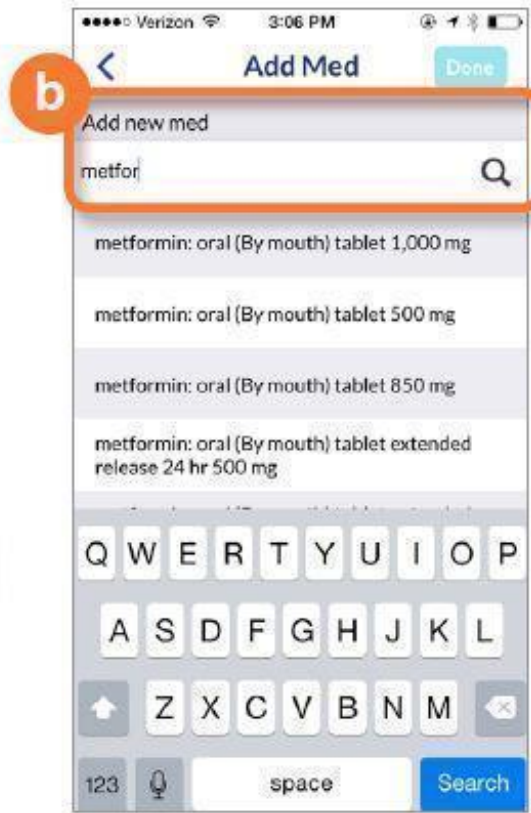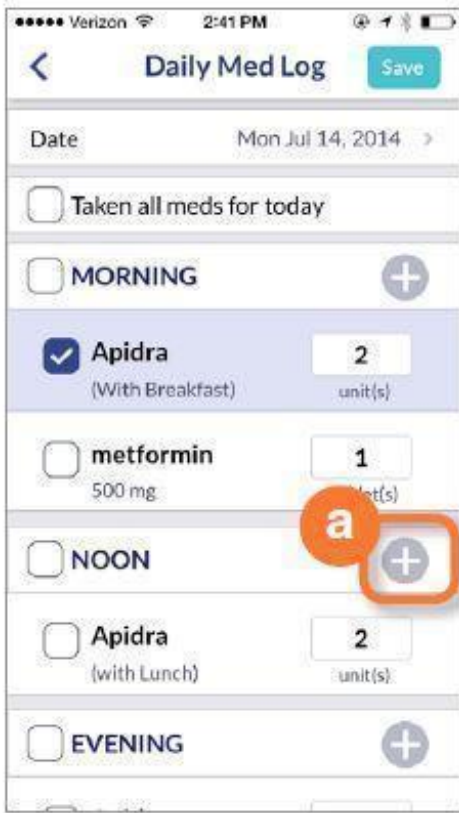

**Real-Time Guidance**

BlueStar supports you with instant guidance about any of your BGs that are entered in real time. BlueStar considers any BG that is entered within 2 hours of actually checking it to be a “real time” BG.

These messages can help you with:

- Immediate guidance on your BG value and how to correct it
- Information on nutrition
- Education about diabetes related health issues like blood pressure and cholesterol
- Messages to motivate and inspire you, including personal stories from other people with diabetes




## Messages

BlueStar will send you personal messages through the message center. These messages can cover a range of topics, from introducing new features in BlueStar to reminding you to share your health info with your doctor before your next office visit. It is an easy way to communicate to you without cluttering your personal email.

- a. The number displayed over the **MESSAGES ICON** shows you how many unread messages are in your Inbox. To view your messages, tap the **MESSAGES NUMBER ICON**
- b. Tap the message you wish to view. Tap **BACK ARROW** to return to list of messages or **DELETE** to remove the message from your Inbox

Note: Your BlueStar® messaging system is a read-only system. You cannot send or forward messages from it.

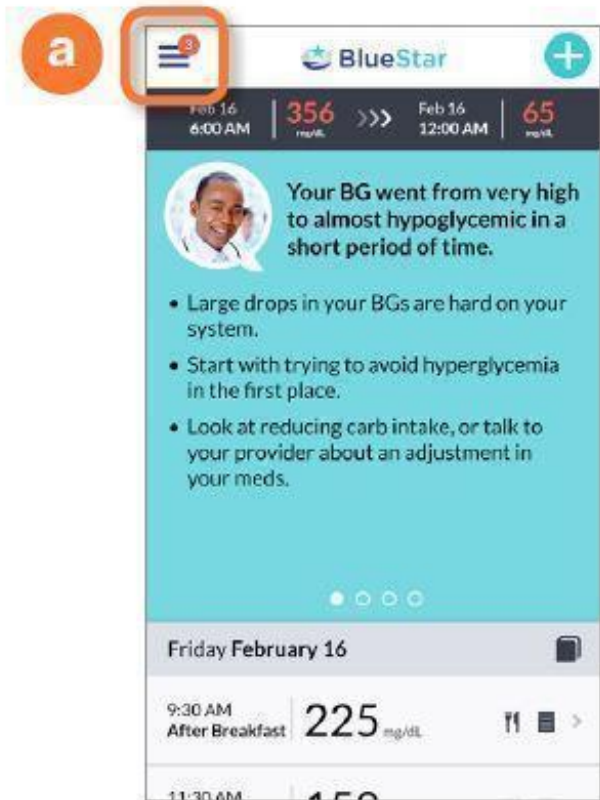


Logbook is a record of all your BG readings and related information such as carbs, meds, and notes you have entered into BlueStar®.

Be sure to check your BG targets set by your healthcare provider. You will see your targets listed right below dates and Activity Type.

**To make a new Logbook entry:**

- a. Tap the + icon top right
- b. Enter **BLOOD GLUCOSE** and **CARBS** for that meal
- c. Select current **DATE/TIME** or an earlier date
- d. Select **ACTIVITY TYPE**
- e. Select **ADD A NOTE** to add important information about your BG entry

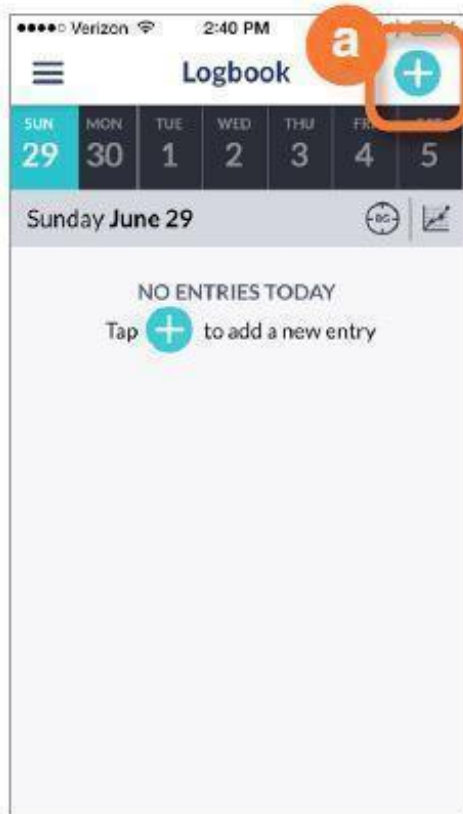


**Tap graph icon right screen to see how your BG numbers are trending.**

Note: Only the last thirty (30) days of data will appear in the logbook on your phone.

To edit any logbook entry, just click on it to make changes and then save them. If you would like to view your complete logbook display or print it, please Log In to the BlueStar website.

Logbook is where you can record and view your individual entries.



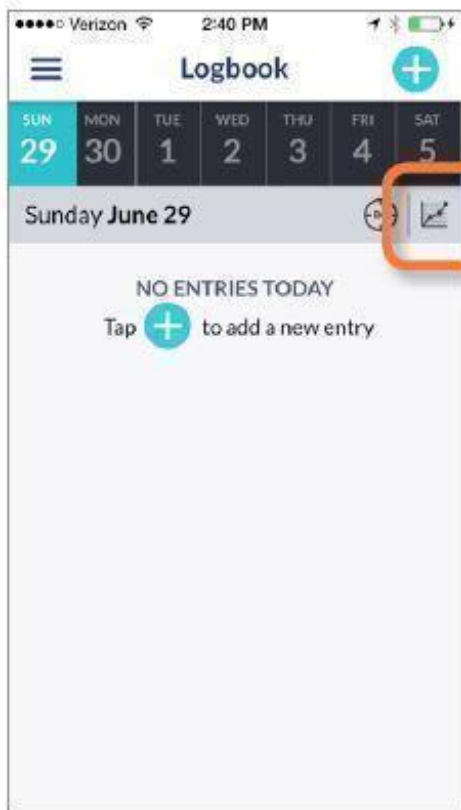

## Health Info

You need to be connected to the Internet to access Health Info and to add new entries to it. Health Info is where you can review your Medical History, Clinical Results, Preventive Health, Lifestyle, Provider, and Prescription information.

- Tap on **HEALTH INFO**
- Enter your PIN. If you have not created a PIN, BlueStar® will prompt you to create one
- Tap your selection from the **HEALTH INFO** menu

### **IMPORTANT SAFETY INFORMATION:**

**For medical questions, please contact your healthcare provider. If you are experiencing an emergency, please dial 911.**





### **Health Info (Medical History)**

**To complete your Medical History:**a. Select year diagnosed with Diabetes


b. Tap Yes or No to answer each of the medical history questions

Note: If you're not sure of the answer to some of the questions, be sure to discuss it with your healthcare provider.

Be sure to update your Medical History if there are changes to any diseases or conditions you may have.

#### **IMPORTANT SAFETY INFORMATION:**

**For medical questions, please contact your healthcare provider. If you are experiencing an emergency, please dial 911.**

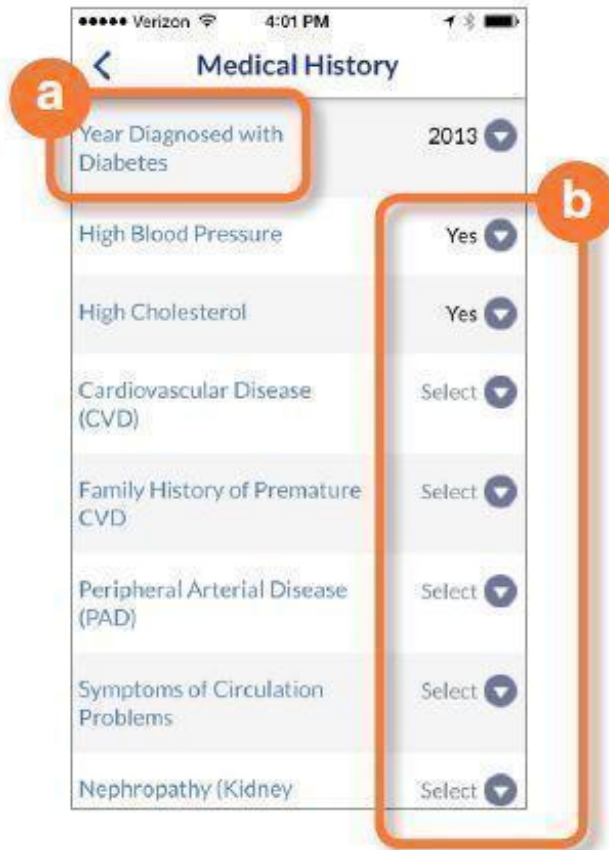

### **Health Info (Clinical Results)**

BlueStar lets you know if your numbers are in range, out of range, or you are overdue for a lab test.


**To enter Clinical Results:** a. Enter the numbers and dates for your current lab results and blood pressure check.

Remember to update your information each time you have new results.

If you're not sure of the answers to some questions, be sure to discuss these with your healthcare provider.

### **IMPORTANT SAFETY INFORMATION:**

**For medical questions, please contact your healthcare provider. If you are experiencing an emergency, please dial 911.**

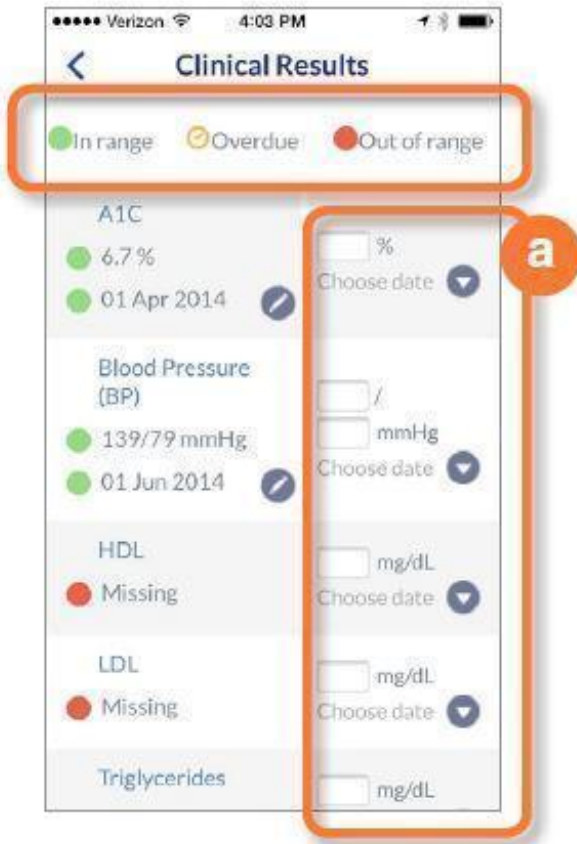

### Health Info (Preventive Health)

Preventive Health is a great tool for staying on track with exams and vaccinations that can help you manage your health. It lets you know if you are overdue or missing an item on the list.

a. Choose the date for each exam or vaccination you have had on the list

Remember to update your list whenever you have had a new exam or vaccination.



### **Health Info (Lifestyle)**

Lifestyle displays your weight, height, calculates your BMI, and your smoking history.

- a. Enter your current weight and choose date
- b. Enter height in feet and inches and choose date
- c. Your BMI is calculated automatically from your weight and height
- d. Tap to enter your smoking history



### Health Info (Provider)


You can review your healthcare provider information here and send a SMART Visit to your provider. To make changes to your healthcare provider information, you must call BlueStar® Customer Care at (888) 611-4794. Available: Monday-Friday, 8:00 a.m.-7:00 p.m. (ET).

### What is a SMART Visit?

SMART Visit is a summary of the information you entered into BlueStar you can send to your healthcare provider. It includes your BGs, lab results, screening tests dates, vaccinations, and medications.

Send SMART Visit 2-3 days prior to an appointment so your provider will have the information to review with you during the visit.

a. Tap the Smart Visit button to send your information to your provider.

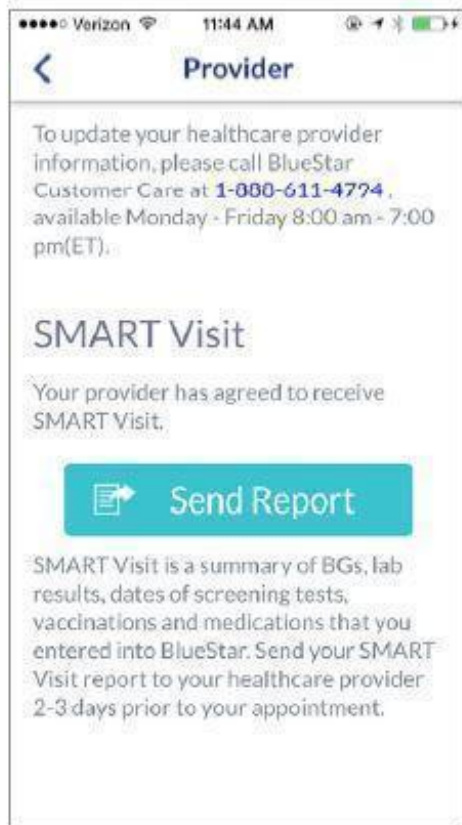

### **Health Info (Prescription Information)**


Prescription Information lets you know when your BlueStar prescription will expire along with helpful information about refills.







## Tips

Tips can help you build your knowledge and get useful tips about your health, nutrition, exercise, and BG.

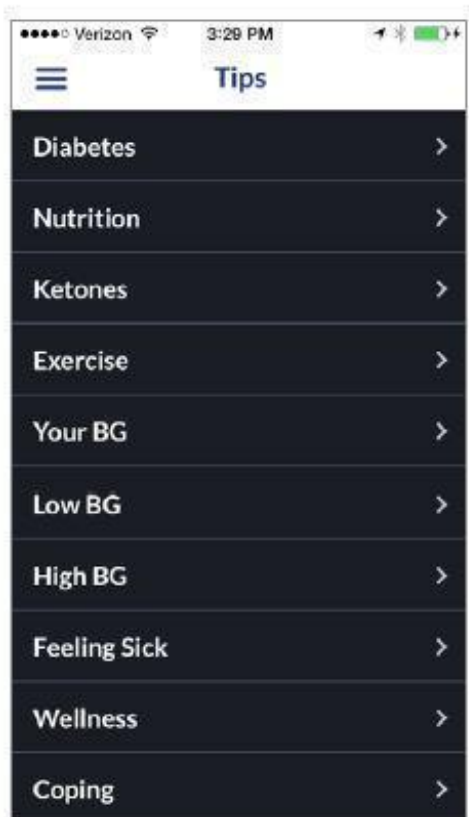


Tap your selection from the **TIPS** menu. Most of the menu categories also have sub-menu topics to select from.

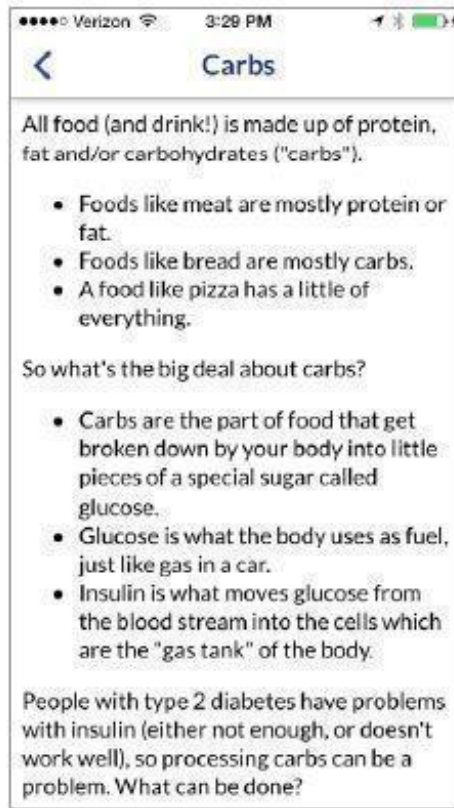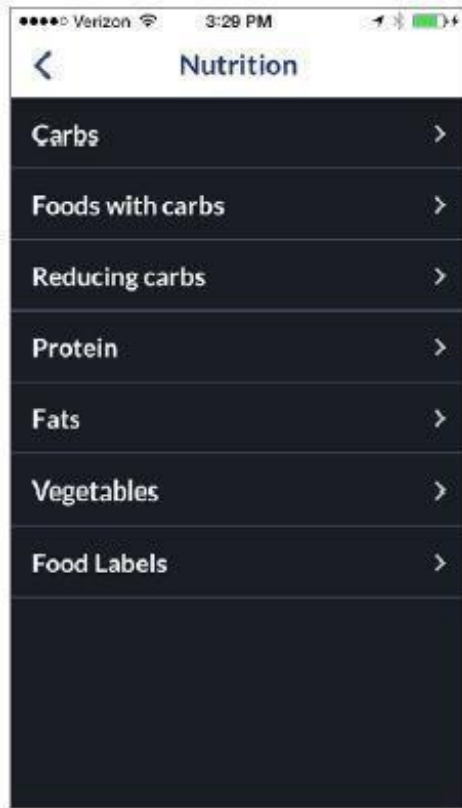

## Help

**HELP** is here to help you make the most of BlueStar to stay on track with your diabetes care plan. If you are new to BlueStar, Tour or Overview are great places to start.

- Virtual training—learn to setup your medication profile and enter diabetes data
- Welcome video—from the founder of BlueStar
- Overview—explains each main section of BlueStar
- Tour—takes you on a tour of how BlueStar works
- New Entry—explains what each entry category is for
- About BlueStar—tells you what version you are using of the mobile app
- Privacy Policy
- Tap the Customer Care number to speak with a BlueStar customer care representative. Available: Monday-Friday, 8:00 a.m.-7:00 p.m. (ET).


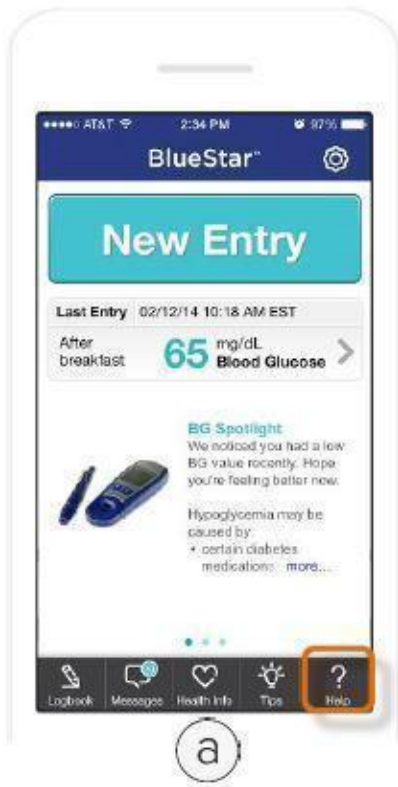


## Settings

Use PIN Settings to create or change your PIN number or lock BlueStar®. You will need to use a PIN to access your Health Info.


### To create or change your PIN:

- Tap on the **COG** icon
- Tap on **CREATE PIN** or **CHANGE PIN**
- Enter your current PIN on dotted line
- Enter new PIN on dotted line then re-enter

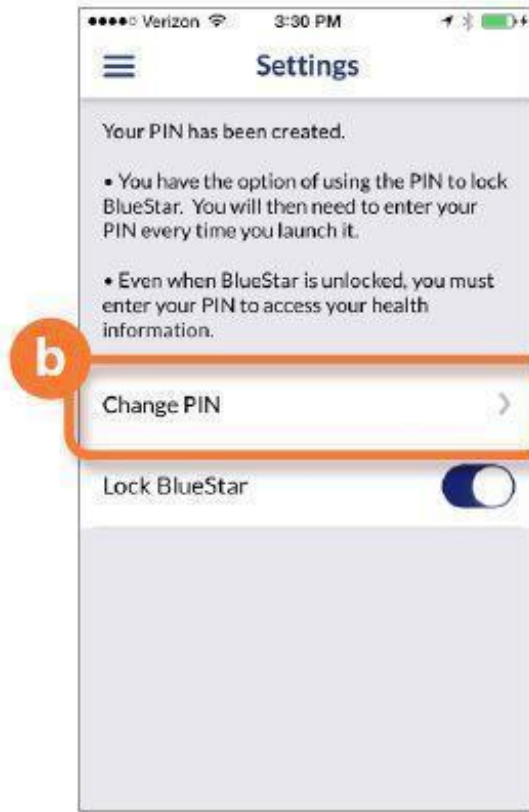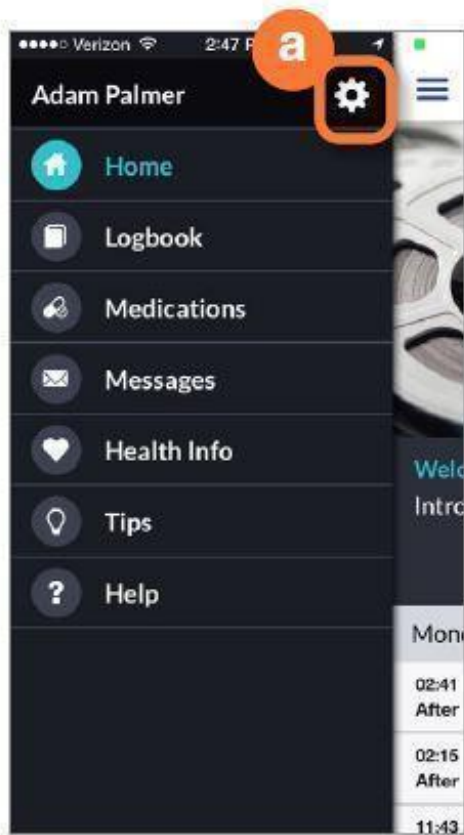

**To lock BlueStar:**

a. Move the **LOCK BLUESTAR** slider control to the right to lock your application

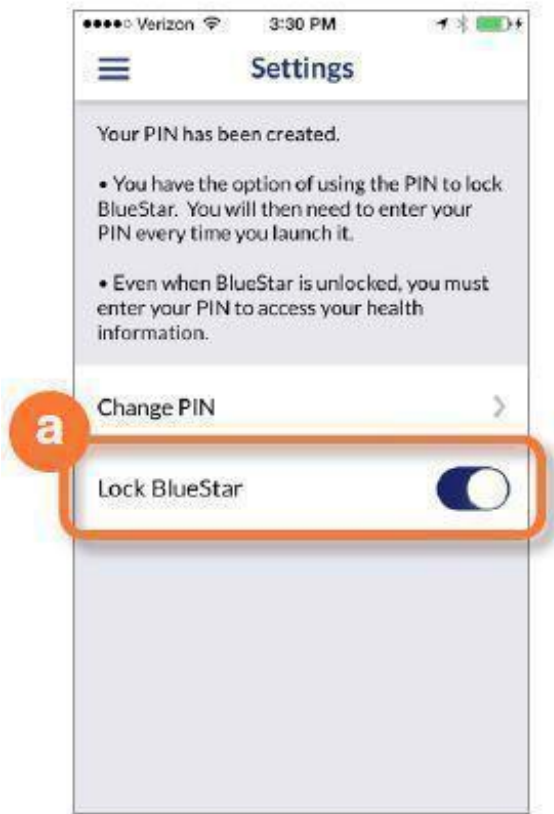

## Using BlueStar on the Web



BlueStar® is recommended for use on the following web browsers:

Internet Explorer 10 and 11  
Mozilla Firefox 31 and 32,  
Google Chrome 37  
Safari 7.0.6

BlueStar® is recommended for use on the following tablets:

iPad Retina/Air - 2048x1536  
iPad Mini - 1024x768  
Samsung tab 10.1 - 1280x800  
Samsung tab 7.0 - 1024x600

## Log In


BlueStar® is the first prescription app for adults with Type 2 diabetes. BlueStar gives you real-time guidance that makes managing diabetes between office visits easier and more doable. BlueStar must be prescribed by a provider.

## Log In

Enter your Username and Password then click **LOG IN**

## Trouble logging in?

Click **FORGOT PASSWORD?** to retrieve your BlueStar password or **FORGOT USERNAME?** to retrieve your username

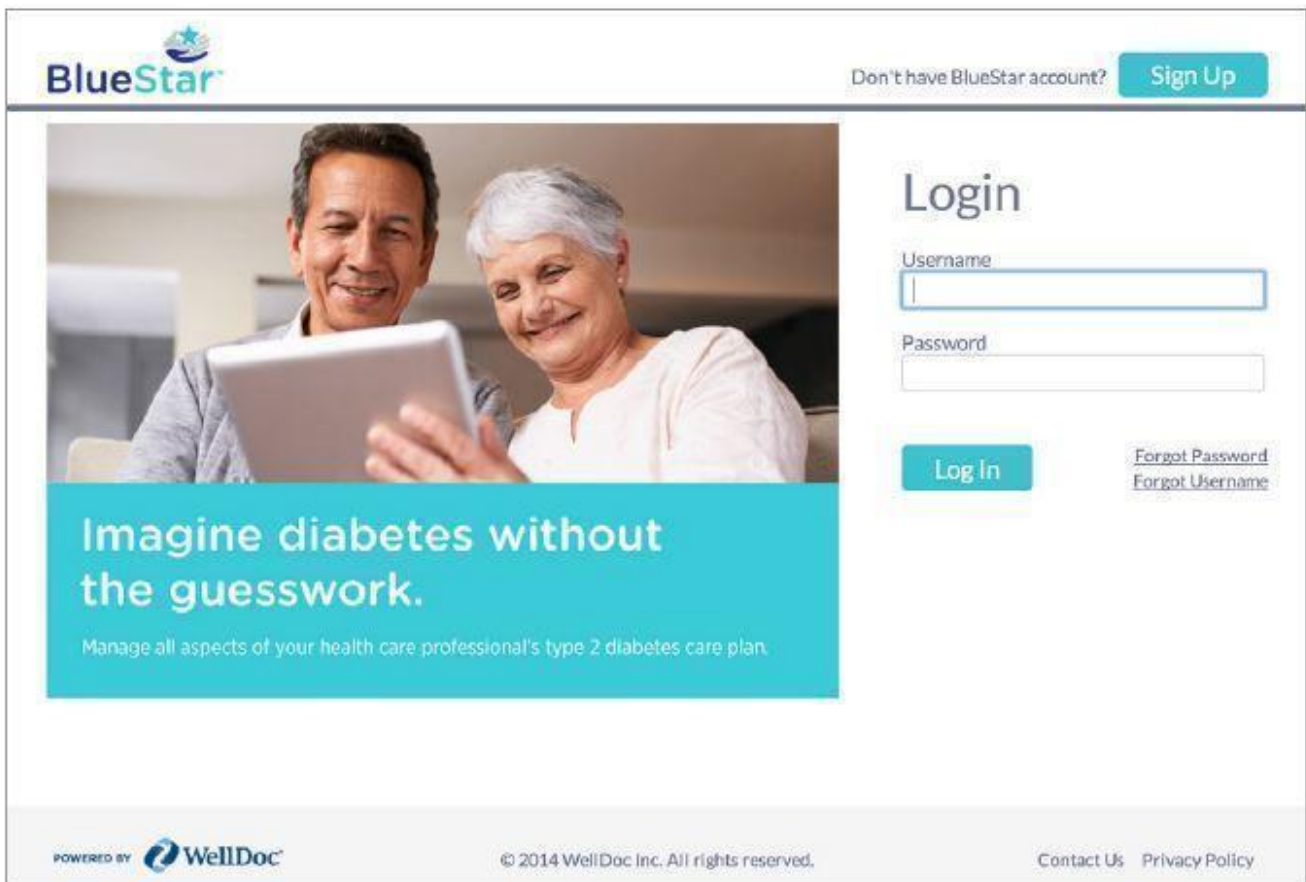

BlueStar

Don't have BlueStar account? [Sign Up](#)

## Login


Username

Password

[Log In](#) [Forgot Password](#)  
[Forgot Username](#)

**Imagine diabetes without the guesswork.**

Manage all aspects of your health care professional's type 2 diabetes care plan.

POWERED BY  WellDoc

© 2014 WellDoc Inc. All rights reserved. [Contact Us](#) [Privacy Policy](#)

## Home Screen


The BlueStar® home screen carousel displays important information to help you manage your diabetes

and build your knowledge.

The carousel on the BlueStar home screen displays your last 4 BG entries. When BlueStar sees something interesting about your BG, a message is delivered to your carousel and logbook for that day.



Get a daily boost with a new message to motivate you each day about your general health and well-being.

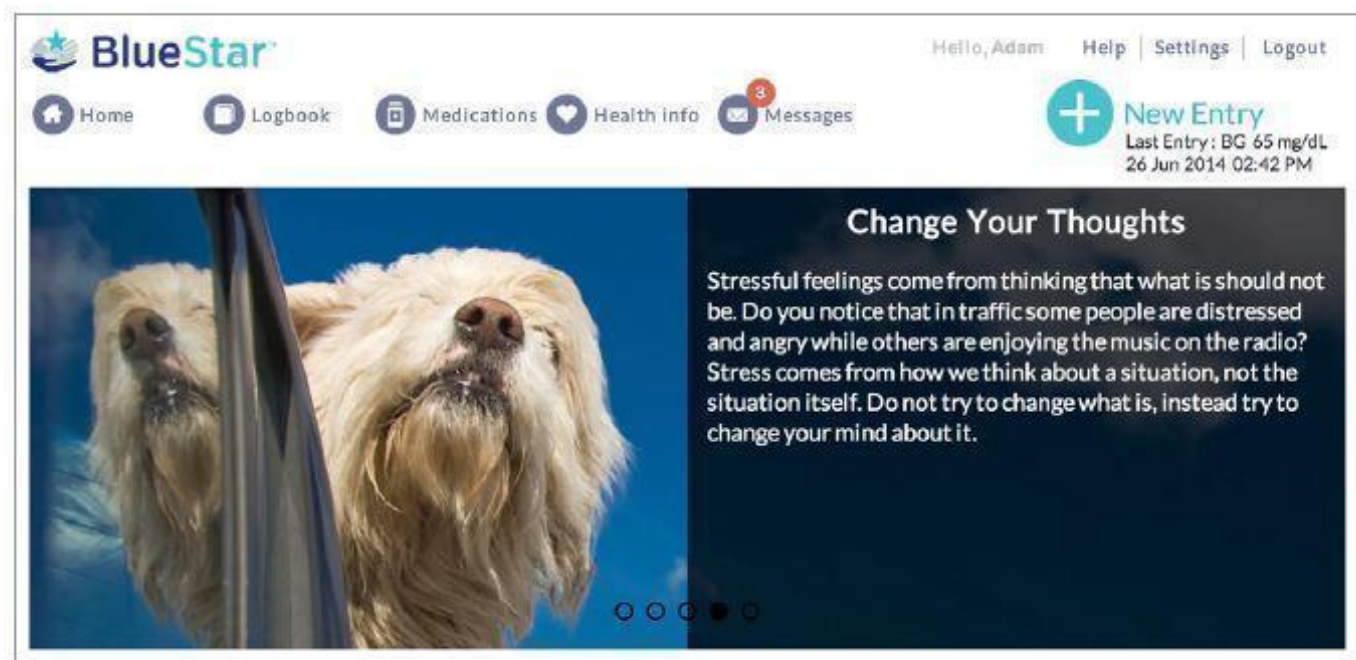

Short, 2- to 5-minute educational videos are designed to build your knowledge about diabetes and fit into your busy schedule. Each video is available for 48 hours before it is replaced by a new one.





### Home Screen (continued)

Check for renewal reminders for your BlueStar prescription on the carousel.



a. Health-related tips and reminders for completing your Profile

b. Profile status shows how much of your Health Info Profile is complete and what sections may need completing. Click on the pencil icon to complete any section. Or click on **HEALTH INFO** from the main menu to complete your profile.

Keeping your Clinical Results, Preventive Health, Medical History, and Lifestyle information up to date will help us customize BlueStar™ for you.

c. SMART Visit is a summary of the information you entered into BlueStar you can send to your healthcare provider. It includes your BGs, lab results, screening tests dates, vaccinations, and

medications.

Send SMART Visit 2-3 days prior to an appointment so your provider will have the information to review with you during the visit. Click on the **SMART VISIT** button to send your information.

**a** Take action to gain traction on your diabetes!

- Stay on track with your diet and physical activity - So if it has a drive through window, *AVOID IT!*
- Go for healthcare visits - As Ben Franklin said, "An ounce of prevention is worth a pound of cure!"
- Check your BGs readings - *And remember you're never just a number to us.*

**b** Your Profile is **62%** complete.

Medical History	100%	
Clinical Results	12%	
Preventive Health	62%	
Lifestyle	67%	

**c** Send an update to your doctor through SMART Visit

SMART Visit is a summary of BGs, lab results, dates of screening tests, vaccinations and medications that you entered into BlueStar. Send your SMART Visit report to your healthcare provider 2-3 days prior to your appointment.

## Making Entries

**Entering your blood glucose and medications can help you stay on track with your diabetes treatment plan. That way BlueStar can personalize real-time guidance for you every day.**


You can enter information for any of the following categories:

- **Blood Glucose (BG):** enter your numbers after you test yourself
- **Carbs:** enter how many carbs you had in a meal
- **Date/Time:** displays current Date and Time. You have the flexibility to adjust the date/time to enter entries from an earlier date
- **Activity Type:** (Before Breakfast, Fasting, Before Lunch, etc.) This defaults to how your Daily Schedule is preset. You can update your Daily Schedule activity times in Settings
- **Record Medications:** record all your prescribed, over-the-counter medications, vitamins, and supplements. BlueStar will personalize your guidance based on your diabetes medications
- **Add a Note:** add important information that relates to your BG entry

### To make a New Entry:

- a. Click on New Entry + icon top right from any screen
- b. Select **DATE AND TIME**. Note: Will default to current date and time
- c. Select **ACTIVITY** (Breakfast, Fasting, Before Lunch, etc.) Note: Will default to an activity based on your Daily Schedule
- d. Enter your **BLOOD GLUCOSE** reading (BG) and **CARBS** (if applicable)
- e. Click on **TAKEN ALL MEDS FOR THE DAY** or individual medications you took for that meal or time of the day
- f. Click on **ADD A NOTE** to add information about your BG entry

g. Click on **SAVE** to save your entry information

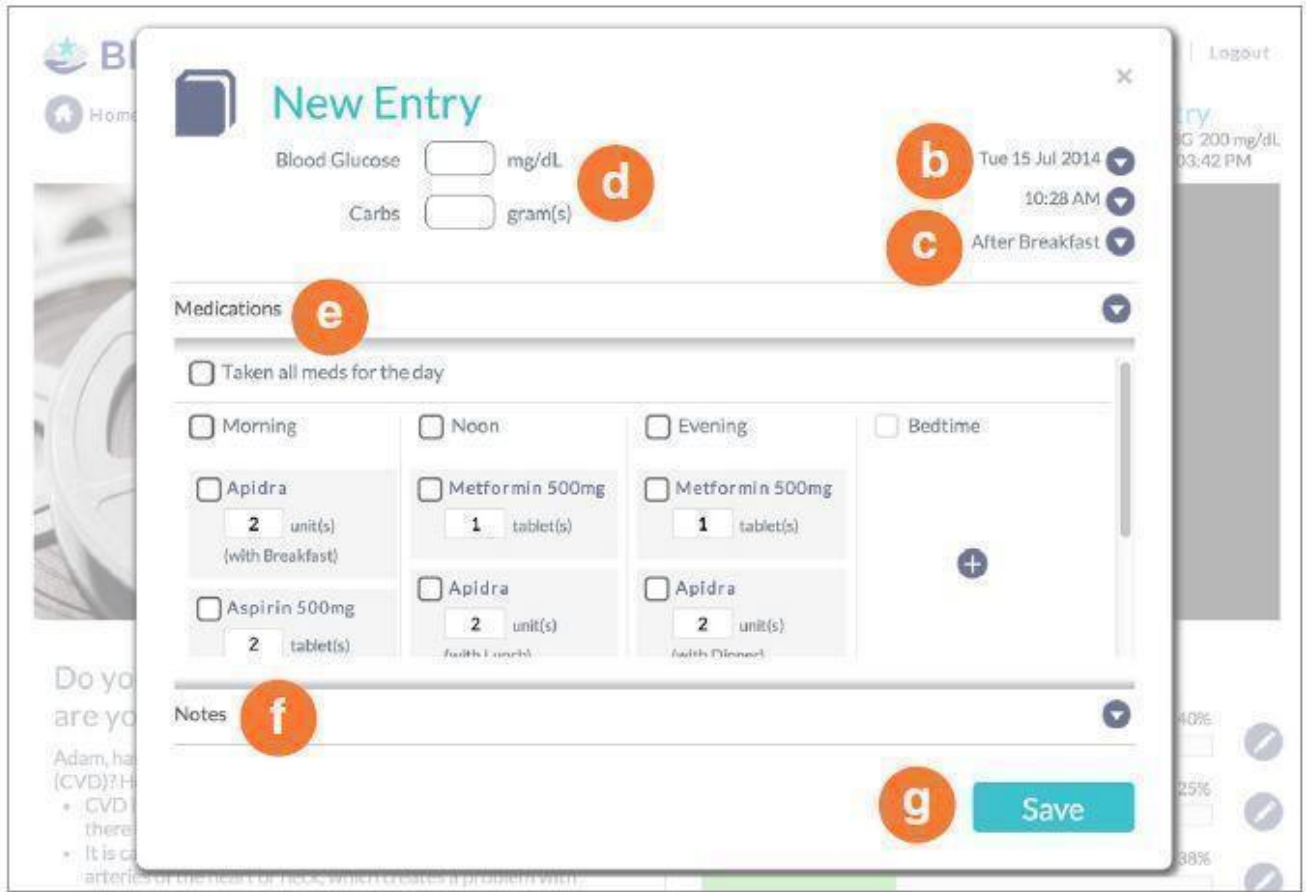

### Medications (Medication List)

This is where you can view your medication information. It is important to record all meds you are taking—prescription, over-the-counter, supplements, and vitamins. BlueStar will personalize your guidance, based on your diabetes medications.

- Your Medication List gives you a view of your medications and how many times you take them daily. You can add a new med to your list, make changes to a med, or delete a med from your list

Note: Always continue to take your medications as prescribed and confirm your prescription information with your healthcare provider at your earliest convenience.

#### Add a new medication:

- Enter at least 3 letters to search for a med
- Select the medication you want to add
- Enter how often you take your med
- Enter the amount you take and save

**Add Medication**

metformin: oral (By mouth) tablet 850 mg

Reason for taking Select

How often do you take it? 1 time a day

How much do you take? tablet(s)

Save

### **Edit medication:**

You can change the strength of your med, your dose, how often you take your med, or delete it from your list

- a. Click pencil icon next to the medication name
- b. Change the arrow by the med name to change the strength
- c. Click the arrow to change the reason you take your med, how often you take it, or the amount you take
- d. Click the delete button to remove the med from you list
- e. Click save

### Medications (Schedule Setup)

Your medications are scheduled by Time of the Day (Morning, Noon, Evening and Bedtime or by Meal (Breakfast, Lunch or Dinner)

#### Change your med schedule:

a. Change your schedule by dragging your med to another time of day or meal type. Your med schedule is now updated for you to record in your Daily Med Log



# Medications

Daily Med Log

Schedule Setup

Medication List

## Setup your Medication Schedule

Save

Drag medications to move them to different time slots. Add medications and change doses in the [Medication List](#)

Schedule By: Time of the Day

**a** Morning 08:00 AM

Metformin 500mg  
1 tablet(s)

Apidra  
2 unit(s)  
(with Breakfast)

Aspirin 500mg  
2 tablet(s)

Drag and Drop Med here

Noon

Apidra  
2 unit(s)  
(with Lunch)

Metformin 1,000mg  
1 tablet(s)

Drag and Drop Med here

Evening

Metformin 500mg  
1 tablet(s)

Apidra  
2 unit(s)  
(with Dinner)

Aspirin 500mg  
2 tablet(s)

Drag and Drop Med here

Bedtime

Drag and Drop Med here

### Medication reminders:

a. You can set a reminder to take your medication. Click on the bell icon by the meal type or time of day. Select the reminder time and save. A notification will pop up on your phone when it is time for you to take your med

**Medications**

Daily Med Log | Schedule Setup | Medication List

### Setup your Medication Schedule

Drag medications to move them to different time slots. Add medications and change doses in the [Medication List](#)

Schedule By: Time of the Day

**a** Morning 08:00 AM

- Metformin 500mg  
1 tablet(s)
- Apidra  
2 unit(s)  
(with Breakfast)
- Aspirin 500mg  
2 tablet(s)
- Drag and Drop Med here

Noon

- Apidra  
2 unit(s)  
(with Lunch)
- Metformin 1,000mg  
1 tablet(s)
- Drag and Drop Med here

Evening

- Metformin 500mg  
1 tablet(s)
- Apidra  
2 unit(s)  
(with Dinner)
- Aspirin 500mg  
2 tablet(s)
- Drag and Drop Med here

Bedtime

Drag and Drop Med here

## Medications (Daily Med Log)

Your medications are displayed for a day according to your schedule (Meal or Time of Day)

You can change the date if you want to record medications for an earlier day

a. Select the arrow by the date

b Use to calendar to change the day to a previous day

**Medications**

Daily Med Log | Schedule Setup | Medication List

### Log your medications for 14 Jul 2014

Save

Taken all meds for the day

Morning

- Apidra  
2 unit(s)  
(with Breakfast)
- Aspirin 500mg

Noon

Bedtime

**a** 14 Jul 2014

**b** Calendar pop-up showing 14 Jul 2014 selected.

Check that you have taken all your meds for the day, by meal or time of day. You can also record one medication at a time.

- a. Click Medications
- b. Daily Med Log
- c. Check the medications you have taken

You have the flexibility to add a dose of med from your list or record a one-time dose of a med that is not on your med list

The screenshot displays the BlueStar Medications interface. At the top, the navigation bar includes 'Home', 'Logbook', 'Medications' (highlighted with 'a'), 'Health info', and 'Messages'. A 'New Entry' section shows 'Last Entry: BG 200 mg/dL 02 Jul 2014 03:42 PM'. The main heading is 'Medications', with 'Daily Med Log' (highlighted with 'b') selected. Below this, the date '14 Jul 2014' is set, and a 'Save' button is present. A summary row indicates 'Taken all meds for the day' (unchecked) and '2 medication(s) recorded.'. The main area is a grid with columns for 'Morning', 'Noon', 'Evening', and 'Bedtime'. Under 'Morning', 'Apidra' (2 unit(s) with Breakfast) is checked (highlighted with 'c'), and 'Aspirin 500mg' (2 tablet(s)) is unchecked. Under 'Noon', 'Metformin 500mg' (1 tablet(s)), 'Apidra' (2 unit(s) with Lunch), and 'Metformin 1,000mg' (1 tablet(s)) are unchecked, while 'Metformin 1,000mg' is checked. Under 'Evening', 'Metformin 500mg' (1 tablet(s)), 'Apidra' (2 unit(s) with Dinner), and 'Aspirin 500mg' (2 tablet(s)) are unchecked. The 'Bedtime' column is empty. Plus signs are visible at the bottom of each column to add more medications.

### Medications (Daily Med Log) (continued)

Add a dose of med from your list:

- a. Click + in the meal or time of day
- b. Select a med from your list
- c. Click to check that you have taken your med



### Medications (Daily Med Log) (continued)

#### To add a one-time dose of a new med:

- a. Search for the med
- b. Select to add med
- c. Enter the amount you have taken
- d. Click OK
- e. Click add to prescription to add the med to your list

Last Entry: BG 200 mg/dL  
02 Jul 2014 03:42 PM

### Medication List

Search for medication

**a** **b**

Metformin 500mg  tablet(s) Daily

Metformin 1,000mg  tablet(s) Daily

Log your medication

Taken all meds for

Morning:

Apidra

Last Entry: BG 200 mg/dL  
02 Jul 2014 03:42 PM

### Medication List

Search for medication

Metformin 500mg  tablet(s) Daily

Metformin 1,000mg  tablet(s) Daily

Tylenol 325mg  tablet(s)

**c** **d**

Log your medication

Taken all meds for

Morning:

Apidra  unit(s) (with Breakfast)

Aspirin 500mg  tablet(s)

Metformin 1,000mg  tablet(s)

**Medications**

Daily Med Log | Schedule Setup | Medication List

Log your medications for **14 Jul 2014** Save

Taken all meds for the day 3 medication(s) recorded.

Morning	Noon	Evening	Bedtime
<input checked="" type="checkbox"/> Apidra 2 unit(s) (with Breakfast)	<input type="checkbox"/> Metformin 500mg 1 tablet(s)	<input type="checkbox"/> Metformin 500mg 1 tablet(s)	
<input type="checkbox"/> Aspirin 500mg 2 tablet(s)	<input type="checkbox"/> Apidra 2 unit(s) (with Lunch)	<input type="checkbox"/> Apidra 2 unit(s) (with Dinner)	
<input checked="" type="checkbox"/> Tylenol 325mg 1 tablet(s) <span>Add To Prescription</span>	<input type="checkbox"/> Metformin 1,000mg 1 tablet(s)	<input type="checkbox"/> Aspirin 500mg 2 tablet(s)	
	<input checked="" type="checkbox"/> Metformin 1,000mg -- tablet(s)		

## Real-Time Guidance

BlueStar supports you with instant guidance about any of your BGs that are entered in real time. BlueStar considers any BG that is entered within 2 hours of actually checking it to be a “real time” BG.

These messages can help you with:

- Immediate guidance on your BG value and how to correct it
- Information on nutrition
- Education about diabetes related health issues like blood pressure and cholesterol
- Messages to motivate and inspire you, including personal stories from other people with diabetes

## GREAT JOB-KEEP IT UP



Doing multiple injections and BG checks every day isn't easy, Adam, but we're here to support you. Checking in pairs, before and after meals, helps you get the most out of your insulin and avoid highs and lows.

Close


### Messages

BlueStar will send you personal messages through the message center. These messages can cover a range of topics, from introducing new features in BlueStar to reminding you to share your health info with your doctor before your next office visit. It is an easy way to communicate to you without cluttering your personal email.

#### To view your messages:

- Click on **MESSAGES** at the top of the screen
- Review your list of messages and click on the message you would like to read
- Click **X** icon to delete a message

Note: Your BlueStar™ messaging system is a read-only system. You cannot send or forward messages from it

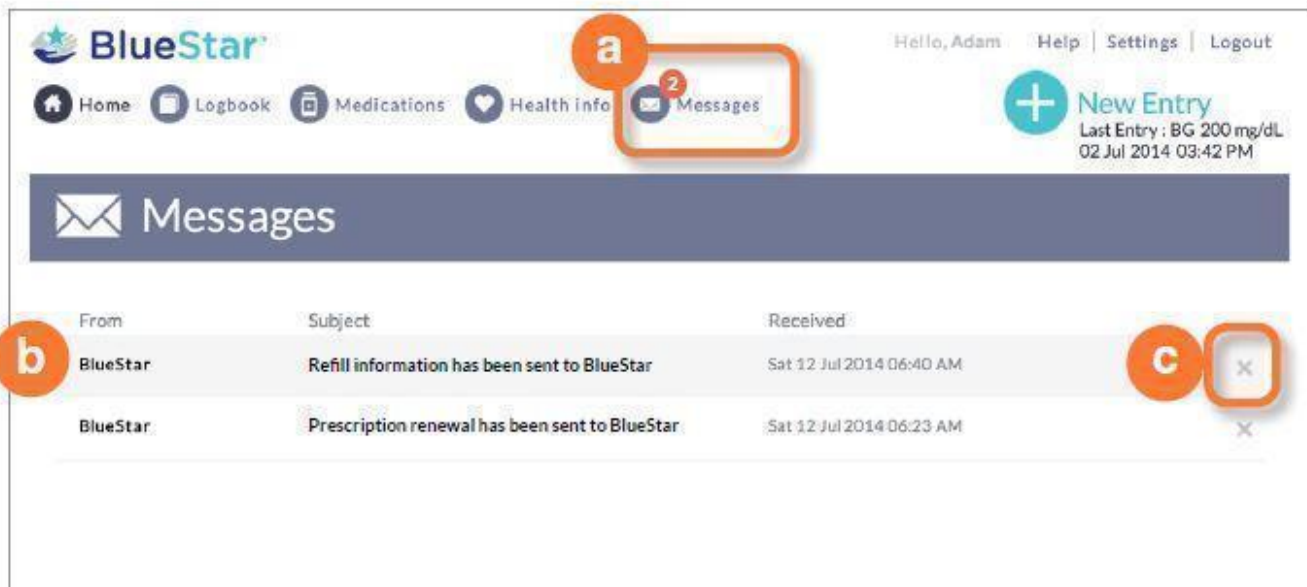

## Logbook

Logbook is a record of all your BG readings and related information such as carbs, meds, and notes you have entered into BlueStar. You can view and edit your entries here.

Be sure to check your BG targets set by your healthcare provider. You will see your targets listed right below dates and Activity Type.

### To view your entries :

- Click on date ranges to set the date range you want to view
- Click on Activity Type to change the view to view by time

### To make a new entry:


- Click on any empty square on your Logbook to add an entry for that date and activity type or click on the + icon top right screen

### To edit Logbook entries:

- Click on any entry to make changes, including entries for earlier dates

### To view Trending Graph:

- Click Graph top right screen to see how your BG numbers are trending

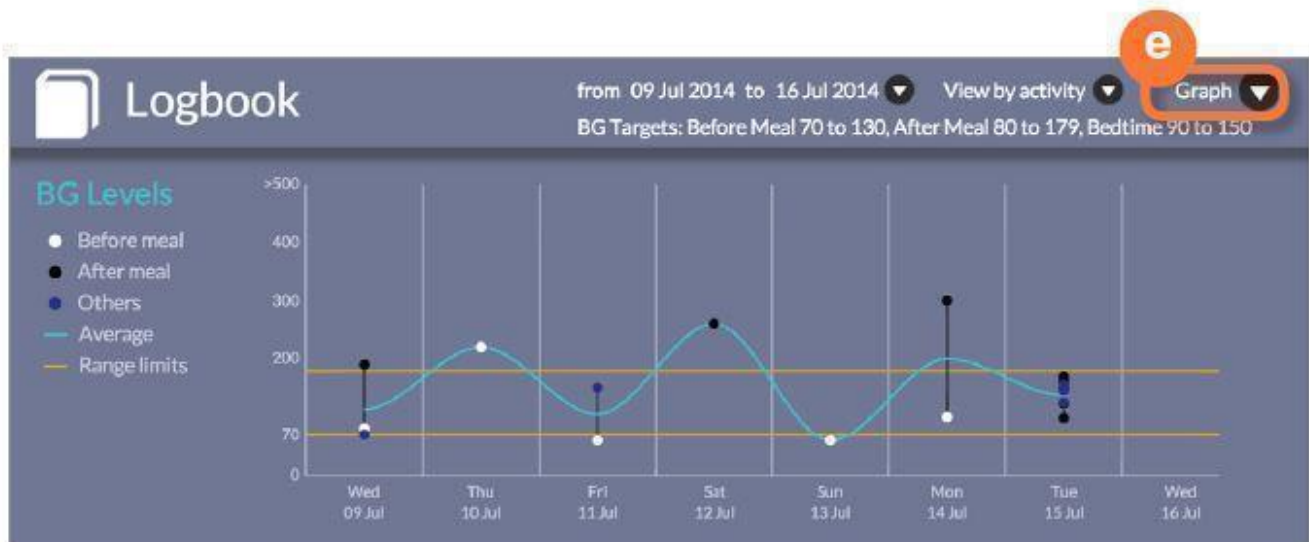

## Health Info (Medical History)

Use Health Info to record your important health information for BlueStar®, including:

- Medical History (other medical problems that can affect your diabetes management)
- Clinical Results (A1C, cholesterol, kidney function tests, etc.).
- Preventive Health (vaccinations, foot exam, eye exam, etc.).
- Lifestyle (height, weight, smoking history).

- Provider
- Prescription

**IMPORTANT SAFETY INFORMATION:**

**For medical questions, please contact your healthcare provider. If you are experiencing an emergency, please dial 911.**

a. Click any of these links to learn more about these diseases and conditions

The screenshot shows the BlueStar patient portal interface. At the top, there is a navigation bar with the BlueStar logo, user name 'Hello, Adam', and links for 'Help', 'Settings', and 'Logout'. Below this is a secondary navigation bar with icons for 'Home', 'Logbook', 'Medications', 'Health info', and 'Messages'. A 'New Entry' section shows a blood glucose reading: 'Last Entry: BG 220 mg/dL 30 Jun 2014 04:31 PM'. The main content area is titled 'Health Info' and has tabs for 'Medical History', 'Clinical Results', 'Preventive Health', 'Lifestyle', 'Provider', and 'Prescription'. The 'Medical History' tab is active, showing a progress bar at 40% completion. A message states: 'Your Medical History is 40% complete. Fill in the missing information and click Save.' Below this is a list of medical history questions with dropdown menus for answers. A red box highlights the first question, 'Year Diagnosed with Diabetes', and its dropdown menu, which is currently set to '2013'. A red circle with the letter 'a' is placed to the left of this question.

Question	Answer
Year Diagnosed with Diabetes	2013
High Blood Pressure	Yes
High Cholesterol	No
Cardiovascular Disease (CVD)	No
Family History of Premature CVD	Select
Peripheral Arterial Disease (PAD)	Select
Symptoms of Circulation Problems	Select

**To complete your Medical History:**

- Select year diagnosed with Diabetes
- Click Yes or No to answer each of the medical history questions

Note: If you're not sure of the answer to some of the questions, be sure to discuss it with your healthcare provider.

Be sure to update your Medical History if there are changes to any diseases or conditions you may have.

<b>a</b>	Year Diagnosed with Diabetes	2013 ▼
<b>b</b>	High Blood Pressure	Yes ▼
	High Cholesterol	No ▼
	Cardiovascular Disease (CVD)	No ▼
	Family History of Premature CVD	Select ▼
	Peripheral Arterial Disease (PAD)	Select ▼
	Symptoms of Circulation Problems	Select ▼
	Nephropathy (Kidney Problems)	Select ▼

### Health Info (Clinical Results)

BlueStar lets you know if your numbers are in range, out of range, or you are overdue for a lab test

#### To enter Clinical Results:

a. Enter the numbers and dates for your current lab results and blood pressure check

Remember to update your information each time you have new results. If you're not sure of the answers to some questions, be sure to discuss these with your healthcare provider.

#### IMPORTANT SAFETY INFORMATION:

**For medical questions, please contact your healthcare provider. If you are experiencing an emergency, please dial 911.**





# Health Info

Medical History

**Clinical Results**

Preventive Health

Lifestyle

Provider

Prescription

## Clinical Results

25%

Your Clinical Results are 25% complete. Fill in the missing information and click Save.

● In range  
● Overdue  
● Out of range

Your last update

a

Add a recent visit

A1C	● 6.7 %	● 01 Apr 2014	<input type="text"/> %	Choose date
Blood Pressure (BP)	● 139/79 mmHg	● 01 Jun 2014	<input type="text"/> / <input type="text"/> mmHg	Choose date
HDL	● Missing		<input type="text"/> mg/dL	Choose date
LDL	● Missing		<input type="text"/> mg/dL	Choose date
Triglycerides	● Missing		<input type="text"/> mg/dL	Choose date
Serum Creatinine	● Missing		<input type="text"/> mg/dL	Choose date

### Health Info (Preventive Health)

Preventive Health is a great tool for staying on track with exams and vaccinations that can help you manage your health. It lets you know if you are overdue for an exam or vaccination or missing an item on the list.

- Choose the date for each exam or vaccination you have had on the list  
Remember to update your list whenever you have had a new exam or vaccination

**Health Info**

Medical History Clinical Results **Preventive Health** Lifestyle Provider Prescription

### Preventive Health

38%

Your Preventive Health are 38% complete. Fill in the missing information and click Save.

**Your last update** **Add a recent visit**

Dilated Eye Exam	<span style="color: green;">●</span> 20 May 2014	<input type="checkbox"/>	
Comprehensive Foot Exam	<span style="color: green;">●</span> 02 Apr 2014	<input type="checkbox"/>	
ABI (Ankle-Brachial Index)	<span style="color: red;">●</span> Missing		Choose date ▼
Influenza Vaccine	<span style="color: red;">●</span> Missing		Choose date ▼
Pneumonia Vaccine	<span style="color: red;">●</span> Missing		Choose date ▼
Hepatitis B Vaccine Series	<span style="color: orange;">●</span> No		<input type="radio"/> Yes <input checked="" type="radio"/> No
Medical Nutritional Therapy (MNT)	<span style="color: red;">●</span> Missing		Choose date ▼
Diabetes Self-Management Education/Support (DSME/S)	<span style="color: red;">●</span> Missing		Choose date ▼

**a**

### Health Info (Lifestyle)

Lifestyle displays your weight, height, calculates your BMI, and your smoking history.

- Enter your current weight and choose date
- Enter height in feet and inches and choose date
- Your BMI is calculated automatically from your weight and height
- Click Tobacco Usage to enter your smoking history

**Health Info**

Medical History Clinical Results Preventive Health **Lifestyle** Provider Prescription

### Lifestyle

67%

Your Lifestyle info is 67% complete. Fill in the missing information and click Save.

**In range**  
**Overdue**  
**Out of range**

	Your last update	Add a recent visit
Weight	280 lbs 21 May 2014	<input type="text"/> lbs Choose date
Height	6ft 2in 05 Feb 2014	ft in Choose date
BMI	35.9	Calculated from height and weight
Tobacco Usage	Missing	Select

### Health Info (Provider)

This is where you can review your healthcare provider information and send a SMART Visit to your healthcare provider. To make changes to your healthcare provider information, you must call BlueStar® Customer Care at (888) 611-4794. Available: Monday-Friday, 8:00 a.m.-7:00 p.m. (ET).

#### What is a SMART Visit?

SMART Visit is a summary of the information you entered into BlueStar you can send to your healthcare provider. It includes your BGs, lab results, screening tests dates, vaccinations, and medications.

Send SMART Visit 2-3 days prior to an appointment so your provider will have the information to review with you during the visit.

a. Click the Smart Visit button to send your information to your provider.

BlueStar

Hello, Adam | Help | Settings | Logout

Home | Logbook | Medications | Health Info | Messages

New Entry  
Last Entry : BG 143 mg/dL  
15 Jul 2014 11:10 AM

## Health Info

Medical History | Clinical Results | Preventive Health | Lifestyle | **Provider** | Prescription

### Your Prescribing Provider

Dr. Janine Salsa MD MD

SuperDocs  
1 Health Way,  
Baltimore, Maryland,  
United States, 21212

Fax: 4105586751

To update your healthcare provider information, please call BlueStar Customer Care at **1-888-611-4794**, available Monday - Friday 8:00 am - 7:00 pm(ET).

### SMART Visit

Your provider has agreed to receive SMART Visit.

**a** **Send Report**

SMART Visit is a summary of BGs, lab results, dates of screening tests, vaccinations and medications that you entered into BlueStar. Send your SMART Visit report to your healthcare provider 2-3 days prior to your appointment.

## Health Info (Prescription Information)

Prescription Information lets you know when your BlueStar prescription will expire along with helpful information about refills.

BlueStar

Hello, Adam | Help | Settings | Logout

Home | Logbook | Medications | Health Info | Messages

New Entry  
Last Entry : BG 220 mg/dL  
30 Jun 2014 04:31 PM

## Health Info

Medical History | Clinical Results | Preventive Health | Lifestyle | Provider | **Prescription**

### Your BlueStar Prescription Details

BlueStar Prescription(Rx) Number: 100

If your BlueStar prescription expires or you need more refills, please contact the pharmacy at <xxx-xxx-xxxx>

Expires	30 Jun 2015
Last Refill	30 Jun 2014
Refills Prescribed	12
Refills Left	12

## Settings

Click on **SETTINGS** top right on any screen to make updates to any of your settings. Then select the tab for the screen you would like to update.

- **My Profile:** your email address

Note: No changes can be made after registration to basic profile information. If you have changed your name due to marriage, divorce or other legal means, please call Customer Care at (888) 611-4794.

Available: Monday-Friday, 8:00 a.m.-7:00 p.m. (ET).

- **Security and Password:** Password, Security Questions, Username

- **Daily Schedule:** Breakfast, Lunch, Dinner, Bedtime Schedules

- **Notifications:** Lets BlueStar know which notifications you would like to receive

The screenshot shows the BlueStar user interface. At the top right, the 'Settings' link is highlighted with an orange box. Below the navigation bar, the 'Settings' page is displayed with tabs for 'My Profile', 'Security and Password', 'Daily Schedule', and 'Notifications'. The 'My Profile' tab is selected, showing 'Your personal profile information' for Mr. Adam Palmer. The profile information includes contact details (address, phone number) and demographic information (ethnicity, race, marital status). A 'New Entry' notification is visible in the top right corner.

## Help

### HELP is where you can:

- Watch Virtual Training and BlueStar Welcome Video
- Download a PDF version of BlueStar User Guide
- Send a link to your phone to download BlueStar Diabetes Mobile app
- Get contact information to speak to a Customer Care representative

## Help

### BlueStar Virtual Training

- [Introduction](#)
- [Setup medication profile](#)
- [Enter diabetes data](#)
- [Get started](#)

### BlueStar Welcome Video

[Watch now](#)

### BlueStar User Guide



[Click to download.](#)

### About BlueStar

Version: 2.3

Build: 7252.98315

Caution: Federal law restricts this device to sale by or on the order of any licensed health care provider.

### BlueStar Diabetes Mobile App



[Send download link to your phone](#)

### Contact Us

"BlueStar™" Customer Care will be happy to help you with the following

- How to use BlueStar
- Reporting errors when trying to use BlueStar
- Technical questions related to BlueStar on mobile and/or Web
- Questions about your prescription or health insurance reimbursement

Phone: 1-888-611-4794

Available: Monday - Friday 8:00 am - 7:00 pm(ET)

## BlueStar Diabetes Mobile Prescription Therapy

### INDICATION FOR USE

BlueStar™ is indicated for use by healthcare providers and their adult patients, aged 21 years and older, who have type 2 diabetes. BlueStar is not intended to replace the care provided by a licensed healthcare professional, including prescriptions, diagnosis, or treatment. BlueStar is not indicated for people with type 1 diabetes, gestational diabetes or who use an insulin pump.

### PRODUCT DESCRIPTION

BlueStar is a mobile prescription therapy for adults with type 2 diabetes. BlueStar gives people living with type 2 diabetes real-time guidance to help improve the self-management of their diabetes. BlueStar also gives healthcare providers clinical decision support to help optimize patients' treatment regimens. BlueStar must be prescribed by a licensed healthcare provider.

### CAUTION

Federal law restricts this device to sale, distribution and use by or on the order of any licensed healthcare provider. BlueStar is for Single Patient Use Only. Allowing other people to enter their self-management data into your BlueStar could impact the ability for your physician to accurately assess and manage your diabetes. Before using BlueStar please read all instructions on incorporating your physician's treatment plan and entering your diabetes self-management data into BlueStar. Certified BlueStar Trainers are available to help you begin using BlueStar; to speak with a trainer call 1-888-611-4794. If you experience a medical emergency while using BlueStar, please contact emergency services (dial 911).

## **DOSAGE AND ADMINISTRATION**

The National Council for Prescription Drug Programs (NCPDP) has determined the Pack Size for BlueStar: 1 Each = 30 days.

## **GETTING STARTED**

If your physician has prescribed BlueStar or provided you with a sample, visit [www.bluestardiabetes.com](http://www.bluestardiabetes.com) to get started. Click **PATIENT LOGIN**. Then click, **SIGN UP** to start registering. Follow the instructions on the website to create your BlueStar account. You need an Access Code to finalize the account set-up. The Access Code will either be provided by your physician or BlueStar Customer Care. BlueStar is dispensed by a licensed pharmacist. You will need to speak with a pharmacist before accessing your BlueStar Prescription is filled.

## **ACCESSING BLUESTAR**

BlueStar is available on smartphones, computers and tablets. On smartphones it is recommended for use on devices with an iOS 7 or greater or Android OS 4.0 or greater. On the web it is recommended for use on computers with Internet Explorer 10 and 11, Mozilla Firefox 29 and 30 and Chrome 35.

## **IMPORTANT SAFETY INFORMATION**

For medical questions, please contact your healthcare provider. If you are experiencing an emergency, please dial 911. For complete instructions on how to use BlueStar, please refer the user guide available on the BlueStar web application under **HELP**. For more information visit [www.bluestardiabetes.com](http://www.bluestardiabetes.com) or call **BlueStar Customer Care at 1-888-611-4794**.

## **OUR COMMITMENT TO YOU**

Our goal is to provide you with quality health care products and dedicated customer service. If you are not fully satisfied with BlueStar, or if you have questions about the use of any WellDoc product, please call us at **1-888-611-4794**.

## **ABOUT BLUESTAR**

Powered by WellDoc, 1501 Saint Paul Street, Suite 118, Baltimore, MD. 21202. BlueStar (product) Version 2.3 released on 10/09/2014

BlueStar User Guide, DSM-161 Version 9 published on BlueStar web application on 10/09/2014

©2014 WellDoc, Inc. All Rights Reserved. WellDoc and the WellDoc logo are registered trademarks, and BlueStar, the BlueStar logo, and Automated Expert Analytic System are trademarks of WellDoc, Inc. BlueStar™ is manufactured by WellDoc, Inc., Baltimore, MD. iPhone is a registered trademark of Apple Inc.

Access to complete User Guide can be found under the **HELP** section on the web site at: [www.bluestardiabetes.com/Portal/Guest/Login.htm](http://www.bluestardiabetes.com/Portal/Guest/Login.htm).

**WELLD OC BLUESTAR**

accessories, pump, infusion

**Product Information**

<b>Product Type</b>	MEDICAL DEVICE	<b>Item Code (Source)</b>	NHRIC:89 129-100
---------------------	----------------	---------------------------	------------------

**Packaging**

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:89 129-100-01	1 in 1 PACKAGE		

**Marketing Information**

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
premarket notification	K141273	10/09/2014	

**Labeler** - WellDoc, Inc (021217115)**Registrant** - WellDoc, Inc (021217115)**Establishment**

Name	Address	ID/FEI	Business Operations
WellDoc, Inc		021217115	manufacture