

LIDOBAND- lidocaine ring
Alberta Veterinary Laboratories Ltd

Disclaimer: This drug has not been found by FDA to be safe and effective, and this labeling has not been approved by FDA. For further information about unapproved drugs, click [here](#).

LidoBand Castration Bands

LidoBand Castration Bands

Solvat

LidoBand Castration Bands

Each band contains 80 mg of lidocaine.

For castration of calves and lambs, and to deliver lidocaine as a local anesthetic. For use in calves under 250 pounds or lambs under 50 pounds.

FOR ANIMAL USE ONLY

KEEP OUT OF REACH OF CHILDREN

TAKE TIME OBSERVE LABEL DIRECTIONS

Description:

LidoBand™ is a latex rubber band designed to be placed around the base of the scrotum of calves and lambs for castration. Each castration band is infused with 80 mg of lidocaine, which provides local anesthesia to the application area beginning at 2 hours post-application and lasting up to 42 days.

Precautions:

Lidocaine is typically well tolerated. As with all local anesthetics, untoward effects may occur due to hypersensitivity.

LidoBand should not be used on animals with a known lidocaine hypersensitivity.

User Safety:

If skin comes in contact with LidoBand, wash affected area immediately with soap and water.

Storage:

Keep stored in a cool, dry place.

Do not re-use the band.

Directions for use: (8, 40, or 80-count box)

Each band is individually packaged in a foil-sealed blister. Do not touch the band with bare fingers. Wear gloves when applying LidoBand. Only apply LidoBand with a commercial band applicator (not included).

Step 1. Puncture the blister surface with the band applicator.

Step 2. Remove the band from packaging while loaded on the prongs of the applicator.

Step 3. Squeeze the applicator handle to spread the band.

Step 4. Pull the testicles into the scrotum, place the band above the testicles, and remove the applicator.

Directions for use: (250 count jar)

Bands contained within are to be used with a hopper chamber (not included). Do not touch the band with bare fingers. Wear gloves when applying LidoBand. Only apply LidoBand with a commercial band applicator (not included). Pour unused bands back into jar.

Step 1. Put bands into hopper chamber, and rotate to direct band into dispenser.

Step 2. Stick the prongs of the applicator into the band loop and remove from dispenser.

Step 3. Squeeze the applicator handle to spread the band.

Step 4. Pull the testicles into the scrotum, place the band above the testicles, and remove the applicator.

Product Label**LidoBand 40 Count Box**

<div>40 Castration Bands</div> <div><div>FOR ANIMAL USE ONLY</div><div>KEEP OUT OF REACH OF CHILDREN</div><div>CAUTION</div><div>Observe Label</div><div>TAKE TIME</div></div> <div><div>lido</div><div>band</div></div> <div>NDC 86174-101-03</div>			
<div><div>lido</div><div>band</div></div> <div><div>solvet</div></div> <div><div>Castration Bands</div><div>Each band contains 80 mg of lidocaine.</div><div>For castration of calves and lambs, and to deliver lidocaine as a local anesthetic. For use in calves under 250 pounds or lambs under 50 pounds.</div><div><div><div>Foils</div><div>LidoBand</div><div>Blister</div></div><div><div>TAKE TIME</div><div>Observe Label</div><div>DIRECTIONS</div></div></div><div><div>FOR ANIMAL USE ONLY</div><div>KEEP OUT OF REACH OF CHILDREN</div><div>NDC: 86174-101-03</div></div></div>	<div><div>lido</div><div>band</div></div> <div><div>solvet</div></div> <div><div>Castration Bands</div><div><div><div><div>Step 1. Puncture the blister surface with the band applicator.</div><div>Step 2. Remove the band from packaging while loaded on the prongs of the applicator.</div><div>Step 3. Squeeze the applicator handle to spread the band.</div><div>Step 4. Pull the testicles into the scrotum, place the band above the testicles, and remove the applicator.</div></div></div></div></div>	<div><div>lido</div><div>band</div></div> <div><div>solvet</div></div> <div><div>Castration Bands</div><div>For castration of calves and lambs, and to deliver lidocaine as a local anesthetic. For use in calves under 250 pounds or lambs under 50 pounds.</div><div><div>Description: LidoBand™ is a latex rubber band designed to be placed around the base of the scrotum of calves and lambs for castration. Each castration band is infused with 80 mg of lidocaine, which provides local anesthesia beginning at 2 hours post-application and lasting up to 42 days.</div><div><div>Directions for use:</div><div>Each band is individually packaged in a foil-sealed blister. Do not touch the band with bare fingers. Wear gloves when applying LidoBand. Only apply LidoBand with a commercial band applicator (not included).</div></div><div><div><div>California requires:</div><div>WARNING: Cancer Risk</div><div>www.P65Warnings.ca.gov</div></div><div><div>6</div><div>28224150119</div><div>8</div></div></div></div></div>	<div><div>lido</div><div>band</div></div> <div><div>solvet</div></div> <div><div>Castration Bands</div><div><div>Precautions: Lidocaine is typically well tolerated. As with all local anesthetics, untoward effects may occur due to hypersensitivity. LidoBand should not be used on animals with a known lidocaine hypersensitivity.</div><div>User Safety: If skin comes in contact with LidoBand, wash affected area immediately with soap and water.</div><div>Storage: Keep stored in a cool, dry place. Do not re-use the band.</div></div><div><div>Lot:</div><div>Exp:</div></div></div>
<div>40 Castration Bands</div>	<div><div>Manufactured by:</div><div>AVL/Solvet Ltd. 7226 107 Ave, Calgary, AB T2C 6N6</div><div>(877) 456-2766. www.solvet.ca</div></div>	<div><div>U.S. Patent #: 11,596,510</div><div>Solvet is a trademark of Alberta Veterinary Laboratories Ltd.</div></div>	<div><div>Manufactured by:</div><div>AVL/Solvet Ltd. 7226 107 Ave, Calgary, AB T2C 6N6</div><div>(877) 456-2766. www.solvet.ca</div></div>

LidoBand 80 Count Box

Compatible with existing band applicators.

Comes in easy-to-use blister

Bloodless castration

No injections necessary

Prevents wound fly strike

<250 lbs >50 lbs

U.S. Patent #: 11,596,510

www.lidoband.com

lidoband™ solvet

Castration Bands

Each band contains 80 mg of lidocaine.

For castration of calves and lambs, and to deliver lidocaine as a local anesthetic. For use in calves under 250 pounds or lambs under 50 pounds.

FOR ANIMAL USE ONLY

KEEP OUT OF REACH OF CHILDREN

NDC 86174L-101-04

8 Castration Bands

www.lidoband.com

Manufactured by:

AVL Solvet Ltd. 228, 107 Ave. Calgary, AB T2C 5J6

(877) 665-2755, www.solvet.ca

TAKE TIME

OBSERVE LABEL DIRECTIONS

Description: Lidoband™ is a thin rubber band designed to be placed around the base of the scrotum of calves and lambs for castration. Each castration band is infused with 80 mg of lidocaine, which provides local anesthesia beginning at 2 hours post-application and lasting 24 hours. For use: Each band is individually packaged in a foil-sealed blister. Do not touch the band with bare fingers. Wear gloves when applying Lidoband. Only apply Lidoband with a commercial band applicator (not included).

Precautions: Lidocaine is typically well tolerated. As with all local anesthetics, unwanted side effects may occur if the drug is absorbed systemically. Lidoband should not be used in animals with a known lidocaine hypersensitivity.

User Safety: If skin comes in contact with Lidoband, wash affected area immediately with soap and water.

Storage: Keep stored in a cool, dry place.

Do not re-use the band.

www.lidoband.com

6 28224 150192 1

Step 1: Puncture the blister surface with the applicator.

Step 2: Remove the band from packaging while the applicator handle is spread the band.

Step 3: Squeeze the applicator handle to spread the band.

Step 4: Pull the testicles into the scrotum, place the band around the testicles, and remove the applicator.

v1 Rev 01/25

LidoBand 8 Count Sample

lidoband™ solvet

Castration Bands

Each band contains 80 mg of lidocaine.

For castration of calves and lambs, and to deliver lidocaine as a local anesthetic. For use in calves under

TAKE TIME

OBSERVE LABEL DIRECTIONS

NOT FOR RETAIL SALE

DIRECTIONS FOR USE -

SEE INSIDE COVER

U.S. Patent #: 11,596,510

www.lidoband.com

U.S. Patent #: 11,596,510

www.lidoband.com

250 pounds or lambs under 50 pounds.

FOR ANIMAL USE ONLY

KEEP OUT OF REACH OF CHILDREN

NDC: 86174-101-04



8 Castration Bands

www.lidoband.com

Manufactured by :
AVL/Solvat Ltd. 7226 107 Ave, Calgary, AB T2C 5N6
(877) 456-2755. www.solvat.ca

Description: LidoBand™ is a latex rubber band designed to be placed around the base of the scrotum of calves and lambs for castration. Each castration band is infused with 80 mg of lidocaine, which provides local anesthesia beginning at 2 hours post-application and lasting up to 42 days.

Directions for use:

Each band is individually packaged in a foil-sealed blister. Do not touch the band with bare fingers. Wear gloves when applying LidoBand. Only apply LidoBand with a commercial band applicator (not included).

Precautions: Lidocaine is typically well tolerated. As with all local anesthetics, untoward effects may occur due to hypersensitivity. LidoBand should not be used on animals with a known lidocaine hypersensitivity.

User Safety: If skin comes in contact with LidoBand, wash affected area immediately with soap and water.


Storage: Keep stored in a cool, dry place.
Do not re-use the band.

www.lidoband.com




California requires:
WARNING: Cancer Risk
www.P65Warnings.ca.gov


lidoband™ solvet

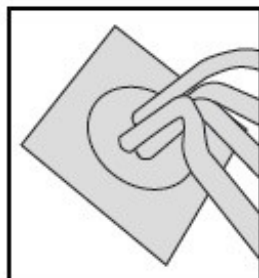


Step 1. Puncture the blister surface with the band applicator.

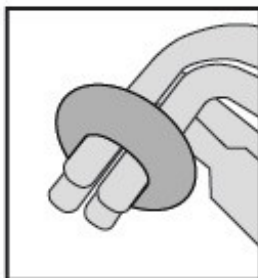

Step 2. Remove the band from packaging while loaded on the prongs of the applicator.

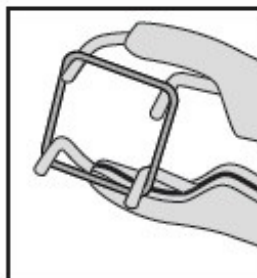

Step 3. Squeeze the applicator handle to spread the band.

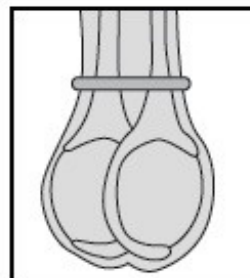

Step 4. Pull the testicles into the scrotum, place the band above the testicles, and remove the applicator.

LIDOBAND

lidocaine ring

Product Information

Product Type	OTC ANIMAL DRUG	Item Code (Source)	NDC:86174-101
Route of Administration	TOPICAL		

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
LIDOCAINE (UNII: 98PI200987) (LIDOCAINE - UNII:98PI200987)	LIDOCAINE	80 mg

Inactive Ingredients

Ingredient Name	Strength
ISOPROPYL MYRISTATE (UNII: 0RE8K4LNJS)	30 mg

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:86174-101-01	80 in 1 BOX		
2	NDC:86174-101-02	250 in 1 JAR		
3	NDC:86174-101-03	40 in 1 BOX		
4	NDC:86174-101-04	8 in 1 BOX		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
unapproved drug other		06/20/2023	

Labeler - Alberta Veterinary Laboratories Ltd (204293518)

Registrant - Alberta Veterinary Laboratories Ltd (204293518)

Establishment

Name	Address	ID/FEI	Business Operations
Alberta Veterinary Laboratories Ltd		204293518	manufacture

Establishment			
Name	Address	ID/FEI	Business Operations
Swati Spentose Private Limited		650621472	api manufacture

Revised: 2/2025

Alberta Veterinary Laboratories Ltd