RELIAMED - zinc oxide, calamine dressing

Reliamed

Disclaimer: Most OTC drugs are not reviewed and approved by FDA, however they may be marketed if they comply with applicable regulations and policies. FDA has not evaluated whether this product complies.

Drug Facts

Active Ingredients

Zinc Oxide 25%

Calamine 1%

Purpose

Skin Protectant

Uses

- Dries the oozing and weeping of poison ivy, poison oak, or poison sumac.
- Ideal for the managing extremities (e.g. varicose ulcers) requiring protection and support.

Warning

For external use only.

If condition worsens or does not improve within 7 days, consult a doctor.

Avoid Contact with eyes.

Keep this and all medicines out of children's reach.

Directions

- Begin bandaging at base of toes, without pressure, keeping foot and leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg (STEP 1).
- Complete bandaging to just below the knee (STEP 2).
- Apply an elastic adhesive bandage (STEP 3).

Other Ingredients

Acacia, Castor oil, Glycerin, and White Petrolatum.

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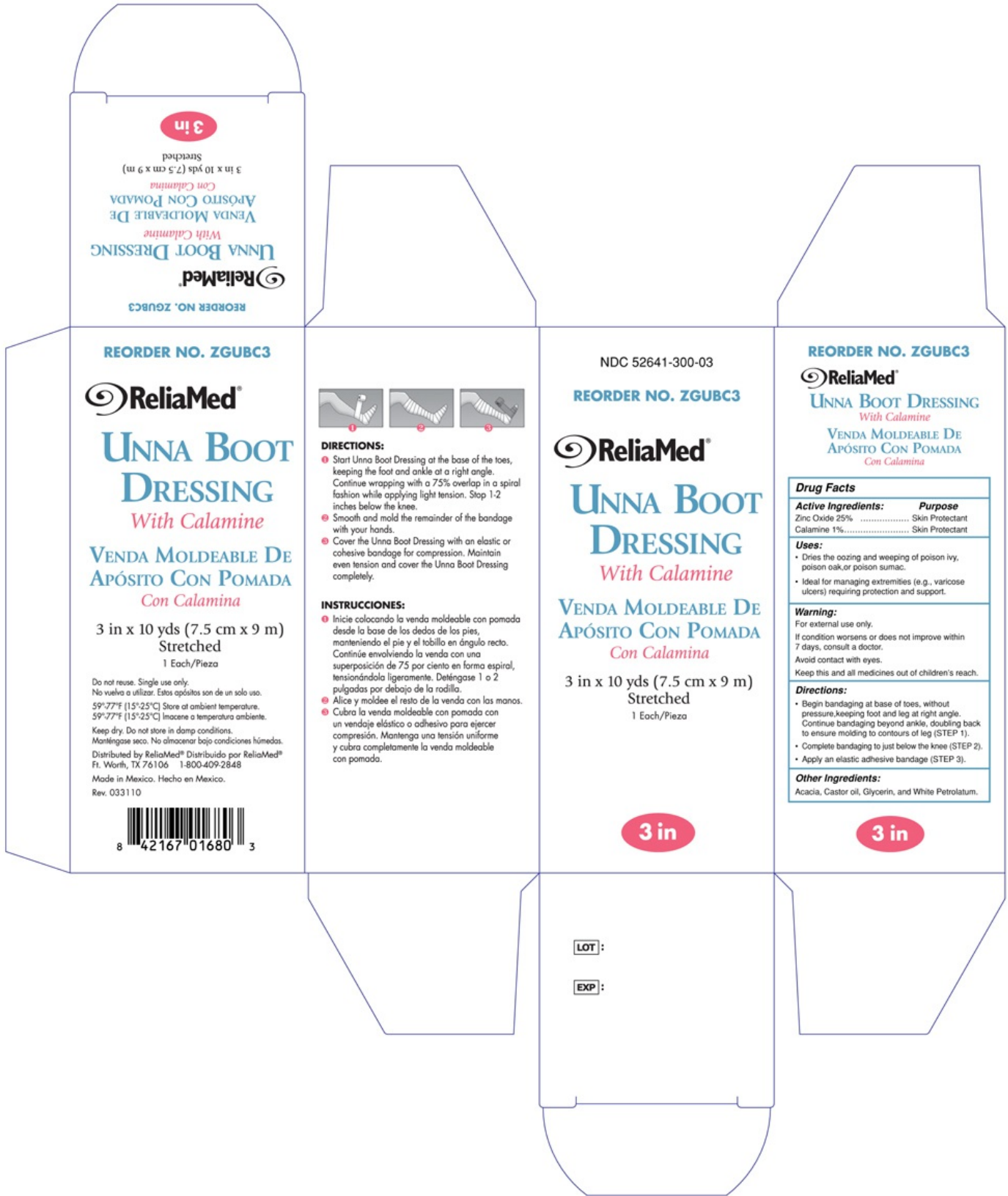
DIRECTIONS:

- 1 Start Unna Boot Dressing at the base of the toes, keeping the foot and ankle at a right angle. Continue wrapping with a 75% overlap in a spiral fashion while applying light tension. Stop 1-2 inches below the knee.
- 2 Smooth and mold the remainder of the bandage with your hands.
- 3 Cover the Unna Boot Dressing with an elastic or cohesive bandage for compression. Maintain even tension and cover the Unna Boot Dressing completely.

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Uses:

- Dries the oozing and weeping of poison ivy, poison oak or poison sumac.
- Ideal for managing extremities (e.g., varicose ulcers) requiring protection and support.

Warning:

For external use only.
If condition worsens or does not improve within 7 days, consult a doctor.
Avoid contact with eyes.
Keep this and all medicines out of children's reach.

Directions:

- Begin bandaging at base of toes, without pressure, keeping foot and leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg (STEP 1).
- Complete bandaging to just below the knee (STEP 2).
- Apply an elastic adhesive bandage (STEP 3).

Other Ingredients:

Acacia, Castor oil, Glycerin, and White Petrolatum.

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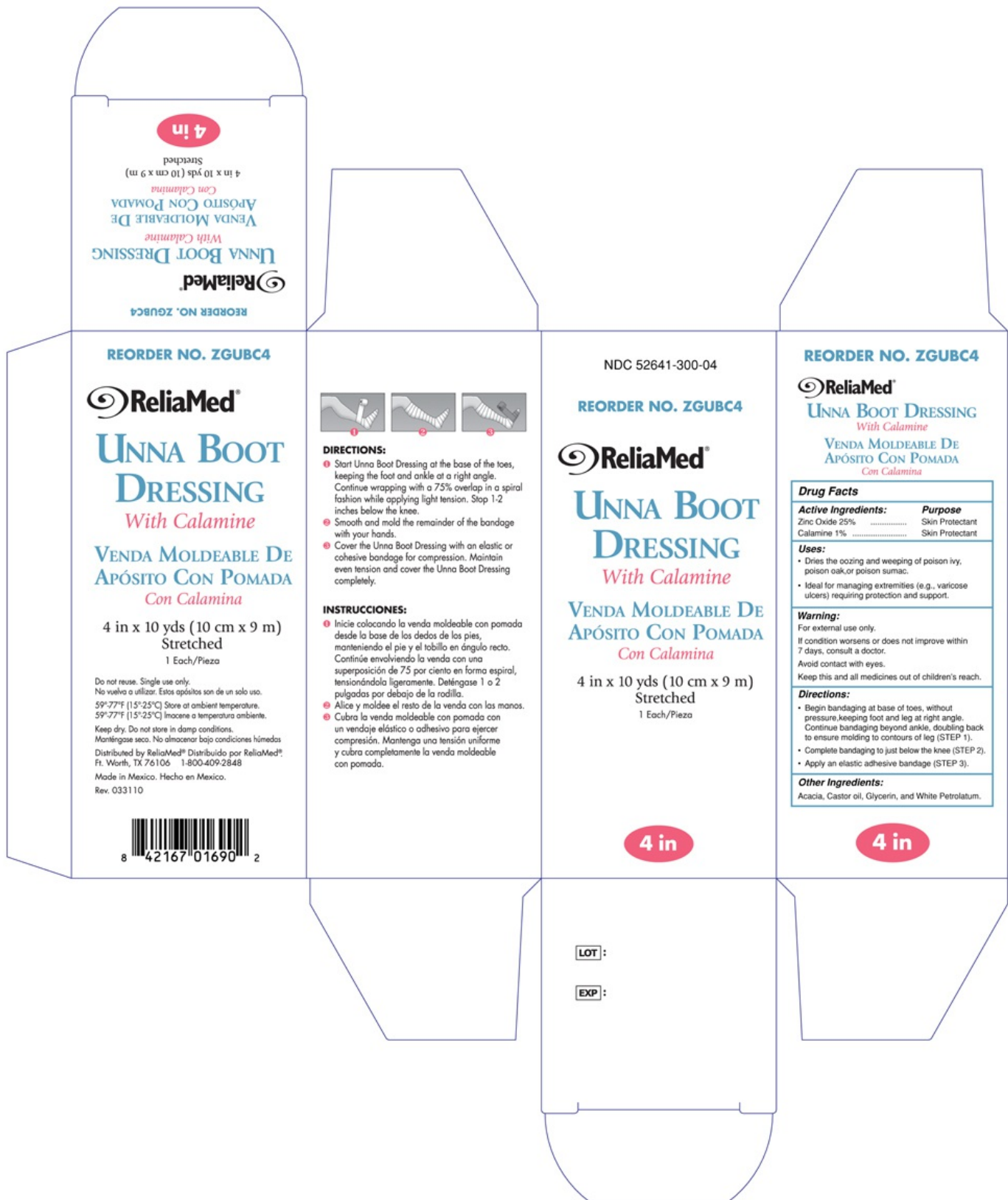
zinc oxide, calamine dressing

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:52641-300
Route of Administration	TOPICAL		

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
ZINC OXIDE (UNII: SOI2LOH54Z) (ZINC CATION - UNII:13S1S8SF37)	ZINC OXIDE	25 g
FERRIC OXIDE RED (UNII: 1K09F3G675) (FERRIC OXIDE RED - UNII:1K09F3G675)	FERRIC OXIDE RED	0.015 g
ZINC OXIDE (UNII: SOI2LOH54Z) (ZINC CATION - UNII:13S1S8SF37)	ZINC OXIDE	0.985 g

Inactive Ingredients

Ingredient Name	Strength
ACACIA (UNII: 5C5403N26O)	
CASTOR OIL (UNII: D5340Y2I9G)	
GLYCERIN (UNII: PDC6A3C0OX)	
PETROLATUM (UNII: 4T6H12BN9U)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:52641-300-03	1 in 1 BOX		
2	NDC:52641-300-04	1 in 1 BOX		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph final	part347	08/13/2010	

Labeler - Reliamed (049386139)