

**ADSOL RED CELL PRESERVATION- anticoagulant citrate phosphate dextrose (cpd) and adsol preservation
Fenwal, Inc.**

**Code 6R1783, 6R1787 (20x) Code 6R1790 (18x) Rx only
Fresenius Kabi**

Blood-Pack Unit with an Integral Fenwal Bioflex RC Leukocyte Reduction Filter Using CPD/Adsol Red Cell Preservation Solution for Whole Blood Collection and Filtration of Red Blood Cells and Fenwal HighFlo Needle

Contains Sample Diversion System for the collection of unanticoagulated whole blood samples for laboratory testing.

Store at Controlled Room Temperature. Protect from freezing. Avoid excessive heat.

Definition of "Controlled Room Temperature":

"A temperature maintained thermostatically that encompasses the usual and customary working environment of 20° to 25° C (68° to 77° F); that results in a mean kinetic temperature calculated to be not more than 25° C; and that allows for excursions between 15° C and 30° C (59° and 86° F) that are experienced in pharmacies, hospitals, and warehouses. Provided the mean kinetic temperature remains in the allowed range, transient spikes up to 40°C are permitted as long as they do not exceed 24 hours ... The mean kinetic temperature is a calculated value that may be used as an isothermal storage temperature that simulates the non isothermal effects of storage temperature variations."

Reference: United States Pharmacopeia, General Notices. United States Pharmacopeial Convention, Inc.

12601 Twinbrook Parkway, Rockville, MD.

Integral filter unit intended for leukocyte reduction of AS-1 red blood cells:

- At ambient temperature up to 8 hours after blood collection.
- At refrigerated temperature (1° to 6° C) up to 3 days after blood collection if AS-1 red blood cells are prepared within 3 days after whole blood collection.

The leukocyte reduced red blood cells may then be stored for up to 42 days after collection.

Instructions for Use

Collection Procedure:

Use aseptic technique.

Notes:

- If sample diversion system is not used, donor samples may be collected using an alternative method following standard procedures.
- Nominal tubing dimensions of product are 0.118" inner diameter x 0.025" wall thickness.

Precautions:

- **Upon removal of Blood-Pack unit from the clear plastic overwrap, visually inspect the unit.**
 - **Do not use the product if the in-line cannula is broken and/or anticoagulant is present in the sample pouch or in the tubing from the in-line cannula to the sample pouch and donor needle (see Figure 1). Note that condensation in the empty tubing of the Blood-Pack unit is expected as a result of the sterilization process.**
 - **Do not use unless the solutions are clear.**
1. Identify Blood-Pack unit using appropriate donor identification system.
 2. Adjust donor scale to desired collection weight and position primary container on the donor scale as far as possible below donor arm.
 3. Clamp donor tubing between Fenwal HighFlo1 needle and Y-junction with a clamp. (This step may be performed prior to step 1 or 2.)
 4. Visually inspect the tubing from the in-line cannula to the sample pouch and donor needle, as well as the sample pouch to reconfirm that there is no anticoagulant present.

Note: Ensure that the sample pouch remains below the donor's arm.

5. Following blood center procedures, apply pressure to donor's arm and disinfect site of venipuncture.
6. Remove needle cover per instructions below:
 - a) Holding the hub and cover near the tamper-evident seal, twist cover and hub in opposite directions to break seal.
 - b) Remove needle cover, being careful not to drag the cover across the needle point.
7. Following blood center procedures, perform venipuncture, appropriately secure donor needle and/or tubing and release clamp on donor tubing. When good blood flow is established, stabilize the front of the needle guard to arm with tape (see Figure 2).
8. Allow the sample pouch to fill with blood according to center procedure. Monitor blood flow into sample pouch.

Notes:

- The sample pouch contains an average fill volume of approximately 53 mL with a maximum fill volume of approximately 60 mL when filled to capacity.
- If less blood sample volume is required, the flow to the sample pouch may be stopped prior to completely filling the pouch. For example, in order to target a fill volume of approximately 40 mL, fill to the level indicated by the arrows in Figure 1. Ensure the pouch is hanging vertically.
- The tube leading from the Y-junction to the sample pouch contains an additional volume of approximately 2 mL.

Precautions:

- **Do not elevate or squeeze the sample pouch as this could cause blood**

to backflow from the sample

- **pouch into the collection system.**
 - **Once the sample pouch is filled to desired volume, complete steps 9 - 17 within approximately 4 minutes to avoid possible clot formation in the tubing and/or sample pouch.**
9. Close the blue clamp on tubing between the Y-junction and the sample pouch.
 10. Break the in-line cannula below the Y-junction in the donor tubing to the primary container allowing blood collection to proceed. To completely break the in-line cannula, grasp with both hands. Snap it at a 90° angle in one direction, and then bend it at a 90° angle in the opposite direction. Ensure the in-line cannula is completely broken and that the blood flows freely to the primary container.

Precaution: Failure to break the in-line cannula completely may result in restricted blood flow.

11. Mix blood and anticoagulant in the primary container immediately, at several intervals during collection, and immediately after collection.
12. Following blood center procedures, hermetically seal the tubing between the sampling site and the Y-junction to maintain sterility of the blood collection system prior to removing blood samples.

Warning:

- Do not proceed with the remaining steps until the tubing leading to the sample pouch is hermetically sealed between the sampling site and the Y-junction. To maintain the whole blood collection container as a closed system, the tubing between the sample pouch and Y-junction must be hermetically sealed prior to inserting the access device into the sampling site. Failure to do so may lead to contamination of the whole blood collection.
13. To collect samples, insert the access device by pushing firmly into the sampling site until the membrane seal is penetrated.

Note: If the access device is assembled such that the outer barrel is screwed onto the Luer, make sure to rotate clockwise upon insertion to avoid barrel detaching from Luer.

14. Open the cap on the access device (if applicable). Hold access device so that the sample pouch hangs down.
15. Directly align the vacuum sample tube with the internal needle in the access device. Insert vacuum sample tube into device until the stopper is punctured.
16. Allow vacuum sample tube to fill with blood then remove from the access device.
17. Repeat steps 15 and 16 until the desired number of vacuum sample tubes have been filled.

Notes:

- If the access device needs to be replaced, clamp the tubing between the sampling site and the sample pouch. Then grasp base of sampling site with one hand and pull the access device out with the other hand. Firmly insert the new access device. Remove clamp and continue sampling.
- If the access device is assembled such that the outer barrel is screwed onto the Luer, make sure to rotate clockwise upon removal to avoid barrel detaching from

Luer.

- The access device can only be replaced one time.

Precaution: When replacing access device, be careful to avoid contact with any blood droplets on the Luer or sampling site. Discard used access device appropriately.

18. Collect the appropriate volume based on Blood-Pack unit used.

Note: The volume of anticoagulant is sufficient for the blood collection indicated on Blood-Pack unit \pm 10%.

Precaution: Once the desired blood volume is collected, complete steps 19 - 23 within approximately

4 minutes to avoid possible clot formation in the tubing.

19. Release pressure on donor's arm. If appropriate, apply clamp to donor tubing between needle and the Y-junction.

20. Hermetically seal donor tubing between the in-line cannula and the primary container.

21. **Withdrawal of Needle** (see Figure 3).

Precaution: The needle guard must be held stationary while the needle is withdrawn into it.

- a) Place folded sterile gauze over puncture site and hold in place with finger tip without exerting pressure.
- b) Hold sides of needle guard near the front, between the index finger and thumb. Pull the hub back smoothly until the needle is completely enclosed and securely locked into the needle guard.
- c) Confirm the needle is completely enclosed and securely locked into the needle guard.

22. Remove and discard the sample diversion system and needle guard into an appropriate biohazardous waste container following established procedures. If donor tubing is also to be discarded, hermetically seal donor tubing directly above the primary container and remove.

Note: Step 23 may be performed prior to step 22 if desired.

23. If the donor tubing is not hermetically sealed directly above the primary container, then strip the blood from the remaining donor tubing into the primary container and mix. If desired, allow tubing to refill.

Component Preparation:

Notes:

- If a platelet concentrate is to be prepared, it should be separated within 8 hours after blood collection.
- Fresh frozen plasma should be separated from the red blood cells and placed in the

freezer at -18° C or colder within 8 hours after blood collection.

- Adsol red cell preservation solution should be added to the red blood cells immediately after the removal of plasma. Preparation of AS-1 red blood cells may vary depending on processing option selected:
 - a) Within 8 hours of blood collection if whole blood is held at ambient temperature.
 - b) Within 3 days of blood collection if whole blood is refrigerated.
24. At the appropriate time, prepare the Blood-Pack unit with integral filter for centrifugation.
 25. Thoroughly mix the primary container end over end.
 26. Load the Blood-Pack unit into a centrifuge cup per the instructions on page 3 or 4.

Page 3 describes Option A, the method of centrifuge cup loading when using standard oval centrifuge cups. Page 4 describes Option B, the method of centrifuge cup loading when using large oval centrifuge cups. This method differs from Option A in steps 7 - 9 only and shows the use of a blood bag insert to help fill extra space inside the cup.

Notes:

- This guide is one method for centrifuge cup loading and applies to all Bioflex RC configurations. The specific stacking order and methods may vary depending on the workstation setup, centrifuge equipment, and your facilities Standard Operating Procedures.
 - It is important to pack the filter properly in the centrifugation cup to avoid damage to the filter during centrifugation.
 - Do not place the filter in the lower half of the centrifuge cup. See step 3 on page 3 or 4 for proper placement of filter.
 - Do not press directly on the cannulae while performing any of the cup loading steps as this may cause preactivation of the cannulae.
27. After loading the Blood-Pack unit into the centrifuge cup, perform centrifugation according to center procedures.
 28. Following centrifugation, remove containers from the centrifuge cup taking care not to disturb the red blood cell/plasma interface.
 29. Place primary container in a plasma extractor and apply pressure. Clamp off tubing above the filter to prevent plasma flow into filter during plasma transfer. If applicable, clamp off tubing below Y-junction leading to secondary Transfer Pack container not to be filled with plasma. Open the cannula on the top of the primary container to transfer plasma into the empty Transfer Pack container.
 30. When the desired amount of plasma has been removed, clamp the tubing between the plasma container and the Y-connector closest to the plasma container and release pressure on the primary container. Seal and remove satellite container(s).
 31. Position the Adsol red cell preservation solution container ensuring that the primary container remains below the level of the filter during prime. Open the cannula on the Adsol solution container and remove the clamp between the primary container and the filter. Transfer the Adsol through the filter into the primary container. Clamp the tubing between the filter and the primary container after prime is complete.
 32. Mix the Adsol red cell preservation solution and red cells thoroughly.

Filtration Procedure:

Precaution: Red blood cell products collected from certain donors may have extended filtration times

and the potential for ineffective filtration and leukoreduction.

33. **Mix unfiltered AS-1 red blood cells thoroughly.** Invert the unfiltered AS-1 red blood cells and hang the filter set such that the filter remains vertical. Filtration may be performed from 165 cm up to full length. To achieve maximum flow rate, allow the set to hang to full length. The storage container must remain below the level of the filter during filtration.
34. Filtration must be initiated up to 8 hours after collection at ambient condition or up to 3 days at refrigerated temperature.
35. Inspect all tubing to ensure it hangs freely without kinks.
36. Remove the clamp above the filter to start filtration.

Note: Manual or mechanical pressure should not be used to increase the flow rate through the filter. Tubing below the filter should not be stripped at any time during the filtration process.

Note: If filtration of red cells is initiated at ambient temperature, the filtration process can be completed at either ambient or refrigerated temperature prior to storing the red cells between 1 and 6° C. However, for optimal filtration time, it is recommended to complete the filtration at ambient temperature.

37. When filtration is complete, air can be observed in the inlet side of the filter. Hermetically seal and detach the tubing below the filter.
38. Make segments from the post filter tubing and leave segments attached to the filtered red cell storage container. (QC samples may be prepared by thoroughly mixing the filtered red cells and stripping this tubing prior to sealing the segments.)
39. Store the AS-1 red blood cells, leukocytes reduced between 1 and 6° C.
40. Infuse the red cells within 42 days of collection.
41. Quality control should be per FDA "Guidance for Industry - Pre- Storage Leukocyte Reduction of Whole Blood and Blood Components Intended for Transfusion", current issue.

Warning: Failure to achieve closed system processing conditions negates the extended storage claim

and the red blood cell product must be transfused within 24 hours.

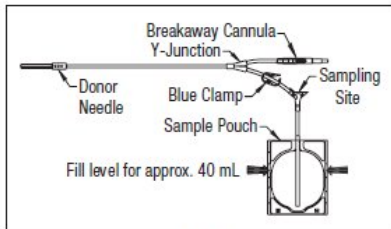



Figure 1



Figure 2



Figure 3

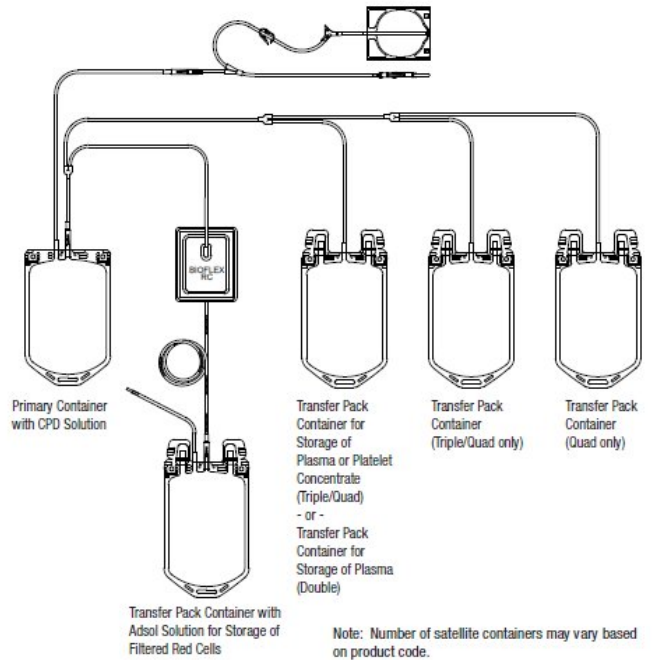

Representative Product Drawing

Symbols with Definitions:



Caution, consult instructions for use



Sterilized by steam. Sterile fluid path.



Non-pyrogenic fluid path



Do not reuse



Do not use if the product sterile barrier system is compromised



Do Not Vent



Lot



Code



Manufacturer



Manufacturing facility/Manufactured by



Expiration date

Rx only

For US Only. United States federal law restricts this device to sale by or on the order of a licensed health care practitioner.

Centrifuge Cup Loading Instructions


Option A: Standard Oval Cup Method



- 1** Place the Blood-Pack unit on a work surface and arrange all containers with the bottoms of the containers facing the same direction.



- 2** Ensure the Adsol container is label side up and the primary container is label side down. Place the filter directly on top of the Adsol container label with the filter port facing the bottom of the container.

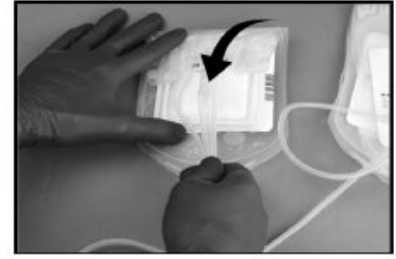

- 3** Fold the top of the Adsol container over the filter. This action will prevent tabs from interfering with the rotor arm and provide support for the filter.

Precaution: Do not place filter near the bottom of the container.



- 4** Position tubing on top of the filter.


Precaution: Do not press directly on the Adsol cannula as this may cause pre-activation.



- 5** Fold the top of the empty satellite container(s) over and place the container(s) on top of the tubing, label side facing up.




- 6** Place the containers, tubing, and filter on top of the primary container. Ensure that the back of the Adsol container is directly against the back of the primary container.

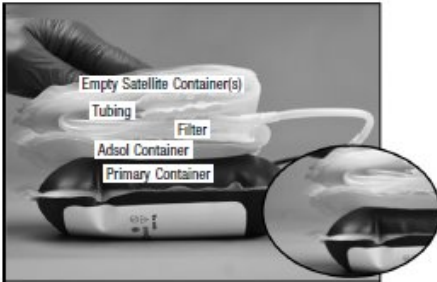

- 7** Ensure the containers are aligned at the bottom. The stacking order of the Blood-Pack unit should be as follows:

- Primary container
- Adsol container
- Filter
- Tubing
- Empty satellite container(s)

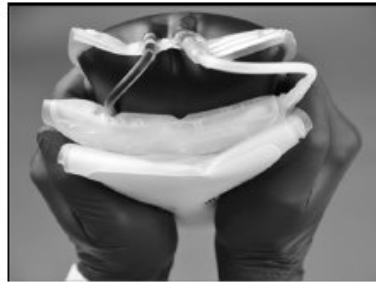

- 8** Holding the containers by the sides, squeeze firmly and slide into the centrifuge cup.

Precaution: Do not press directly on the cannulae when sliding into the centrifuge cup as this may cause pre-activation.



- 9** Ensure that the top edges of the folded satellite containers are level with or below the top of the centrifuge cup. Any excess tubing should be pushed down into the centrifuge cup.

Perform centrifugation according to center procedures.

Centrifuge Cup Loading Instructions


Option B: Large Oval Cup Method



- 1** Place the Blood-Pack unit on a work surface and arrange all containers with the bottoms of the containers facing the same direction.



- 2** Ensure the Adsol container is label side up and the primary container is label side down. Place the filter directly on top of the Adsol container label with the filter port facing the bottom of the container.

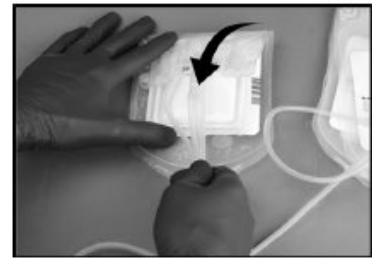


- 3** Fold the top of the Adsol container over the filter. This action will prevent tabs from interfering with the rotor arm and provide support for the filter.

Precaution: Do not place filter near the bottom of the container.



- 4** Position tubing on top of the filter.

Precaution: Do not press directly on the Adsol cannula as this may cause pre-activation.

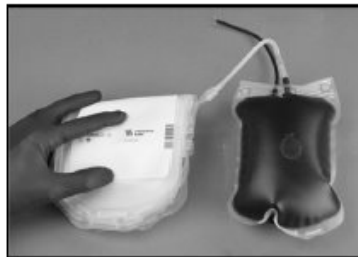

- 5** Fold the top of the empty satellite container(s) over and place the container(s) on top of the tubing, label side facing up.



- 6** Place the containers, tubing, and filter on top of the primary container. Ensure that the back of the Adsol container is directly against the back of the primary container.



- 7** Ensure the containers are aligned at the bottom. The stacking order of the Blood-Pack unit should be as follows:


- Primary container
- Adsol container
- Filter
- Tubing
- Empty satellite container(s)

Note: commercially available spacers (as shown above) or other methods may be followed to take up space in the cup.



- 8** Holding the containers by the sides, squeeze firmly and slide into the centrifuge cup.

Precaution: Do not press directly on the cannulae when sliding into the centrifuge cup as this may cause pre-activation.

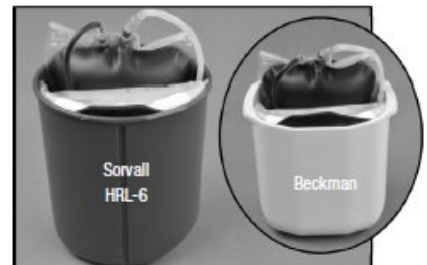

- 9** Ensure that the top edges of the folded satellite containers are level with or below the top of the centrifuge cup. Any excess tubing should be pushed down into the centrifuge cup.

Perform centrifugation according to center procedures.

¹ Van der Meer, P.F., & de Korte, D. "Increase of blood donation speed by optimizing the needle-to-tubing connection: an application of donation software." Vox Sanguinis 2009, 97:21-25.

Fresenius Kabi AG

61346 Bad Homburg / Germany

www.fresenius-kabi.com

1-800-933-6925

©2021 Fresenius Kabi AG. All rights reserved.

All trademarks referred to are property of their respective owners.

472315541 [A] 2021/09

PACKAGE LABEL.PRINCIPAL DISPLAY PANEL

To Open Tear Across at Notch

10x

Rx only

Code 6R1790

Fresenius Kabi

Anticoagulant Citrate Phosphate Dextrose Solution, USP (CPD) Blood-Pack Unit; Transfer-Pack Container with Adsol Red Cell Preservation Solution; Integral Fenwal Bioflex RC Red Cell Leukocyte Reduction Filter

For the Collection and Processing of 500 mL Blood, Sample Diversion System, 16 ga. Ultra Thin Wall

Fenwal HighFlo Needle






Store at Controlled Room Temperature.

Protect from freezing. Avoid excessive heat.

Each unit consists of a primary container with 70 mL of CPD solution containing 1.84 g Sodium

Citrate (dihydrate) USP, 1.78 g Dextrose (monohydrate) USP, 209 mg Citric Acid (anhydrous) USP, 155 mg Monobasic Sodium Phosphate (monohydrate) USP, pH may have been adjusted with sodium hydroxide; three empty 600 mL Transfer-Pack containers; one integral Fenwal Bioflex RC Red Cell Leukocyte reduction filter and one 600 mL Transfer-Pack container with 110 mL of Adsol Red Cell Preservation Solution containing 2.42 g Dextrose (monohydrate) USP, 990 mg Sodium Chloride USP, 825 mg Mannitol USP, 30 mg Adenine USP for the storage of AS-1 red blood cells, leukocytes reduced.

- Open pouch by tearing across at notch.
- Direct handling of product surfaces prior to extended storage in the **foil** pouch, may result in mold growth.
- Unused units in open **foil** pouch may be kept up to 30 days by folding and **securing** open end of **foil** pouch to prevent possible loss of moisture, provided:
 - I. Units are not removed from **foil** pouch, or
 - II. Unused units removed from **foil** pouch are returned to the **foil** pouch within 12 hours. Units may be removed from the pouch and returned only once.
- Units removed from the **foil** pouch (that are not returned to the pouch within 12 hours) must be used within 4 days (96 hours). Units out of the foil pouch for longer than 96 hours must be discarded.

-  Caution, consult instructions for use
-  Sterilized by steam. Sterile fluid path.
-  Non-pyrogenic fluid path
-  Do not reuse
-  Do not use if the product sterile barrier system is compromised

Manufacturer

Fresenius Kabi AG

61346 Bad Homburg / Germany

www.fresenius-kabi.com

Manufacturing facility /

Manufactured by

Fenwal International, Inc.

Road 357, KM 0.8

Maricao, Puerto Rico 00606

Made in US

472614282 REV: A

Rx only



NDC No. 0942 6518 02

9x

REF

Code 6R1790



Anticoagulant Citrate Phosphate Dextrose Solution, USP (CPD) Blood-Pack Unit; Transfer-Pack Container with Adsol Red Cell Preservation Solution; Integral Fenwal Bioflex RC Red Cell Leukocyte Reduction Filter






For the Collection and Processing of 500 mL Blood, Sample Diversion System, 16 ga. Ultra Thin Wall Fenwal HighFlo Needle

Store at Controlled Room Temperature. Protect from freezing. Avoid excessive heat.

Each unit consists of a primary container with 70 mL of CPD solution containing 1.84 g Sodium Citrate (dihydrate) USP, 1.78 g Dextrose (monohydrate) USP, 209 mg Citric Acid (anhydrous) USP, 155 mg Monobasic Sodium Phosphate (monohydrate) USP, pH may have been adjusted with sodium hydroxide; three empty 600 mL Transfer-Pack containers; one integral Fenwal Bioflex RC Red Cell Leukocyte reduction filter and one 600 mL Transfer-Pack container with 110 mL of Adsol Red Cell Preservation Solution containing 2.42 g Dextrose (monohydrate) USP, 990 mg Sodium Chloride USP, 825 mg Mannitol USP, 30 mg Adenine USP for the storage of AS-1 red blood cells, leukocytes reduced.

- Open pouch by tearing across at notch.
- Direct handling of product surfaces prior to extended storage in the **foil** pouch, may result in mold growth.

- Unused units in open **foil** pouch may be kept up to 30 days by folding and **securing** open end of **foil** pouch to prevent possible loss of moisture, provided:
 - I) Units are not removed from **foil** pouch, or
 - II) Unused units removed from **foil** pouch are returned to the foil pouch within 12 hours. Units may be removed from the pouch and returned only once.
- Units removed from the **foil** pouch (that are not returned to the pouch within 12 hours) must be used within 4 days (96 hours). Units out of the **foil** pouch for longer than 96 hours must be discarded.

-  Caution, consult instructions for use
-  **STERILE** Sterilized by steam. Sterile fluid path.
-  Non-pyrogenic fluid path
-  Do not reuse
-  Do not use if the product sterile barrier system is compromised

 **Manufacturer**
Fresenius Kabi AG
 61346 Bad Homburg / Germany
 www.fresenius-kabi.com

LOT

Exp


 **Manufacturing facility/**
Manufactured by
Fenwal International, Inc.
 Road 357, KM 0.8
 Maricao, Puerto Rico 00606

Made in US



(01)10309426518029

472614282 REV: A



ADSOL RED CELL PRESERVATION

anticoagulant citrate phosphate dextrose (cpd) and adsol preservation kit

Product Information

Product Type	HUMAN PRESCRIPTION DRUG	Item Code (Source)	NDC:0942-6518
---------------------	-------------------------	---------------------------	---------------

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:0942-6518-03	2 in 1 CARTON		
1	NDC:0942-6518-02	9 in 1 POUCH		
1	NDC:0942-6518-01	1 in 1 KIT; Type 1: Convenience Kit of Co-Package		

Quantity of Parts

Part #	Package Quantity	Total Product Quantity
Part 1	1 BAG	70 mL
Part 2	1 BAG	110 mL

Part 1 of 2

CPD

citrate phosphate dextrose solution

Product Information

Route of Administration	INTRAVENOUS
--------------------------------	-------------

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
TRISODIUM CITRATE DIHYDRATE (UNII: B22547B95K) (ANHYDROUS CITRIC ACID - UNII:XF417D3PSL)	ANHYDROUS CITRIC ACID	1.84 g in 70 mL
DEXTROSE MONOHYDRATE (UNII: LX22YL083G) (ANHYDROUS DEXTROSE - UNII:5SL0G7R0OK)	DEXTROSE MONOHYDRATE	1.78 g in 70 mL
ANHYDROUS CITRIC ACID (UNII: XF417D3PSL) (ANHYDROUS CITRIC ACID - UNII:XF417D3PSL)	ANHYDROUS CITRIC ACID	209 mg in 70 mL
SODIUM PHOSPHATE, MONOBASIC, MONOHYDRATE (UNII: 593YOG76RN) (PHOSPHATE ION - UNII:NK08V8K8HR)	SODIUM PHOSPHATE, MONOBASIC, MONOHYDRATE	155 mg in 70 mL

Inactive Ingredients

Ingredient Name	Strength
SODIUM HYDROXIDE (UNII: 55X04QC32I)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1		70 mL in 1 BAG; Type 1: Convenience Kit of Co-Package		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
NDA	BN811104	02/28/2017	

Part 2 of 2

ADSOL RED CELL PRESERVATION

adsol red cell preservation solution

Product Information

Route of Administration INTRAVENOUS

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
DEXTROSE MONOHYDRATE (UNII: LX22YL083G) (ANHYDROUS DEXTROSE - UNII:5SL0G7R00K)	DEXTROSE MONOHYDRATE	2.42 g in 110 mL
SODIUM CHLORIDE (UNII: 451W47IQ8X) (SODIUM CATION - UNII:LYR4M0NH37)	SODIUM CHLORIDE	990 mg in 110 mL
MANNITOL (UNII: 3OWL53L36A) (MANNITOL - UNII:3OWL53L36A)	MANNITOL	825 mg in 110 mL
ADENINE (UNII: JAC85A2161) (ADENINE - UNII:JAC85A2161)	ADENINE	30 mg in 110 mL

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1		110 mL in 1 BAG; Type 1: Convenience Kit of Co-Package		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
--------------------	--	----------------------	--------------------

NDA	BN811104	02/28/2017	
Marketing Information			
Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
NDA	BN811104	12/09/2021	

Labeler - Fenwal, Inc. (794519020)

Establishment

Name	Address	ID/FEI	Business Operations
Fenwal International, Inc.		091164590	MANUFACTURE(0942-6518) , ANALYSIS(0942-6518) , LABEL(0942-6518) , PACK(0942-6518) , STERILIZE(0942-6518)

Revised: 6/2022

Fenwal, Inc.