

**ALL-HEALTH WOUND CARE KIT- ethyl alcohol
ASO LLC**

ALL-HEALTH WOUND CARE KIT

Contents - All-Health Wound Care Kit

ALL-HEALTH®

WOUND CARE PRODUCTS

STEP 1 CLEAN

10 GAUZE PADS To clean wound, apply medication, or use a secondary dressing.
4 IN x 4 IN (101 mm x 101 mm)
(Made in China)

10 ANTISEPTIC WIPES Use to clean hands if soap and water are not available. Use to clean healthy skin around the wound site to prevent contamination.
4.75 IN x 7.75 IN (120 mm x 196 mm)
(Made in USA)

Gently clean the wound using a sterile Gauze Pad with mild soap and water. Carefully dry the affected area and apply medication if necessary (not included). Discard the used pad.

STEP 2 COVER

12 NON-STICK* PADS Primary dressing protects and absorbs for low to moderately draining wounds.
3 IN x 4 IN (76 mm x 101 mm)
(Made in China)

4 TRANSPARENT DRESSINGS Protect wound and/or sutures from moisture and environment.
2.36 IN x 2.75 IN (59 mm x 69 mm)
(Made in China)

Carefully cover the wound with a Non-Stick* Pad to help absorb fluid.

STEP 3 SECURE

3 ROLLED GAUZE Secure dressings for areas that bend or flex.
4 IN x 2.5 YDS (101 mm x 2.28 m)
(Made in China)

1 PAPER TAPE Secure dressings for areas that do not bend or flex.
1 IN x 10 YDS (25 mm x 9.14 m)
(Made in China)

Secure pad with Rolled Gauze, taping the ends if necessary.

*Will not adhere to sutures, staples, or adhesive wound closures. Ideal for low to moderately draining wound.

Antiseptic Wipes: Keep these and all drugs products out of the reach of children.

Transparent Dressings: Do not use as a primary dressing on moderately to heavily draining wounds.

Other Safety Information - All-Health Wound Care Kit

3-STEP

WOUND CARE KIT

FOR SMALL TO MEDIUM SIZED WOUNDS

**For wound sites ranging in size from
1 INCH - 3 INCH which may include:**

- Common Wounds
 - Lacerations
 - Abrasions
 - Minor burns
- Dressings may be cut or folded for specific wound size.

Not made with natural rubber latex.

Gauze Pads, Non-Stick Pads, Transparent Dressings, and Rolled Gauze are sterile unless individual wrapper has been opened or damaged.

Paper Tape and Antiseptic Wipes are non-sterile.

Gauze Pads and Rolled Gauze are Rayon-polyester bend.

Consult with your medical professional for specific treatment of wound site and proper dressing change procedures.

WARNING: In case of deep puncture wounds, serious burns, or signs of infection, consult a physician. Single use only.

Active ingredients - Antiseptic Wipes

Ethyl Alcohol 66.5%

Purpose - Antiseptic Wipes

Antiseptic

Uses - Antiseptic Wipes

- Antiseptic cleansing of face, hands and body to decrease bacteria on skin without soap and water

Warnings - Antiseptic Wipes

For external use only

Flammable: keep away from fire or flame

Do not use - Antiseptic Wipes

in the eyes. If this happens, rinse thoroughly with water.

Stop use and ask a doctor - Antiseptic Wipes

if irritation or redness develop and persists for more than 72 hours

Keep out of reach of children - Antiseptic Wipes

If swallowed get medical help or contact a Poison Control Center right away

Directions - Antiseptic Wipes

- tear open packet, remove towelette
- unfold and use as a washcloth
- supervise children under 6 years of age

Inactive ingredients - Antiseptic Wipes

aloe vera, fragrance, purified water, triethanolamine

Principal Display Panel - All-Health Wound Care Kit

ALL-HEALTH[®]

WOUND CARE PRODUCTS

3-STEP

WOUND CARE KIT

FOR SMALL TO MEDIUM SIZED WOUNDS

Essential items to support

3-STEP WOUND CARE:

CLEAN

COVER

SECURE

- For 1 IN to 3 IN wound sites
- 7 - 10 days of use

40 ITEMS

SEE BACK PANEL FOR KIT CONTENTS

ALL-HEALTH® is an ASO brand
 Distributed by ASO LLC
 Sarasota, FL 34240 | www.asocorp.com



Label

Principal Display Panel - Antiseptic Wipes

Safetec

**Antiseptic
Towelette**

For Professional and Hospital Use
 Contents: 1 single-use, premoistened towelette

Manufactured by **SAFETEC OF AMERICA, Inc.**
 Buffalo, NY 14215 800-456-7077 www.safetec.com



Antiseptic Towelette

For Professional and Hospital Use

Contents: 1 single-use, premoistened towelette

Manufactured by **SAFETEC OF AMERICA, Inc.**
Buffalo, NY 14215 800-456-7077 www.safetec.com

Drug Facts

Active ingredients	Purpose
Ethyl Alcohol 66.5%.....	Antiseptic

Uses ■ Antiseptic cleansing of face, hands and body to decrease bacteria on skin without soap and water

Warnings
For external use only
Flammable: keep away from fire or flame
Do not use in the eyes. If this happens, rinse thoroughly with water.

Stop use and ask a doctor if irritation or redness develop and persist for more than 72 hours

Keep out of reach of children If swallowed get medical help or contact a Poison Control Center right away

Directions ■ tear open packet, remove towelette
■ unfold and use as a washcloth
■ supervise children under 6 years of age

Inactive ingredients aloe vera, fragrance, purified water, triethanolamine

Label

ALL-HEALTH WOUND CARE KIT

ethyl alcohol kit

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:51142-003
---------------------	----------------	---------------------------	---------------

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:51142-003-01	1 in 1 CASE; Type 0: Not a Combination Product	08/21/2019	

Quantity of Parts

Part #	Package Quantity	Total Product Quantity
Part 1	10 PACKET	0.019 L

Part 1 of 1

ANTISEPTIC

alcohol cloth

Product Information

Item Code (Source)	NDC:61010-2017
Route of Administration	TOPICAL

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
ALCOHOL (UNII: 3K9958V90M) (ALCOHOL - UNII:3K9958V90M)	ALCOHOL	665 mL in 1 L

Inactive Ingredients

Ingredient Name	Strength
ALOE VERA LEAF (UNII: ZY81Z83H0X)	
WATER (UNII: 059QF0KO0R)	
TROLAMINE (UNII: 9O3K93S3TK)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:61010-2017-0	0.0019 L in 1 PACKET; Type 0: Not a Combination Product		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC Monograph Drug	505G(a)(3)	05/01/2017	

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC Monograph Drug	505G(a)(3)	08/21/2019	

Labeler - ASO LLC (152793493)