

ELECTROLYZED-OXIDIZING WATER DISINFECTANT- electrolyzed-oxidizing water disinfectant spray

Fuzhou Meimei Environmental Protection Technology Co., Ltd.

Disclaimer: This drug has not been found by FDA to be safe and effective, and this labeling has not been approved by FDA. For further information about unapproved drugs, click here.

Electrolyzed-oxidizing Water Disinfectant

ACTIVE INGREDIENT

Available chlorine 0.9%

PURPOSE

Disinfect

USES

High, medium and low level disinfection of medical devices and endoscopes; Skin and mucous membranes (Note: when

used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection;

sanitary and surgical hand disinfection; genitalia and sexual health instruments disinfection; surface of hard objects, fabrics

and other porous objects disinfection; drinking utensils, fruits and vegetables disinfection; Indoor air spray disinfection;

Warnings

Store in a cool, dry place; Avoid direct sunlight; Avoid eating by children, please keep out of reach of children; It is for external use only, not oral.

WHEN USING SECTION

1. Use the original solution directly and do not mix it with other disinfectants.

2. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.

DIRECTIONS

Spray, wipe, rinse, soak

OTHER INFORMATION

Physical indicators: Orp \geq 1100mv, pH value 2-3;

Store between 15-30C (59-86F); Avoid freezing and excessive heat above 40C (104F)

INACTIVE INGREDIENTS

Purified water

Package Label - Principal Display Panel

 <p>MEIMEI™ Electrolyzed oxidizing water</p> <p>DISINFECTANT</p> <p>Sterilization rate 99.999%</p> <p>MEIMEI IN HAND HEALTH ALL AVAILABLE</p> <ul style="list-style-type: none"> No residue No stimulation No side effects <p>1Floz/30ml</p>	<table border="1"> <tr> <th colspan="2">DRUG FACTS</th> </tr> <tr> <th>ACTIVE INGREDIENT</th> <th>PURPOSE</th> </tr> <tr> <td>Available chlorine 0.9%</td> <td>Disinfect</td> </tr> <tr> <td colspan="2">USES</td> </tr> <tr> <td colspan="2">Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.</td> </tr> <tr> <td colspan="2">Warnings</td> </tr> <tr> <td colspan="2">Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.</td> </tr> <tr> <td colspan="2">WHEN USING SECTION</td> </tr> <tr> <td colspan="2">1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.</td> </tr> <tr> <th>DIRECTIONS</th> <th>INACTIVE INGREDIENTS</th> </tr> <tr> <td>Spray and wipe</td> <td>Purified water</td> </tr> <tr> <td colspan="2">OTHER INFORMATION</td> </tr> <tr> <td colspan="2">Physical indicators: Orp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)</td> </tr> <tr> <td colspan="2">MANUFACTURING ENTERPRISE</td> </tr> <tr> <td colspan="2">Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province</td> </tr> <tr> <td colspan="2"> www.hzmmkj.com 400-8266-996 Made in China </td> </tr> <tr> <td colspan="2">  6 972694 570517 </td> </tr> </table>	DRUG FACTS		ACTIVE INGREDIENT	PURPOSE	Available chlorine 0.9%	Disinfect	USES		Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.		Warnings		Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.		WHEN USING SECTION		1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.		DIRECTIONS	INACTIVE INGREDIENTS	Spray and wipe	Purified water	OTHER INFORMATION		Physical indicators: Orp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)		MANUFACTURING ENTERPRISE		Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province		www.hzmmkj.com 400-8266-996 Made in China		 6 972694 570517	
DRUG FACTS																																			
ACTIVE INGREDIENT	PURPOSE																																		
Available chlorine 0.9%	Disinfect																																		
USES																																			
Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.																																			
Warnings																																			
Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.																																			
WHEN USING SECTION																																			
1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.																																			
DIRECTIONS	INACTIVE INGREDIENTS																																		
Spray and wipe	Purified water																																		
OTHER INFORMATION																																			
Physical indicators: Orp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)																																			
MANUFACTURING ENTERPRISE																																			
Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province																																			
www.hzmmkj.com 400-8266-996 Made in China																																			
 6 972694 570517																																			

 <p>MEIMEI™ Electrolyzed oxidizing water</p> <p>DISINFECTANT</p> <p>Sterilization rate 99.999%</p> <p>MEIMEI IN HAND HEALTH ALL AVAILABLE</p> <ul style="list-style-type: none"> No residue No stimulation No side effects <p>1.7Floz/50ml</p>	<table border="1"> <tr> <th colspan="2">DRUG FACTS</th> </tr> <tr> <th>ACTIVE INGREDIENT</th> <th>PURPOSE</th> </tr> <tr> <td>Available chlorine 0.9%</td> <td>Disinfect</td> </tr> <tr> <td colspan="2">USES</td> </tr> <tr> <td colspan="2">Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.</td> </tr> <tr> <td colspan="2">Warnings</td> </tr> <tr> <td colspan="2">Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.</td> </tr> <tr> <td colspan="2">WHEN USING SECTION</td> </tr> <tr> <td colspan="2">1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.</td> </tr> <tr> <th>DIRECTIONS</th> <th>INACTIVE INGREDIENTS</th> </tr> <tr> <td>Spray and wipe</td> <td>Purified water</td> </tr> <tr> <td colspan="2">OTHER INFORMATION</td> </tr> <tr> <td colspan="2">Physical indicators: Orp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)</td> </tr> <tr> <td colspan="2">MANUFACTURING ENTERPRISE</td> </tr> <tr> <td colspan="2">Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province</td> </tr> <tr> <td colspan="2"> www.hzmmkj.com 400-8266-996 Made in China </td> </tr> <tr> <td colspan="2">  6 972694 570500 </td> </tr> </table>	DRUG FACTS		ACTIVE INGREDIENT	PURPOSE	Available chlorine 0.9%	Disinfect	USES		Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.		Warnings		Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.		WHEN USING SECTION		1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.		DIRECTIONS	INACTIVE INGREDIENTS	Spray and wipe	Purified water	OTHER INFORMATION		Physical indicators: Orp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)		MANUFACTURING ENTERPRISE		Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province		www.hzmmkj.com 400-8266-996 Made in China		 6 972694 570500	
DRUG FACTS																																			
ACTIVE INGREDIENT	PURPOSE																																		
Available chlorine 0.9%	Disinfect																																		
USES																																			
Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.																																			
Warnings																																			
Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.																																			
WHEN USING SECTION																																			
1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.																																			
DIRECTIONS	INACTIVE INGREDIENTS																																		
Spray and wipe	Purified water																																		
OTHER INFORMATION																																			
Physical indicators: Orp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)																																			
MANUFACTURING ENTERPRISE																																			
Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province																																			
www.hzmmkj.com 400-8266-996 Made in China																																			
 6 972694 570500																																			



MEIMEI™ Electrolyzed oxidizing water

DISINFECTANT

**MEIMEI IN HAND
HEALTH ALL AVAILABLE**

Sterilization rate 99.999%

- No residue
- No stimulation
- No side effects

2.7Fl oz/80ml

DRUG FACTS	
ACTIVE INGREDIENT Available chlorine 0.9%	PURPOSE Disinfect
USES Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.	
Warnings Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.	
WHEN USING SECTION 1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.	
DIRECTIONS Spray and wipe	INACTIVE INGREDIENTS Purified water
OTHER INFORMATION Physical indicators: Orp \geq 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)	
MANUFACTURING ENTERPRISE Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province www.hzmmkj.com 400-8266-996 Made in China	



MEIMEI™ Electrolyzed oxidizing water

DISINFECTANT

**MEIMEI IN HAND
HEALTH ALL AVAILABLE**

Sterilization rate 99.999%

- No residue
- No stimulation
- No side effects

3.4Fl oz/100ml

DRUG FACTS	
ACTIVE INGREDIENT Available chlorine 0.9%	PURPOSE Disinfect
USES Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.	
Warnings Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.	
WHEN USING SECTION 1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.	
DIRECTIONS Spray and wipe	INACTIVE INGREDIENTS Purified water
OTHER INFORMATION Physical indicators: Orp \geq 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)	
MANUFACTURING ENTERPRISE Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province www.hzmmkj.com 400-8266-996 Made in China	



MEIMEI™ Electrolyzed oxidizing water

DISINFECTANT

**MEIMEI IN HAND
HEALTH ALL AVAILABLE**

Sterilization rate 99.999%

- No residue
- No stimulation
- No side effects

4Fl oz/120ml

DRUG FACTS	
ACTIVE INGREDIENT Available chlorine 0.9%	PURPOSE Disinfect
USES Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.	
Warnings Store in a cool, dry place Avoid direct sunlight; Avoid eating by children, please keep out of reach of children It is for external use only, not oral.	
WHEN USING SECTION 1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.	
DIRECTIONS Spray and wipe	INACTIVE INGREDIENTS Purified water
OTHER INFORMATION Physical indicators: Orp \geq 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)	
MANUFACTURING ENTERPRISE Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province www.hzmmkj.com 400-8266-996 Made in China	



MEIMEI™ Electrolyzed oxidizing water

DISINFECTANT



Sterilization rate 99.999%

- No residue
- No stimulation
- No side effects

5Floz/150ml

DRUG FACTS	
ACTIVE INGREDIENT Available chlorine 0.9%	PURPOSE Disinfect
USES Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection, sanitary and surgical hand disinfection, object surface disinfection, air spray disinfection.	
Warnings Store in a cool, dry place. Avoid direct sunlight. Avoid eating by children, please keep out of reach of children. It is for external use only, not oral.	
WHEN USING SECTION 1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.	
DIRECTIONS Spray and wipe.	INACTIVE INGREDIENTS Purified water
OTHER INFORMATION Physical indicators: Orp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)	
MANUFACTURING ENTERPRISE Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province www.hzmmkj.com 400-8266-996 Made in China	



MEIMEI™ Electrolyzed oxidizing water

DISINFECTANT



- No residue
- No stimulation
- No side effects

Sterilization rate 99.999%

10Floz/300ml

Sweep and Pay Attention to Health

美美科技
Meimei Technology

Skin

Wound

Air

Object surface

DRUG FACTS	
ACTIVE INGREDIENT Available chlorine 0.9%	PURPOSE Disinfect
USES Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection, sanitary and surgical hand disinfection, object surface disinfection, air spray disinfection.	
Warnings Store in a cool, dry place. Avoid direct sunlight. Avoid eating by children, please keep out of reach of children. It is for external use only, not oral.	
WHEN USING SECTION 1. spray the original solution on the part to be disinfected and let it work for 1-3 minutes. 2. Use the original solution directly and do not mix it with other disinfectants. 3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.	
DIRECTIONS Spray and wipe.	INACTIVE INGREDIENTS Purified water
OTHER INFORMATION Physical indicators: Orp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F) Avoid freezing and excessive heat above 40C (104F)	
MANUFACTURING ENTERPRISE Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province www.hzmmkj.com 400-8266-996 Made in China	



MEIMEI™ Electrolyzed oxidizing water
DISINFECTANT

**MEIMEI IN HAND
HEALTH ALL AVAILABLE**

- No residue
- No stimulation
- No side effects

Sterilization rate 99.999%

17Floz/500ml



Sweep and Pay Attention to Health



Skin Wound Air Object surface

DRUG FACTS

ACTIVE INGREDIENT Available chlorine 0.9%	PURPOSE Disinfect
---	-----------------------------

USES
Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment), wound disinfection; sanitary and surgical hand disinfection, object surface disinfection; air spray disinfection.

Warnings
Store in a cool, dry place. Avoid direct sunlight. Avoid eating by children, please keep out of reach of children. It is for external use only, not oral.

WHEN USING SECTION
1. Spray the original solution on the part to be disinfected and let it work for 1-3 minutes.
2. Use the original solution directly and do not mix it with other disinfectants.
3. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.

DIRECTIONS Spray and wipe	INACTIVE INGREDIENTS Purified water
-------------------------------------	---

OTHER INFORMATION
Physical indicators: Op ≥ 1100mv, ph value 2-3
Store between 15-30C (59-86F)
Avoid freezing and excessive heat above 40C (104F)

MANUFACTURING ENTERPRISE
Fuzhou Meimei Environmental Protection Technology Co., Ltd.
Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chaoqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province

www.hzmmkj.com
400-8266-996
Made in China



972694 570395



MEIMEI™ Electrolyzed oxidizing water
Disinfectant

Sterilization rate 99.999%

**MEIMEI IN HAND
HEALTH ALL AVAILABLE**

- No residue
- No stimulation
- No side effects

**68Floz
2000ml**



Sweep and Pay Attention to Health



DRUG FACTS

ACTIVE INGREDIENT Available chlorine 0.9%	INACTIVE INGREDIENTS Purified water	PURPOSE Disinfect	DIRECTIONS Spray/wipe/rinse/soak
---	---	-----------------------------	--

USES
High, medium and low level disinfection of medical devices and endoscopes; Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment); wound disinfection; sanitary and surgical hand disinfection; genitalia and sexual health instruments disinfection; surface of hard objects, fabrics and other porous objects disinfection; drinking utensils, fruits and vegetables disinfection; Indoor air spray disinfection;

Warnings
Store in a cool, dry place. Avoid direct sunlight. Avoid eating by children, please keep out of reach of children. It is for external use only, not oral.

WHEN USING SECTION
1. Use the original solution directly and do not mix it with other disinfectants.
2. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.

OTHER INFORMATION
Physical indicators: Op ≥ 1100mv, ph value 2-3
Store between 15-30C (59-86F). Avoid freezing and excessive heat above 40C (104F)

MANUFACTURING ENTERPRISE
Fuzhou Meimei Environmental Protection Technology Co., Ltd.
Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chaoqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province

Medical apparatus Skin Mucosa Air Object surface

www.hzmmkj.com
400-8266-996
Made in China



972694 570449



MEIMEI™ Electrolyzed oxidizing water Disinfectant

Sterilization rate 99.999%

MEIMEI IN HAND
HEALTH ALL AVAILABLE

- No residue
- No stimulation
- No side effects

169FloZ
5000ml



DRUG FACTS	ACTIVE INGREDIENT	INACTIVE INGREDIENTS	PURPOSE	DIRECTIONS
	Available chlorine 0.9%	Purified water	Disinfect	Spray/wipe/rinse/soak
USES	High, medium and low level disinfection of medical devices and endoscopes; Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment); wound disinfection; sanitary and surgical hand disinfection; genitalia and sexual health instruments disinfection; surface of hard objects, fabrics and other porous objects disinfection; drinking utensils, fruits and vegetables disinfection; indoor air spray disinfection;			
WARNINGS	Store in a cool/dry place; Avoid direct sunlight; Avoid eating by children, please keep out of reach of children; It is for external use only, not oral.			
WHEN USING SECTION	1. Use the original solution directly and do not mix it with other disinfectants. 2. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.			
OTHER INFORMATION	Physical Indicators: Clp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F); Avoid freezing and excessive heat above 40C (104F)			
MANUFACTURING ENTERPRISE	Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chigao Road, Xindian Town, Jinian District, Fuzhou City, Fujian Province			
	Medical apparatus	Skin	Mucosa	Air Object surface
	www.hzmmkj.com 400-8266-996		Made in China	
			972694 570401	



MEIMEI™ Electrolyzed oxidizing water Disinfectant

Sterilization rate 99.999%

MEIMEI IN HAND
HEALTH ALL AVAILABLE

- No residue
- No stimulation
- No side effects

338FloZ
10000ml



DRUG FACTS	ACTIVE INGREDIENT	INACTIVE INGREDIENTS	PURPOSE	DIRECTIONS
	Available chlorine 0.9%	Purified water	Disinfect	Spray/wipe/rinse/soak
USES	High, medium and low level disinfection of medical devices and endoscopes; Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment); wound disinfection; sanitary and surgical hand disinfection; genitalia and sexual health instruments disinfection; surface of hard objects, fabrics and other porous objects disinfection; drinking utensils, fruits and vegetables disinfection; indoor air spray disinfection;			
WARNINGS	Store in a cool/dry place; Avoid direct sunlight; Avoid eating by children, please keep out of reach of children; It is for external use only, not oral.			
WHEN USING SECTION	1. Use the original solution directly and do not mix it with other disinfectants. 2. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.			
OTHER INFORMATION	Physical Indicators: Clp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F); Avoid freezing and excessive heat above 40C (104F)			
MANUFACTURING ENTERPRISE	Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chigao Road, Xindian Town, Jinian District, Fuzhou City, Fujian Province			
	Medical apparatus	Skin	Mucosa	Air Object surface
	www.hzmmkj.com 400-8266-996		Made in China	
			972694 570456	



MEIMEI™ Electrolyzed oxidizing water Disinfectant

Sterilization rate 99.999%

MEIMEI IN HAND
HEALTH ALL AVAILABLE

- No residue
- No stimulation
- No side effects

676FloZ
20000ml



DRUG FACTS	ACTIVE INGREDIENT	INACTIVE INGREDIENTS	PURPOSE	DIRECTIONS
	Available chlorine 0.9%	Purified water	Disinfect	Spray/wipe/rinse/soak
USES	High, medium and low level disinfection of medical devices and endoscopes; Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment); wound disinfection; sanitary and surgical hand disinfection; genitalia and sexual health instruments disinfection; surface of hard objects, fabrics and other porous objects disinfection; drinking utensils, fruits and vegetables disinfection; indoor air spray disinfection;			
WARNINGS	Store in a cool/dry place; Avoid direct sunlight; Avoid eating by children, please keep out of reach of children; It is for external use only, not oral.			
WHEN USING SECTION	1. Use the original solution directly and do not mix it with other disinfectants. 2. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.			
OTHER INFORMATION	Physical Indicators: Clp ≥ 1100mv, ph value 2-3 Store between 15-30C (59-86F); Avoid freezing and excessive heat above 40C (104F)			
MANUFACTURING ENTERPRISE	Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address: 4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chigao Road, Xindian Town, Jinian District, Fuzhou City, Fujian Province			
	Medical apparatus	Skin	Mucosa	Air Object surface
	www.hzmmkj.com 400-8266-996		Made in China	
			972694 570463	



MEIMEI™ Electrolyzed oxidizing water Disinfectant

Sterilization rate 99.999%

MEIMEI IN HAND
HEALTH ALL AVAILABLE

- No residue
- No stimulation
- No side effects

1015Floz
30000ml



DRUG FACTS	ACTIVE INGREDIENT	INACTIVE INGREDIENTS	PURPOSE	DIRECTIONS
	Available chlorine 0.05%	Purified water	Disinfect	Spray,wipe,flush,soak
USES				
High, medium and low level disinfection of medical devices and endoscopes; Skin and mucous membranes (Note: when used for mucosal disinfection, it is limited to medical and health institutions for diagnosis and treatment); wound disinfection; sanitary and surgical hand disinfection; genitalia and sexual health instruments disinfection; surface of hard objects, fabrics and other porous objects disinfection; drinking utensils, fruits and vegetables disinfection; indoor air spray disinfection;				
WARNINGS				
Store in a cool,dry place; Avoid direct sunlight; Avoid eating by children,please keep out of reach of children; It is for external use only, not oral.				
WHEN USING SECTION				
1. Use the original solution directly and do not mix it with other disinfectants. 2. Organic substances may affect the sterilization efficiency of the product, and the disinfection effect is better after cleaning.				
OTHER INFORMATION				
Physical Indicators:Otp ≥ 1100mw, ph value 2-3 Store between 15-30C (59-86F); Avoid freezing and excessive heat above 40C (104F)				
MANUFACTURING ENTERPRISE				
Fuzhou Meimei Environmental Protection Technology Co., Ltd. Address:4th Floor, Building 3, Qingsheng Business Plaza, No. 539 Chiqiao Road, Xindian Town, Jin'an District, Fuzhou City, Fujian Province				
Medical apparatus Skin Mucosa Air Object surface		www.hzmmkj.com 400-8266-996 Made in China		

ELECTROLYZED-OXIDIZING WATER DISINFECTANT

electrolyzed-oxidizing water disinfectant spray

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:75430-001
Route of Administration	TOPICAL		

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
HYPOCHLOROUS ACID (UNII: 712K4CDC10) (HYPOCHLOROUS ACID - UNII:712K4CDC10)	HYPOCHLOROUS ACID	0.9 g in 100 mL

Inactive Ingredients

Ingredient Name	Strength
WATER (UNII: 059QF0KO0R)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:75430-001-01	30 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
2	NDC:75430-001-02	50 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
3	NDC:75430-001-03	80 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
4	NDC:75430-001-04	100 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
5	NDC:75430-001-05	120 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
6	NDC:75430-001-06	150 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	

7	NDC:75430-001-07	300 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
8	NDC:75430-001-08	500 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
9	NDC:75430-001-09	2000 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
10	NDC:75430-001-10	5000 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
11	NDC:75430-001-11	10000 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
12	NDC:75430-001-12	20000 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	
13	NDC:75430-001-13	30000 mL in 1 BOTTLE; Type 0: Not a Combination Product	05/06/2020	

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
unapproved drug other		05/06/2020	

Labeler - Fuzhou Meimei Environmental Protection Technology Co., Ltd. (560881277)

Registrant - Fuzhou Meimei Environmental Protection Technology Co., Ltd. (560881277)

Establishment

Name	Address	ID/FEI	Business Operations
Fuzhou Meimei Environmental Protection Technology Co., Ltd.		560881277	manufacture(75430-001)

Revised: 5/2020

Fuzhou Meimei Environmental Protection Technology Co., Ltd.