

**MEDI-PAK - zinc oxide, calamine dressing**  
**MCKESSON MEDICAL-SURGICAL**

*Disclaimer: Most OTC drugs are not reviewed and approved by FDA, however they may be marketed if they comply with applicable regulations and policies. FDA has not evaluated whether this product complies.*

-----

**Drug Facts**

**Active Ingredients**

Zinc Oxide 25%

Calamine 1%

**Purpose**

Skin Protectant

**Uses**

- Dries the oozing and weeping of poison ivy, poison oak, or poison sumac.
- Ideal for the managing extremities (e.g. varicose ulcers) requiring protection and support.

**Warnings**

For external use only.

If condition worsens or does not improve within 7 days, consult a doctor.

Avoid contact with eyes.

Keep this and all medicines out of children's reach.

**Directions**

- Begin bandaging at base of toes, without pressure, keeping foot and leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg (STEP 1).
- Complete bandaging to just below the knee (STEP 2).
- Apply an elastic adhesive bandage (STEP 3).

**Other Ingredients**

Acacia, Castor Oil, Glycerin, and White Petrolatum.

**Principal Display Panel**

NDC 68599-6110-3

**McKESSON**

Medi-Pak Performance

3" Unna Boot  
with Calamine

Non-Sterile • Single Use • Latex-Free

Stays Moist  
to Encourage  
Healing

**3 in. x 10 yds.**  
**(7.6 cm x 9.1 m)**

**1 Per Box**

**Reorder No.**  
**65-30C**

- Non-raveling gauze evenly impregnated with a non-hardening zinc oxide paste
- Contains no preservatives

Marketed by  
McKesson Corporation  
McKesson Medical-Surgical  
Richmond, VA 23228

Made in Mexico.

65\_30C

Prints:

1585 Orange



2945 Blue



**65-30C**  
Reorder No.

*Non-Sterile • Single Use • Latex-Free*

with Calamine

**3" Unna Boot**

Medi-Pak<sup>™</sup> Performance

**McKESSON**

NDC 68599-6110-3

**McKESSON**

Medi-Pak<sup>™</sup> Performance

**3" Unna Boot**

with Calamine

*Non-Sterile • Single Use • Latex-Free*

- Non-ravelling gauze evenly impregnated with a non-hardening zinc oxide paste
- Contains no preservatives

**3 in. x 10 yds.**  
**(7.6 cm x 9.1 m)**

1 Per Box

Reorder No.  
**65-30C**

Marketed By  
McKesson Corporation  
McKesson Medical-Surgical  
Richmond, VA 23228

Made In Mexico.

**McKESSON**

Medi-Pak<sup>™</sup> Performance

**3" Unna Boot**

with Calamine

*Non-Sterile • Single Use • Latex-Free*

**1** Begin bandaging at base of toes, without pressure, keeping foot and leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg.



**2** Complete bandaging to just below the knee.



**3** Apply an elastic adhesive bandage (not included).



NDC 68599-6110-3

**McKESSON**

Medi-Pak<sup>™</sup> Performance

**3" Unna Boot**

with Calamine

*Non-Sterile • Single Use • Latex-Free*

Stays Moist  
to Encourage  
Healing



**3 in. x 10 yds.**  
**(7.6 cm x 9.1 m)**

1 Per Box

Reorder No.  
**65-30C**

**McKESSON**

Medi-Pak<sup>™</sup> Performance

**3" Unna Boot**

with Calamine

*Non-Sterile • Single Use • Latex-Free*

Drug Facts	
<b>Active Ingredients:</b>	<b>Purpose</b>
Zinc Oxide 25%.....	Skin Protectant
Calamine 1%.....	Skin Protectant
<b>Uses:</b>	
• Dries the oozing and weeping of poison Ivy, poison oak, or poison sumac.	
• Ideal for managing extremities (e.g., varicose ulcers) requiring protection and support.	
<b>Warnings:</b>	
For external use only.	
If condition worsens or does not improve within 7 days, consult a doctor.	
Avoid contact with eyes.	
Keep this and all medicines out of children's reach.	
<b>Directions:</b>	
• Begin bandaging at base of toes, without pressure, keeping foot and leg at right angle.	
• Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg (STEP 1).	
• Complete bandaging to just below the knee (STEP 2).	
• Apply an elastic adhesive bandage (STEP 3).	
<b>Other Ingredients:</b>	
Acacia, Castor Oil, Glycerin, and White Petrolatum.	



(01)20612479135929  
LS 4275-0410

NDC 68599-6110-4

McKESSON

Medi-Pak Performance

4" Unna Boot  
with Calamine

Non-Sterile • Single Use • Latex-Free

Stays Moist  
to Encourage  
Healing

**4 in. x 10 yds.**  
**(10.2 cm x 9.1 m)**

**1 Per Box**

**Reorder No.**  
**65-40C**

- Non-raveling gauze evenly impregnated with a non-hardening zinc oxide paste
- Contains no preservatives

Marketed by  
McKesson Corporation  
McKesson Medical-Surgical  
Richmond, VA 23228

Made in Mexico.

65\_40C  
 Prints:  
 1585 Orange ■  
 2945 Blue ■



**MEDI-PAK**

zinc oxide, calamine dressing

**Product Information**

<b>Product Type</b>	HUMAN OTC DRUG	<b>Item Code (Source)</b>	NDC:68599-6110
<b>Route of Administration</b>	TOPICAL		

### Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
ZINC OXIDE (UNII: SOI2LOH54Z) (ZINC CATION - UNII:13S1S8SF37)	ZINC OXIDE	25 g
FERRIC OXIDE RED (UNII: 1K09F3G675) (FERRIC OXIDE RED - UNII:1K09F3G675)	FERRIC OXIDE RED	0.015 g
ZINC OXIDE (UNII: SOI2LOH54Z) (ZINC CATION - UNII:13S1S8SF37)	ZINC OXIDE	0.985 g

### Inactive Ingredients

Ingredient Name	Strength
ACACIA (UNII: 5C5403N26O)	
CASTOR OIL (UNII: D5340Y2I9G)	
GLYCERIN (UNII: PDC6A3C0OX)	
PETROLATUM (UNII: 4T6H12BN9U)	

### Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:68599-6110-3	1 in 1 BOX		
2	NDC:68599-6110-4	1 in 1 BOX		

### Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph final	part347	08/05/2010	

**Labeler** - MCKESSON MEDICAL-SURGICAL (023904428)

Revised: 10/2011

MCKESSON MEDICAL-SURGICAL