LIDOBAND TRI- lidocaine ring Alberta Veterinary Laboratories Ltd

Disclaimer: This drug has not been found by FDA to be safe and effective, and this labeling has not been approved by FDA. For further information about unapproved drugs, click here.

LidoBand Tri Castration Bands

LidoBand Tri Castration Bands

Solvet

LidoBand Tri Castration Bands

Each band contains 160 mg of lidocaine.

For castration of bull calves, and to deliver lidocaine as a local anesthetic. For use in bull calves between 200 to 400 pounds.

FOR ANIMAL USE ONLY

KEEP OUT OF REACH OF CHILDREN

TAKE TIME OBSERVE LABEL DIRECTIONS

Description:

LidoBand[™] - Tri is a latex rubber band designed to be placed around the base of the scrotum of calves for castration. Each castration band is infused with 160 mg of lidocaine. Lidocaine will quickly diffuse through the skin and remain in the local tissue due to vasoconstriction from the band.

Precautions:

Lidocaine is typically well tolerated. As with all local anesthetics, untoward effects may occur due to hypersensitivity.

LidoBand- Tri should not be used on animals with a known lidocaine hypersensitivity.

User Safety:

If skin comes in contact with LidoBand - Tri, wash affected area immediately with soap and water.

Storage:

Keep stored in a cool, dry place.

Do not leave bands in direct sunlight or allow exposure to high temperatures.

Do not re-use the band.

Directions for use:

Each band is individually packaged in a plastic blister. Do not touch the band with bare fingers. Wear gloves and eye protection when handling LidoBand - Tri or applying onto the triangle band applicator (not included).

LidoBand - Tri is compatible with the Tri-Band[™] bander. Because lidocaine is used in LidoBand - Tri, change gloves frequently when handling multiple bands. Follow the manufacturer's directions for the triangle band applicator when applying LidoBand - Tri.

For tetanus prevention it is recommended using a Clostridial vaccine with tetanus toxioid prior to or at the time of banding.

The properties of the latex changes with temperature. Very cold and very hot bands may break when expanded. Use bands at temperatures between 60°F and 85°F (15°C and 30°C).


Product Label:

LidoBand Tri Box of 25

LIDOBAND TRI

Product Information

lidocaine ring

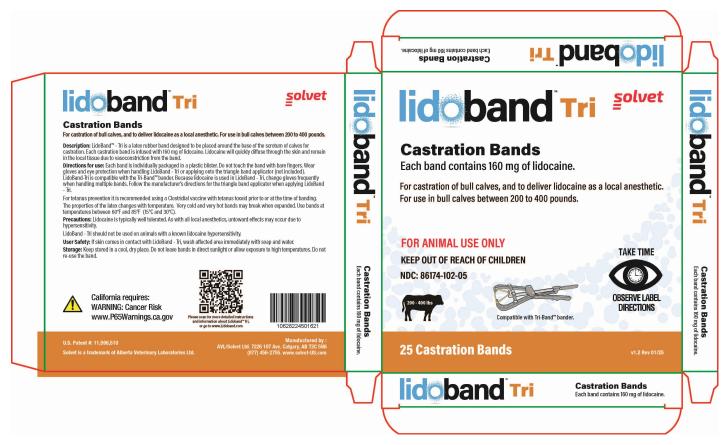

Product Type		OTC ANIMAL DRUG	Item Co	ode (Source)		NDC:86174-102			
Route of Administr	ation	TOPICAL							
Active Ingredient/Active Moiety									
	Ingred	ient Name		Basis of	Stren	gth Strength			
LIDOCAINE (UNII: 98PI200987) (LIDOCAINE - UNII:98PI200987) LIDOCAI						160 mg			
Inactive Ingredie	ents								
Ingredient Name						Strength			
ISOPROPYL MYRISTATE (UNII: 0RE8K4LNJS)					60 mg				
Packaging									
# Item Code	Packa	ge Description	Marketing	Start Date	Mark	ceting End Date			
1 NDC:86174-102-05	25 in 1 BC	DX							
Marketing In	format	ion							
	Applica	Application Number or Monograph N Citation		Marketing Start Date		Marketing End Date			
Marketing Category		Citation		Date		Date			
		Citation		02/10/2025		Date			

Labeler - Alberta Veterinary Laboratories Ltd (204293518)

Registrant - Alberta Veterinary Laboratories Ltd (204293518)

Establishment

Name	Address	ID/FEI	Business Operations
Alberta Veterinary Laboratories Ltd		204293518	manufacture

Establishment									
Name	Address	ID/FEI	Business Operations						
Swati Spentose Private Limited		650621472	api manufacture						

Revised: 2/2025

Alberta Veterinary Laboratories Ltd