

**HYCLODEX- dressing, wound, drug  
Marnel Pharmaceuticals, Inc.**

-----

**Hyclodex**

**INDICATIONS FOR USE:**

**Hyclodex™ Rx is intended to be used under the supervision of a health care professional for cleaning and removal of microorganisms, debris, and foreign material affecting various dermatologic skin conditions. Apply Hyclodex™ solution liberally on site to clean. Do not remove or wipe off. If use of dressing is required, apply liberally to dressing. No rinsing is required.**

DOES NOT CONTAIN ANTIBIOTICS, STEROIDS, PEROXIDE OR ALCOHOL.

**For Topical Use Only - Rx Only**

**PRODUCT SUMMARY:** Hyclodex™ is a proprietary skin solution containing balanced analytic hypochlorious acid as a preservative, which is known to be safe and fast acting in the removal of microorganisms of the skin.

**CONTRAINDICATIONS:** Check for known hypersensitivity to any of the Hyclodex™ solution ingredients.

**DIRECTIONS OF USE:** Spray Hyclodex™ on a clean gauze pad. Thoroughly apply to affected areas and allow product to fully dry on skin. Apply product 2 times per day or as directed.

**STORAGE:** Store at a controlled room temperature of 65° to 75° F. Keep away from direct sunlight and heat sources. Do Not Freeze.

**Handling and Disposal:** Hyclodex™ is non-irritating, non-cytotoxic and nonsensitizing to skin and eyes. Special handling or disposal precautions are not required.

**SAFETY:** Hyclodex™ is non-irritating and nonsensitizing having passed bio-compatibility studies including, ISO ocular irritation and ISO Maximum Sensitivity test.

**WARNINGS AND PRECAUTIONS:** For external use only. Do not use for infusion or injection. Do not swallow. If you experience redness, swelling, irritation or pain, discontinue use and contact your doctor. In case of sensitivities to chlorine, please exercise caution and consult with your doctor prior to use. Do not use expired product. Keep out of reach of children.

**HOW SUPPLIED:** Hyclodex™ is available in 59 ml spray solution.

**INDICATIONS FOR USE:**

Hyclodex™ Rx is intended to be used under the supervision of a health care professional for cleaning and removal of microorganisms, debris, and foreign material affecting various dermatologic skin conditions. Apply Hyclodex™ solution liberally on site to clean. Do not remove or wipe off. If use of dressing is required, apply liberally to dressing. No rinsing is required.

00682-0048-1

**Rx only**  
**Topical use only**

**Hyclodex™**  
(hypochlorous acid .012%)

**Spray  
Solution**

**DOES NOT CONTAIN ANTIBIOTICS,  
STEROIDS, PEROXIDE OR ALCOHOL**



0068200481

**CAUTION: FEDERAL LAW RESTRICTS  
THIS DEVICE TO SALE BY OR ON THE  
ORDER OF A LICENSED HEALTHCARE  
PRACTITIONER**

HYCLODEX™ Rx Manufactured For:  
Marnel Pharmaceuticals, LLC • Charleston, SC 29403  
843-405-0477 www.Marnelpharmaceuticals.com

NET WT. 2 FL. OZ. (59ml)

**PEEL HERE**

**HYCLODEX**

dressing, wound, drug

**Product Information**

<b>Product Type</b>	PRESCRIPTION MEDICAL DEVICE	<b>Item Code (Source)</b>	NHRIC:00682-0048
---------------------	-----------------------------	---------------------------	------------------

**Packaging**

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:00682-0048-1	59 mL in 1 BOTTLE, SPRAY; Type 0: Not a Combination Product		

**Marketing Information**

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
premarket notification	K113693	02/26/2018	

**Labeler** - Marnel Pharmaceuticals, Inc. (080161449)