

PRIMER - zinc oxide dressing
Derma Sciences Canada, Inc.

Disclaimer: Most OTC drugs are not reviewed and approved by FDA, however they may be marketed if they comply with applicable regulations and policies. FDA has not evaluated whether this product complies.

Drug Facts

Active Ingredients

Zinc Oxide 25%

Purpose

Skin Protectant

Uses

- Dries the oozing and weeping of poison ivy, poison oak, or poison sumac.
- Ideal for managing extremities (e.g., varicose ulcers) requiring protection and support.

Warning

For external use only.

If condition worsens or does not improve within 7 days, consult a doctor.

Avoid contact with eyes.

Keep Out of Reach of Children Section

Keep this and all medicines out of children's reach.

Directions

- Begin bandaging at base of toes, without pressure, keeping foot and leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg (STEP 1).
- Continue bandaging to just below the knees (STEP 2).
- Apply an elastic adhesive bandage (STEP 3).

Other Ingredients

Acacia, Castor oil, Glycerin, and White Petrolatum.

Principal Display Panel (Front and Back)

(FRONT SIDE NDC 64772-302-05)

NDC 64772-302-05

Primer ®

3" Modified Unna Boot

3" Venda Bota

Unna Modificada

3" x 10 yds.

(7.6 cm x 9.1 m)

Stays moist
to Encourage
Healing

Permanece
humeda para
alentar
la curacion

Catalog # R99903

DERMA SCIENCES

(FRONT SIDE NDC 64772-302-03)

NDC 64772-302-03

Primer ®

3" Modified Unna Boot

3" Venda Bota
Unna Modificada

3" x 10 yds.

(7.6 cm x 9.1 m)

Stays moist
to Encourage
Healing

Permanece
humeda para
alentar
la curacion

Catalog # GL300-1

DERMA SCIENCES

(BACK SIDE OF NDC 64772-302-03)

Primer ®

3" Modified Unna Boot

- Non - raveling guaze evenly impregnated with a non-hardening zinc oxide paste
- Contains no preservatives

3" Venda Unna Modificada

- Venda ge gasa que no se deshila impregnada uniformemente con pasta de oxido de zinc que no se endurece.

- No contiene conservadores.

Catalog # GL300-1

DERMA SCIENCES

Catalog # GL-300-1

3" Modified Unna Boot
3" Venda Bota Unna Modificada

Primer®

3" Modified Unna Boot

- Non-raveling gauze evenly impregnated with a non-hardening zinc oxide paste
- Contains no preservatives

3" Venda Bota Unna Modificada

- Venda de gasa que no se deshila impregnada uniformemente con pasta de óxido de zinc que no se endurece.
- No contiene conservadores

Catalog # GL-300-1

DERMA SCIENCES

NDC 64772-302-03

Primer®

3" Modified Unna Boot
3" Venda Bota Unna Modificada

3" x 10 yds.
(7.6 cm x 9.1 m)

Stays Moist to Encourage Healing

Permanece húmeda para alentar la curación

Catalog # GL-300-1

DERMA SCIENCES

Drug Facts	
Active Ingredients:	Purpose
Zinc Oxide 25%	Skin Protectant
Uses:	
<ul style="list-style-type: none"> • Dries the oozing and weeping of poison ivy, poison oak, or poison sumac. • Ideal for managing extremities (e.g., varicose ulcers) requiring protection and support. 	
Warning:	
For external use only. If condition worsens or does not improve within 7 days, consult a doctor. Avoid contact with eyes. Keep this and all medicines out of children's reach.	
Directions:	
<ul style="list-style-type: none"> • Begin bandaging at base of toes, without pressure, keeping foot and leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg (STEP 1). • Complete bandaging to just below the knee (STEP 2). • Apply an elastic adhesive bandage (STEP 3). 	
Other Ingredients:	
Acacia, Castor oil, Glycerin, and White Petrolatum.	

Made in Mexico for:
Derma Sciences, Inc.
104 Shorting Road
Toronto, Ontario M1S 3S4
In USA call: 1-800-445-7627
In CANADA call: 1-800-387-5302

Primer® is a registered trade of Derma Sciences, Inc.
Primer® es un nombre comercial registrado por Derma Sciences, Inc.

1



Begin bandaging with Primer® at base of toes, without pressure, keeping foot & leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg.

Empiece a vendar con Primer en la base de los dedos, sin ejercer presión, manteniendo el pie y la pierna en ángulo recto. Continúe vendando más allá del tobillo regresando el vendaje para asegurar seguir moldeando el contorno de la pierna.

2



Complete bandaging to just below the knee. Termine de vendar justo abajo de la rodilla.

3



Apply an elastic adhesive bandage. Aplique un vendaje elástico adhesivo.

Medicare Reimbursable
Costo Médico Reembolsable

40-1156-02



8 09958 00010 6

(FRONT SIDE NDC 64772-302-06)

NDC 64772-302-06

Primer ®

4" Modified Unna Boot

4" Venda Bota
Unna Modificada

4" x 10 yds.

(10.2 cm x 9.1 m)

Stays moist
to Encourage
Healing

Permanece
humeda para
alentar
la curacion

Catalog # R99904

DERMA SCIENCES

(FRONT SIDE NDC 64772-302-04)

NDC 64772-302-04

Primer ®

4" Modified Unna Boot

4" Venda Bota
Unna Modificada

4" x 10 yds.

(10.2 cm x 9.1 m)

Stays moist
to Encourage
Healing

Permanece
humeda para
alentar

la curacion

Catalog # GL-400-1

DERMA SCIENCES

(BACK SIDE NDC 64772-302-04)

Primer ®

4" Modified Unna Boot

- Non - raveling guaze evenly impregnated with a non-hardening zinc oxide paste
- Contains no preservatives

4" Venda Unna Modificada

- Venda ge gasa que no se deshila impregnada uniformemente con pasta de oxido de zinc que no se endurece.
- No contiene conservadores

Catalog # GL-400-1

DERMA SCIENCES

Catalog # GL-400-1

4" Modified Unna Boot
4" Venda Bota Unna Modificada

Primer®

Primer®

4" Modified Unna Boot

- Non-raveling gauze evenly impregnated with a non-hardening zinc oxide paste
- Contains no preservatives

4" Venda Bota Unna Modificada

- Venda de gasa que no se deshila impregnada uniformemente con pasta de óxido de zinc que no se endurece.
- No contiene conservadores

Catalog # GL-400-1



1



Begin bandaging with Primer® at base of toes, without pressure, keeping foot & leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg.

Empiece a vendar con Primer en la base de los dedos, sin ejercer presión, manteniendo el pie y la pierna en ángulo recto. Continúe vendando más allá del tobillo regresando el vendaje para asegurar seguir moldeando el contorno de la pierna.

2



Complete bandaging to just below the knee. Termine de vendar justo abajo de la rodilla.

3



Apply an elastic adhesive bandage. Aplique un vendaje elástico adhesivo.

Medicare Reimbursable
Costo Médico Reembolsable

NDC 64772-302-04

Primer®

4" Modified Unna Boot 4" Venda Bota Unna Modificada

4" x 10 yds.
(10.2 cm x 9.1 m)

*Stays Moist
to Encourage
Healing*



*Permanece
húmeda para
alentar
la curación*

Catalog # GL-400-1



Drug Facts	
Active Ingredients:	Purpose
Zinc Oxide 25%	Skin Protectant
Uses:	
<ul style="list-style-type: none"> • Dries the oozing and weeping of poison ivy, poison oak or poison sumac. • Ideal for managing extremities (e.g., varicose ulcers) requiring protection and support. 	
Warning:	
For external use only. If condition worsens or does not improve within 7 days, consult a doctor. Avoid contact with eyes. Keep this and all medicines out of children's reach.	
Directions:	
<ul style="list-style-type: none"> • Begin bandaging at base of toes, without pressure, keeping foot and leg at right angle. Continue bandaging beyond ankle, doubling back to ensure molding to contours of leg (STEP 1). • Complete bandaging to just below the knee (STEP 2). • Apply an elastic adhesive bandage (STEP 3). 	
Other Ingredients:	
Acacia, Castor oil, Glycerin, and White Petrolatum.	

Made in Mexico for:
Derma Sciences, Inc.
104 Shorting Road
Toronto, Ontario M1S 3S4

In USA call: 1-800-445-7627
In CANADA call: 1-800-387-5302

Primer® is a registered trade of Derma Sciences, Inc.
Primer® es un nombre comercial registrado por Derma Sciences, Inc.

40-1157-02



PRIMER

zinc oxide dressing

Product Information

Product Type	HUMAN OTC DRUG	Item Code (Source)	NDC:64772-302
Route of Administration	TOPICAL		

Active Ingredient/Active Moiety

Ingredient Name	Basis of Strength	Strength
ZINC OXIDE (UNII: SOI2LOH54Z) (ZINC CATION - UNII:13S1S8SF37)	ZINC OXIDE	25 g

Inactive Ingredients

Ingredient Name	Strength
ACACIA (UNII: 5C5403N26O)	
CASTOR OIL (UNII: D5340Y2I9G)	
GLYCERIN (UNII: PDC6A3C0OX)	
PETROLATUM (UNII: 4T6H12BN9U)	

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NDC:64772-302-03	1 in 1 BOX		
2	NDC:64772-302-05	60 in 1 BOX		
3	NDC:64772-302-04	1 in 1 BOX		
4	NDC:64772-302-06	60 in 1 BOX		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
OTC monograph final	part347	05/10/2010	

Labeler - Derma Sciences Canada, Inc. (200564891)

Registrant - LABORATORIOS LE ROY, S.A. DE C.V. (588387394)

Establishment

Name	Address	ID/FEI	Business Operations
LABORATORIOS LE ROY, S.A. DE C.V.		588387394	manufacture