

**SCARCIN ROLL-ON- elastomer, silicone, for scar management liquid
SA3, LLC**

**SCARCIN ROLL-ON
SA3, LLC**

**SCARCIN ROLL-ON
Rx only
For external use only
Not for ophthalmic use**

DESCRIPTION

SCARCIN ROLL-ON is intended for use in the management of closed hyperproliferative scars, both old and new hypertrophic or keloid scars resulting from burns, surgical procedures or trauma wounds. It helps reduce redness, softens and flattens raised scars, relieves itching, discomfort and pain associated with scars, helps prevent excessive and abnormal scar formation, and for use as mono-therapy or in combination with other scar therapies including pressure garments.

SCARCIN ROLL-ON is a self-drying, flexible, gas permeable, waterproof silicone liquid mixture that is colorless and odorless. **SCARCIN ROLL-ON** forms a bond with the stratum corneum (the outer layer of dead skin cells) forming a protective barrier against chemical physical and microbial invasion of the scar site while assisting with hydration.

SCARCIN ROLL-ON is also for use on children or people with sensitive skin. Silicone is internationally recommended as the first line of treatment in scar management.

INGREDIENTS

SCARCIN ROLL-ON contains: Dimethicone, Dimethiconol, Cyclopentasiloxane, Cyclotetrasiloxane, Silicon, Stearyl Alcohol

CLINICAL PHARMACOLOGY

The exact mechanism of action in improving the appearance of scar tissue from using silicone remains unknown. However, various suggestions have been made to explain the efficacy of silicone, including hydration, pressure, temperature, oxygen transmission and silicone absorption. There is some evidence that the treatment affects the stratum corneum and, by reducing evaporation, restores better homeostasis in the tissue. In keloid and hypertrophic scarring, the stratum corneum allows more evaporation of water from the underlying tissue than occurs in normal skin. Silicone may prevent this, keeping the stratum corneum in optimal hydration and protecting the skin from environmental hazards, both of which can reduce abnormal scarring. The liquid may also affect the stratum corneum by inhibiting mast cell activity, diminishing edema, vasodilatation and excessive extracellular matrix formation but the simple changes in temperature, pressure, oxygen tension and hydration produced by wound coverage probably constitute the main mechanism of action. Another hypothesis is that the effect of static electricity on silicone may influence the alignment of collagen deposition.

INDICATIONS AND USES

For use in the management of closed hyperproliferative (hypertrophic or keloid) scars, resulting from

general surgical procedures, trauma, wounds, and burns.

CONTRAINDICATIONS

SCARCIN ROLL-ON is contraindicated in patients with known hypersensitivity to silicone or any of the listed ingredients.

WARNINGS

For external use only. Avoid direct contact with eyes, lips or mucous membranes. Do not apply on areas of broken skin. Do not apply on third degree burns and open wounds. Never use on sutured wound until sutures have been removed. Do not use on dermatological conditions that disrupt the integrity of the skin.

PRECAUTIONS

Stop use and ask a doctor if irritation develops. In rare instances, silicone liquid may cause a rash on the skin. This condition may result from improper cleansing of the scar area where the silicone gel has been applied. If this product is applied properly and skin irritation still occurs, discontinue use and consult your physician. If ingested, get medical help or contact Poison Control Center right away.

KEEP THIS AND ALL MEDICATION OUT OF THE REACH OF CHILDREN.

This medication should be used as directed by your physician during pregnancy or while breastfeeding. Consult your doctor about the risks and benefits.

CAUTION: Federal law restricts this device to sale by or on the order of a physician.

ADVERSE REACTIONS

On rare occasions, the silicone liquid may cause temporary redness, stinging, burning or irritation and normally disappear when the medication is discontinued.

DOSAGE AND ADMINISTRATION

1. Ensure that the affected area is clean and dry.
2. Apply Scarcin ROLL-ON to the area as a very thin coat and allow to dry.
3. Apply Scarcin ROLL-ON twice a day
4. Once dry, Scarcin ROLL-ON can be covered with cosmetics or sunscreen.
5. Recommended duration of treatment is 60-90 days.

How is SCARCIN ROLL-ON Supplied

SCARCIN ROLL-ON is supplied in:

3 oz Roll-on bottle

NDC: 69420-8003-1

Store at 20°-25°C (68° to 77°F); Keep away from heat and protect from freezing. [See USP Controlled Room Temperature.]


Manufactured for:

SA3, LLC

Los Angeles, CA 90064

PRINCIPAL DISPLAY PANEL

NDC 69420-8003-1
 Scarcin Roll-on
 Rx ONLY
 3 oz (90mL)

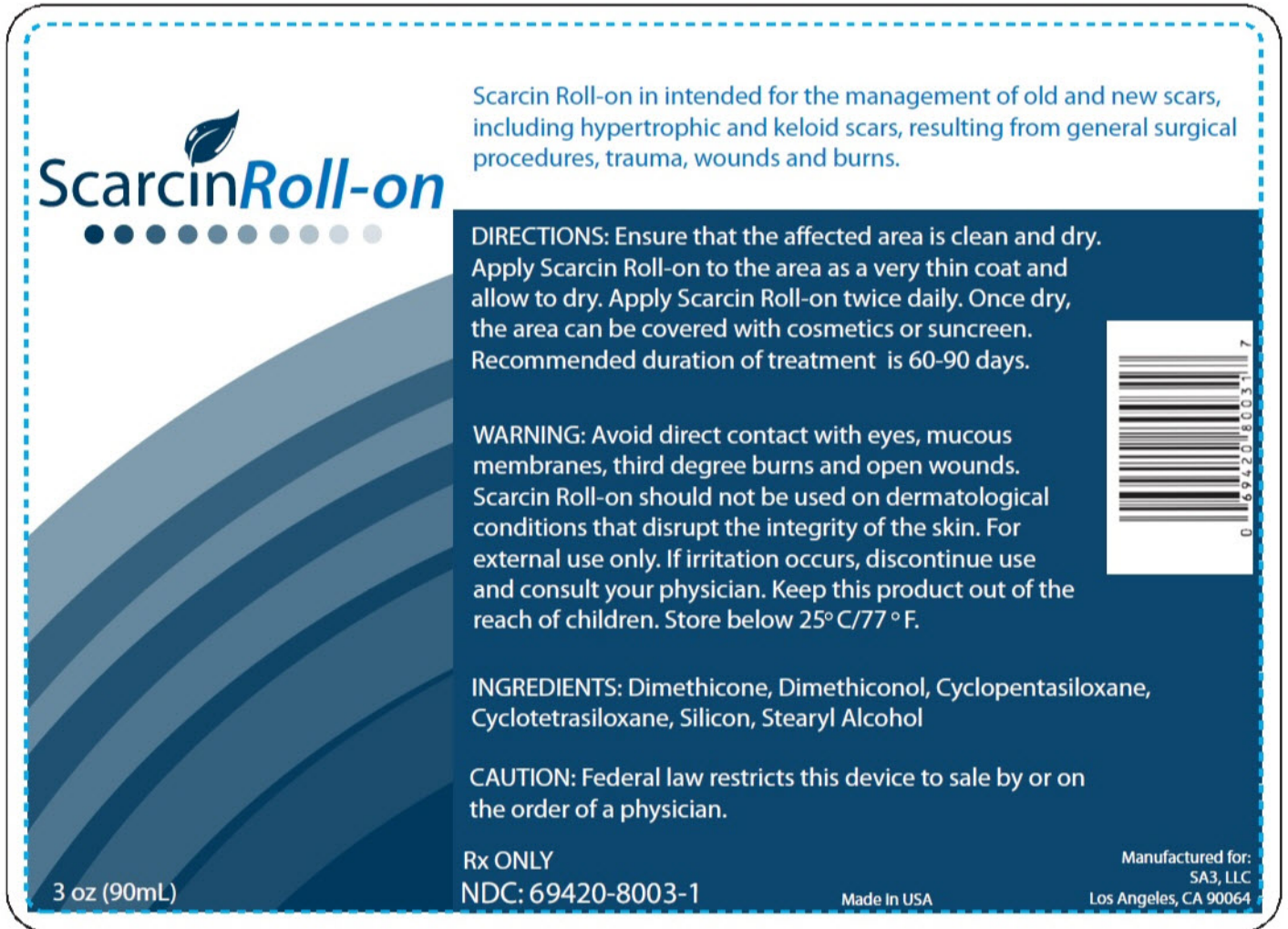

ScarcinRoll-on

Scarcin Roll-on is intended for the management of old and new scars, including hypertrophic and keloid scars, resulting from general surgical procedures, trauma, wounds and burns.

DIRECTIONS: Ensure that the affected area is clean and dry. Apply Scarcin Roll-on to the area as a very thin coat and allow to dry. Apply Scarcin Roll-on twice daily. Once dry, the area can be covered with cosmetics or sunscreen. Recommended duration of treatment is 60-90 days.

WARNING: Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. Scarcin Roll-on should not be used on dermatological conditions that disrupt the integrity of the skin. For external use only. If irritation occurs, discontinue use and consult your physician. Keep this product out of the reach of children. Store below 25°C/77°F.

INGREDIENTS: Dimethicone, Dimethiconol, Cyclopentasiloxane, Cyclotetrasiloxane, Silicon, Stearyl Alcohol

CAUTION: Federal law restricts this device to sale by or on the order of a physician.

Rx ONLY
 NDC: 69420-8003-1

3 oz (90mL)

Made In USA

Manufactured for:
 SA3, LLC
 Los Angeles, CA 90064

SCARCIN ROLL-ON

elastomer, silicone, for scar management liquid

Product Information

Product Type	MEDICAL DEVICE	Item Code (Source)	NHRIC:69420-8003
Route of Administration	TOPICAL		

Inactive Ingredients

Ingredient Name	Strength
DIMETHICONE (UNII: 92RU3N3Y1O)	
DIMETHICONOL (2000 CST) (UNII: T74O12AN6Y)	
CYCLOMETHICONE 5 (UNII: 0THT5PC10R)	
CYCLOMETHICONE 4 (UNII: CZ227117JE)	
SILICON (UNII: Z4152N8IU)	

STEARYL ALCOHOL (UNII: 2KR89I4H1Y)

Packaging

#	Item Code	Package Description	Marketing Start Date	Marketing End Date
1	NHRIC:69420-8003-1	90 in 1 BOTTLE, WITH APPLICATOR; Type 0: Not a Combination Product		

Marketing Information

Marketing Category	Application Number or Monograph Citation	Marketing Start Date	Marketing End Date
EXEMPT DEVICE	MDA	08/01/2018	

Labeler - SA3, LLC (079627454)

Revised: 8/2018

SA3, LLC